

If	Try this
<p>The receiver is not activated when there is an incoming call or message.</p>	<ul style="list-style-type: none"> <li>• Make sure that the sensor is placed on the screen of your device. Also make sure that the screen lights up.</li> <li>• Ensure that all cables are properly connected.</li> <li>• Check the radio link between the transmitter and receiver by pressing the Test-button on the transmitter.</li> </ul>
<p>The receiver is activated even if there is no incoming call or message.</p>	<ul style="list-style-type: none"> <li>• Make sure that the sensor is standing upright on a flat surface</li> </ul>

Patent pending

Visit transmitter and receiver sold separately

**Technical information**

Optical detection	Activated when the display lights up
Light sensitivity	Visible light >3 lux
Placement	Horizontally on the display
Ergonomic design	Ideal for people with impaired vision/dexterity
Low center of gravity	Resumes an upright position if it's knocked over
Connector	2.5 mm teleplug
Weight	20 g
Visit compatible	Integrates mobile devices with the Visit system

# Mobile phone sensor BE9251

[bellman.com](http://bellman.com)

[bellman.com](http://bellman.com)

© 2013, Bellman & Symfon AB, all rights reserved. Bellman, Bellman & Symfon and the Bellman & Symfon logo are registered trademarks of Bellman & Symfon AB.

**Asia Pacific office**  
Bellman & Symfon Asia Ltd.  
1115-16, Main Tower, GD Int'l Hotel  
339 Huanshi Dong Rd  
Guangzhou, China, 510095  
Phone +86 20 8318 2950  
Fax +86 20 8333 5528  
E-mail [info.cn@bellman.com](mailto:info.cn@bellman.com)

**European office**  
Bellman & Symfon Europe AB  
Södra Långebergsgratan 30  
421 32 Västra Frölunda  
Sweden  
Phone +46 31 68 28 20  
Fax +46 31 68 28 90  
E-mail [info@bellman.com](mailto:info@bellman.com)

BE9251\_001MAN002


# 1

- GB Connect
- DE Anschliessen
- FR Connectez
- NL Aansluiten

- SE Anslut
- NO Tilkobling
- DK Tilslutning
- FI Liitä

- ES Conectar
- IT Collegare
- PT Ligar
- GR Συνδέω

- PL Podłączyć
- CZ Připojit
- HU Telepítés
- SRB Povezati

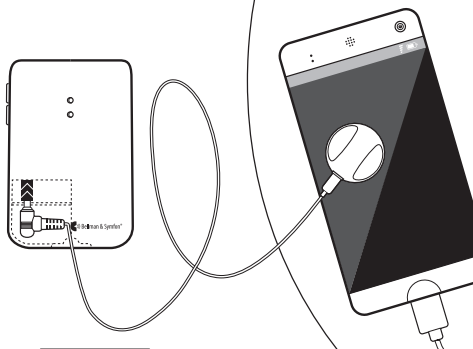


# 2

- GB Test
- DE Testen
- FR Tester
- NL Test

- SE Testa
- NO Testing
- DK Test
- FI Testaus

- ES Probar
- IT Provar
- PT Testar
- GR Έλεγχος

- PL Testowanie
- CZ Test
- HU Tesztelés
- SRB Provera

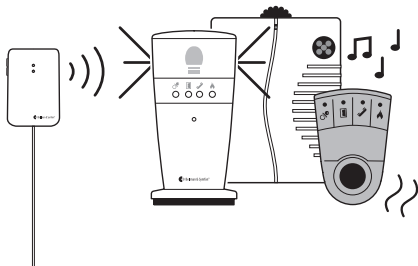



# 1

- GB Connect
- DE Anschliessen
- FR Connectez
- NL Aansluiten

- SE Anslut
- NO Tilkobling
- DK Tilslutning
- FI Liitä

- ES Conectar
- IT Collegare
- PT Ligar
- GR Συνδέω

- PL Podłączyć
- CZ Připojit
- HU Telepítés
- SRB Povezati

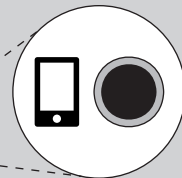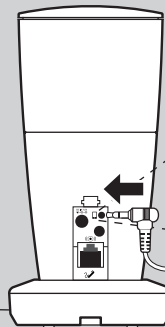

# 2

- GB Test
- DE Testen
- FR Tester
- NL Test

- SE Testa
- NO Testing
- DK Test
- FI Testaus

- ES Probar
- IT Provar
- PT Testar
- GR Έλεγχος

- PL Testowanie
- CZ Test
- HU Tesztelés
- SRB Provera

