

# Getting Started

Vibio Bluetooth® bed shaker

ENGLISH


## Meet Vibio.

### What you'll need

- A mobile or tablet with iOS 8 / Android 4.4 or later.
- An USB power adapter, max 12W.


### In the box

- 1 Charging cable
- 2 Bed shaker

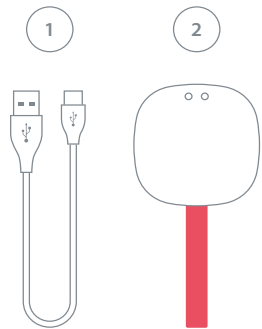

### Look for the blinking eyes

- **White** means it's activated by an alarm or communicating with your phone.
- **Blue** means it's ready for pairing.
- **Green** means it's charging.
- **Yellow** means you have an incoming call or message.
- **Red** means the battery level is low.

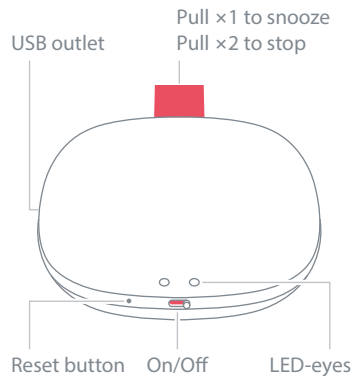


## Ready. Steady. Go.

### Set up your Vibio

Download the Vibio app from the App Store® or Google Play™ and follow the simple setup instructions.

It should only take a few minutes.

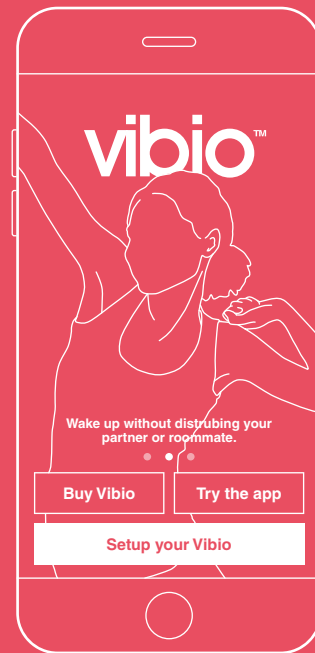

# DESIGN FOR EARS™

**Bellman & Symfon**

Södra Långebergsgatan 30  
421 32 Västra Frölunda, Sweden

Phone +46 31 68 28 20

E-mail [info@bellman.com](mailto:info@bellman.com)

[bellman.com](http://bellman.com)

BE1220\_024MAN003

© and TM 2017 Bellman & Symfon AB. All rights reserved.

