

Installing the app

1 Turn on Bluetooth®

Swipe down from the top of the screen and tap the Bluetooth icon.



2 Install the Visit app



Download the **Bellman Visit** app from the App Store® or Google Play™ and follow the setup instructions.



Installing the transmitter

3 Mount it


Remove the protective film from the Velcro on the back of the transmitter and mount it on the wall. Alternatively use the supplied screw and plug.

4 Turn it on

Pull the battery tab to start the transmitter. The indicators light up in green to show that it is on.

5 Connect it

Connect the USB cable to the power adapter. Then plug the adapter into a wall outlet. If you have a landline telephone, connect it to the telephone jack using the supplied splitter.

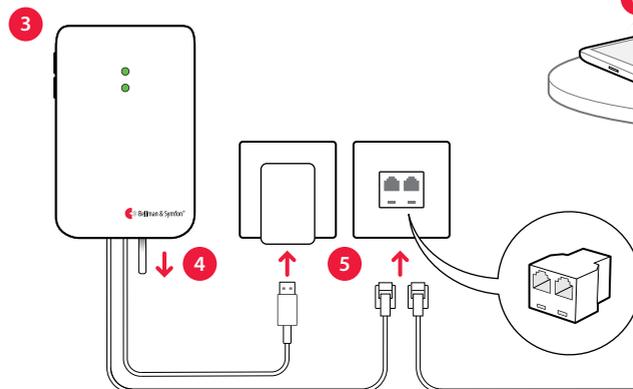


6 Test the mobile phone

Have someone call your mobile. The yellow LED on the Visit receiver blinks and it starts to flash, sound or vibrate (depending on the receiver).

7 Test the landline phone

Call your landline phone. The yellow LED on the Visit receiver lights up and it starts to flash, sound or vibrate (depending on the receiver).

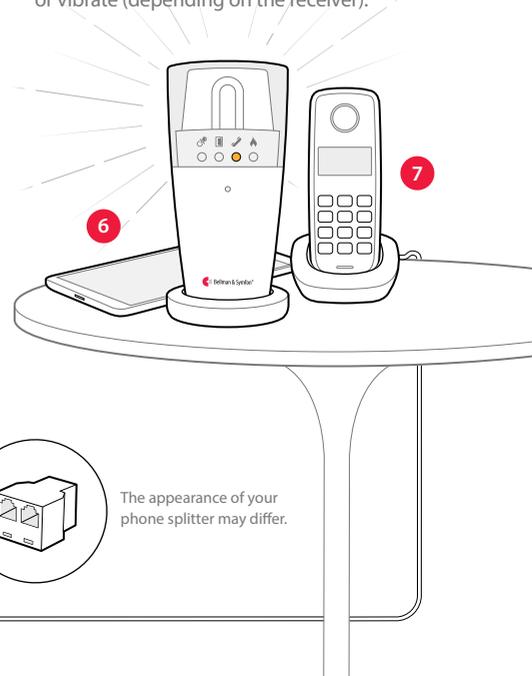

The appearance of your phone splitter may differ.

Using the app

There are times during the day when you probably don't want to be disturbed. The app features four preset **Activities** that gives you full control of your Visit alerts. Just tap an activity to select it.



Good morning

Visit alerts for both calls and messages during this activity. Select it during daytime, when you don't want to miss out on anything.



Good night

All Visit alerts are off during this activity. Select it when you go to bed, so you are not woken up by alerts during your sleep.




Movie night

Visit only alerts for calls during this activity. Select it when you are watching TV, so you're not distracted by a constant flow of messages.



Dinner time

Visit only alerts for messages during this activity. Select it at dinner, so you don't have to answer incoming calls in the middle of the main course.

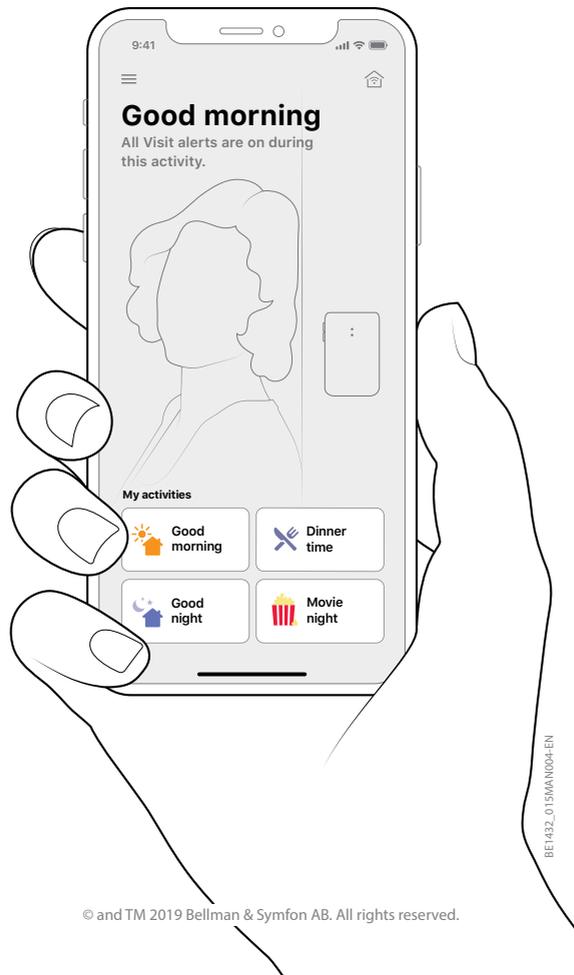

BE1432_013MAN1004-EN

© and TM 2019 Bellman & Symfon AB. All rights reserved.


Welcome!

What you'll need

- A mobile phone with iOS 11 / Android 4.4 or later.

What's in the box

- | | |
|-------------------------|----------------------|
| 1 Telephone transmitter | 4 Screw and plug |
| 2 Power cable | 5 Telephone splitter |
| 3 Telephone cord | 6 Power adapter |

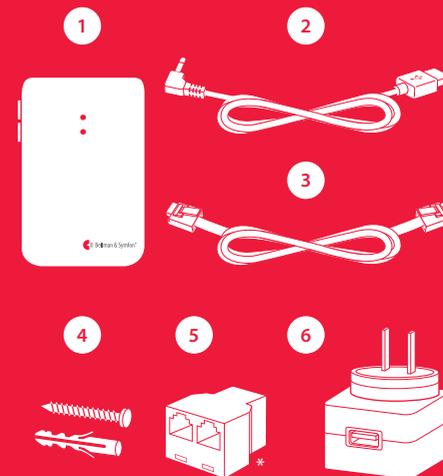

* The appearance of your telephone splitter may differ.

For technical information, advanced settings and troubleshooting, go the [app Help section](#).