Isättning

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Insertion

- 1 Hold the cap between your thumb and forefinger.
- 2 Pull your ear gently upwards/outwards.
- 3 Insert the earplug in the ear canal, using a slight turning movement, until it fits comfortably and seals the ear canal.

Removal

- 1 Remove the earplug by gently twisting the cap as you pull it out.
- 2 Store your earplugs in the protective case after use.

Warning

- Do not use damaged earplugs, it can affect the attenuation.
- Do not disassemble the earplugs.
- Do not use the earplugs if the sound level exceeds 95 dBA for more than 8 consecutive hours.

- Always use the earplugs for the entire time while being exposed to harmful noise.
- Regularly check that your ER20XS works as intended. Note that the effect of the earplugs may be impaired by misuse or poor maintenance. Therefore, follow the recommendations strictly in this manual.

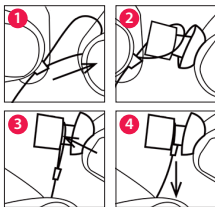
Cleaning

- Wipe the tip with a damp cloth. The product can be adversely affected by certain chemicals. More information is available from the manufacturer.

Warranty


- The warranty is valid for 2 years from the date of purchase at one of our dealers. Guarantee claims can only be made at retailers by presenting the original receipt. The warranty does not cover normal wear and tear, damage caused by improper use or sound leakage.

Neck cord installation


- 1 Pull slider to make a loop.
- 2 Slide loop over the earplug.
- 3 Position loop against the earplug.
- 4 Pull cord tight on the earplug.

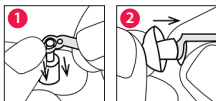Pull tab installation

- 1 Place tab on stem. Pull down on tab until it snaps in place.
- 2 Replace eartip.

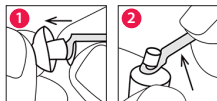Pull tab removal

- 1 Remove eartip.
- 2 Grasp tab and pull up.

* **Warning:** These ear plugs should not be used where there is a risk that the connecting cord could be caught during use.

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H = 16 dB, M = 15 dB, L = 12 dB, SNR = 16 dB

H Average attenuation characteristics in the high frequency spectrum (> 2 kHz)

M Average attenuation characteristics in the mid frequency spectrum (0.5-2 kHz)

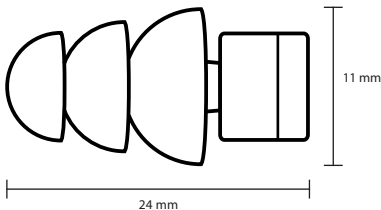
L Average attenuation characteristics in the low frequency spectrum (<0.5 kHz)

APV (Assumed Protection Value) **SNR** (Single Number Rating): Average attenuation characteristics in the standard frequency spectrum.

Dimensions

Length: 24 mm

Diameter: 11 mm

For more information please contact:

Bellman & Symfon

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Tel +46 31 68 28 20

Fax +46 31 68 28 90

Email info@bellman.com

bellman.com

EN352-2:2002