

Visit Baby Monitor

BE1491



Features

Reliable transmission

Up to 120 m operating range

Flexible

Adjustable sensitivity and delay

Customizable notification

Use a unique signal for each child

Optional contact mat

Know if your child leaves the bed

Easy to use

No complicated menus

Be there for your baby.

Caring for your baby's health and safety is one of the most important things you do as a parent. Bellman & Symfon offers a fully secure baby monitoring solution for parents with hearing loss, so you are always close to your baby. The Visit Baby Monitor will notify you as soon as your baby needs you, even if you are asleep or a couple of rooms away.

How it works

When the Baby Monitor is activated, it signals your Visit receiver that notifies you with sound, flashes or vibrations (depending on the receiver).

Secure and reliable

Visit uses a secure radio link that covers your entire home, so you always know that you are connected.

Adjustable sensitivity and delay

You can adjust the sensitivity and delay of the Baby Monitor so it's not activated when your child moves in her sleep.

Customizable notification

If you have more than one child you can choose different signals so you know who needs your attention.



Technical specifications




Size and weight

Height: 100 mm, 4.0"
Width: 65 mm, 2.6"
Depth: 27 mm, 1.1"
Weight: 118 g, 4.2 oz (incl. batteries)



Buttons and connectors

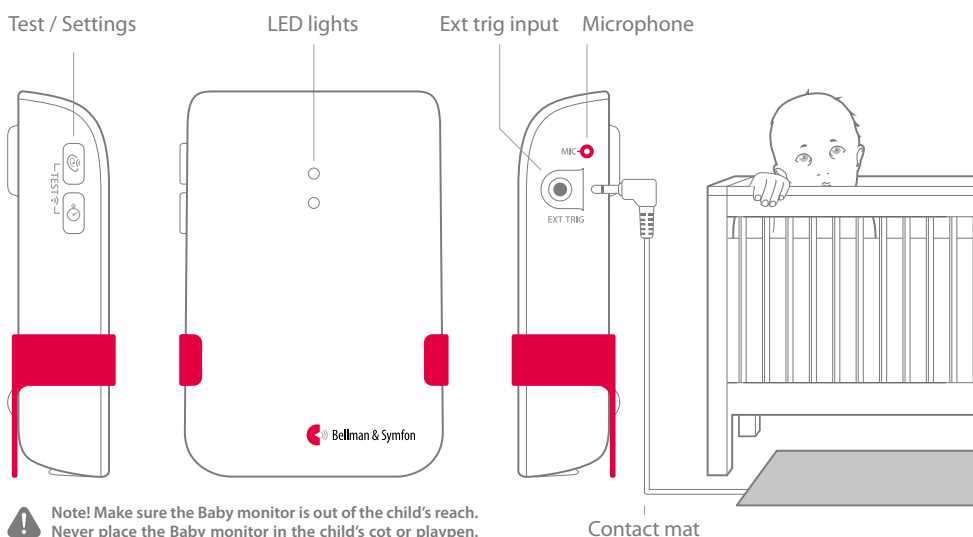

Inputs/outputs

- 3.5 mm external trigger input

Activation

- The test buttons and the internal microphone
- The contact mat accessory

Settings

- Sensitivity: High, Medium, Low
- Delay: 30 sec, 10 sec, 1 sec

Number of radio keys

- 64 as standard. Up to 256 with optional software.

Frequency and coverage

- Radio frequency: 315 MHz, 433.92 MHz
- Coverage by region:
315 MHz: 30 – 80 m, (98 – 262 ft)
433 MHz: 70 – 120 m, (230 – 394 ft)
Depending on the characteristics of the building.

Power and battery

- Power supply
Battery power 2 x 1.5 V AA lithium or alkaline
- Power consumption
Active < 70 mA, idle position < 400 µA
- Operation time
Alkaline ~ 6 months, lithium ~ 1 year

Maintenance and cleaning

- Maintenance free. Clean with a dry cloth.
- Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, or abrasives.
- Not reusable.

Environmental requirements

- For indoor use only.
- Temperature: 0 to 35° C, 32 to 95° F
- Relative humidity: 15% - 90% (non condensing)

Compliance

This product complies with applicable FCC and ISED Canada regulations.
本機器は日本の電波法に適合しています。

In the box

- BE1491 Visit Baby Monitor
- Table stand
- 2 x 1.5V AA Alkaline batteries



Accessories

The following accessories are available:

- BE9026 Contact Mat

© 2025 Bellman & Symfon AB. All rights reserved.

Bellman, Bellman & Symfon and the Bellman & Symfon logo are registered trademarks of Bellman & Symfon AB. This leaflet is subject to change without prior notice.

BE1491_001DSH1.0_EN

bellman.com