
Visit telephone transmitter

BE1431

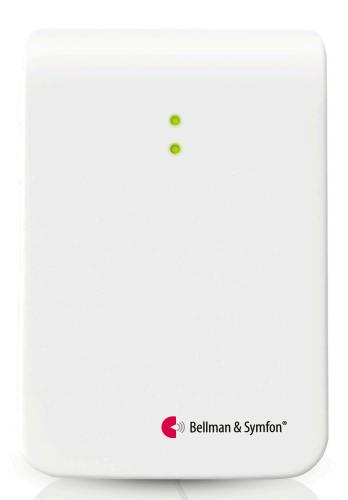





Features

Works for most phone types Both landline and mobile phones*

Multiple connections

Three separate trigger inputs

Customizable notification

Choose one for each trigger

Long lasting power

Up to 10 years of battery life

Secure radio link

Up to 250 m operating range

* Mobile phone sensor accessory required

Stay in touch with your loved ones.

The telephone transmitter from Bellman & Symfon is part of the Visit smart home system that helps you to keep tabs on your home. It can be used to detect both your landline and mobile phone – or even an incoming message on your tablet (accessory required).

How it works

When the telephone transmitter is activated, it signals your Visit receiver that notifies you with sound, flashes or vibrations (depending on the receiver).

It is equipped with three triggers that, aside from telephone detection, can be used for:

- A contact mat accessory
- A magnetic switch accessory
- An existing doorbell, etc.

The telephone transmitter allows you to choose an individual alert for each trigger and it uses a robust radio link that covers your entire home. It is very easy to set up, virtually maintenance free and operates for years on standard batteries.





Size and weight

100 mm, 4.0" Height: Width: 65 mm, 2.6" Depth: 27 mm, 1.1"

Weight: **120** g, **4.2** oz. (incl. batteries)



Inputs/outputs

- RJ11 input to analogue telephone network
- 2.5 mm ext trig input for the mobile phone sensor
- 3.5 mm ext trig input for other accessories

Activation

- The test buttons and the landline telephone
- A smartphone/tablet via the mobile phone sensor
- The contact mat or magnetic switch accessory
- A doorbell, etc. connected to the ext trig


Frequency and coverage

- Radio frequency: 315 MHz, 433.92 MHz or 868.3 MHz, depending on the region
- Coverage: 50 250 m, depending on the radio frequency and building's characteristics

Environmental requirements

- For indoor use only.
- Temperature: 15°-35° C, 59° to 95° F
- Relative humidity: 5% -95% (non condensing)

Buttons and connectors

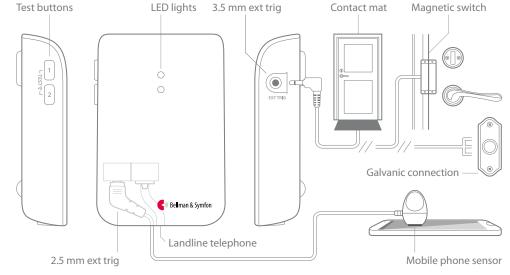

Power and battery

- Power supply Battery power 2 x 1.5 V AA lithium or alkaline
- Power consumption Active < 70 mA, idle position < 15 μA
- Alkaline ~ 5 years, lithium ~ 10 years

Number of radio keys

• 64 as standard. Up to 256 with optional software.

Maintenance and cleaning

- Maintenance free. Clean with a dry cloth.
- Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, or abrasives.
- Not reusable.

Regulatory requirements

Complies with the following requirements: CE, FCC, RoHS, RCM, WEEE, Der Grüne punkt











In the box

- BE1431 Visit telephone transmitter
- 2 x 1.5V AA alkaline batteries
- Velcro for wall mounting
- Alcohol wipe
- Screw and plug
- User manual

Accessories

The following accessories are available:

- BE9251 Mobile phone sensor
- BE9023 Magnetic switch
- BE9024 Contact mat
- BE9253 Ext trig cable (3.5 mm connector)

© 2016 Bellman & Symfon AB. All rights reserved.

Bellman, Bellman & Symfon and the Bellman & Symfon logo are registered trademarks of Bellman & Symfon AB. This leaflet is subject to change without prior notice.

BE1431 001DAS002

bellman.com