
**BE1441**  
**Visit Flash Receiver**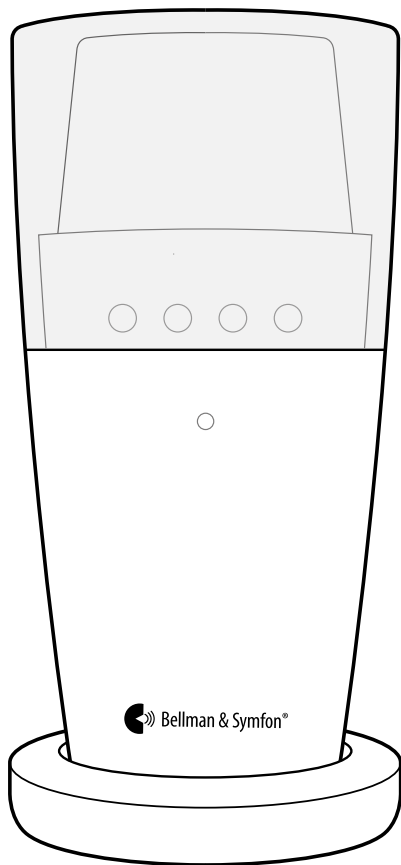

## BE1441 Visit Flash Receiver

---

4 User manual

---

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---

19 Appendix (EN)

---

EN

JP


App

# Read this first

This booklet guides you on how to use and maintain your new Flash Receiver. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your point of purchase for further assistance.


- Intended user**
- The intended user is a person with a wish to receive augmented signals.
- Intended user group**
- The intended user group consists of people of all ages.
- Principle of operation**
- The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

## Overview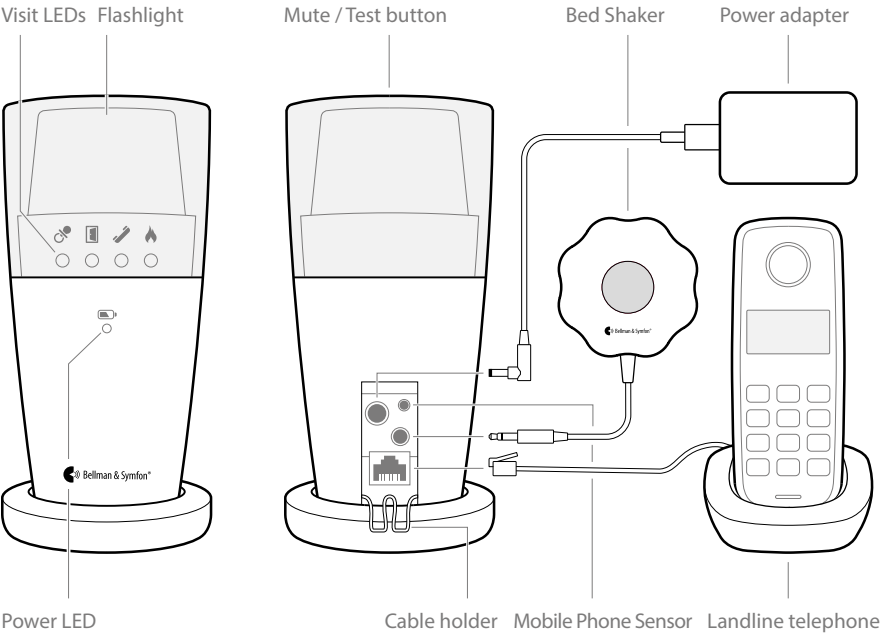

# Getting started

- 1 Connect the power supply to the receiver and the mains outlet. Place the receiver on a level surface or mount it on the wall using the wall bracket accessory (see separate instructions).
- 2 To test the radio link you need a Visit transmitter. Press the test button/s on the transmitter.
- 3 The receiver lights up a Visit LED and starts to flash. If a bed shaker is connected, it will vibrate. A short press on the mute/test button repeats the last indication. If nothing happens, see **Troubleshooting**.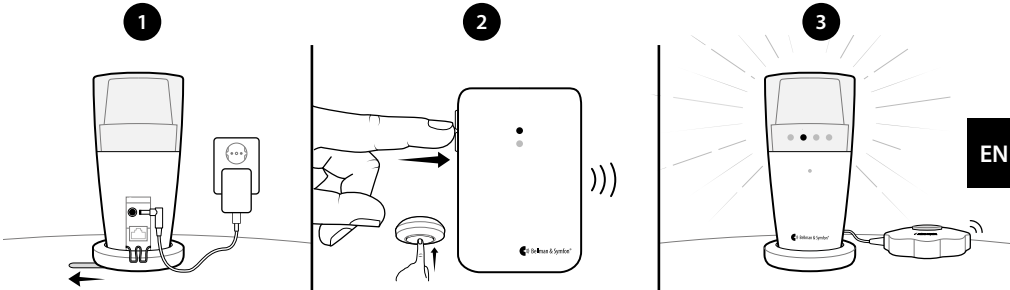

## Default signal pattern

When a transmitter is activated, the Flash Receiver lights up an LED, flashes and the Bed Shaker starts to vibrate with a certain pace. This is called signal pattern. The transmitters determine the pattern, and the default is as follows:

Transmitter	Flash Receiver		Bed Shaker
Activated source	Visit LED	Flash	Vibration
▪ Door Transmitter	Green	Yes	Slow
▪ Push Button Transmitter	Green	Yes	Slow
▪ Telephone Transmitter	Yellow	Yes	Medium
▪ Baby Monitor	Orange	Yes	Fast
▪ Smoke Alarm	Red	Yes	Long
▪ CO Alarm	Orange/red	Yes	Long

## Changing the signal pattern

The signal pattern can only be changed on the transmitters. See **Changing the signal pattern** for the relevant transmitter.

## Power LED indications

When the Flash Receiver is connected to mains power, the power LED lights up in green.

## Personalizing your receiver


### Directing the flash

The Flash Receiver features a rotating top that makes it easy to direct the light. Point it for example towards a wall if you feel that the flash is too intense. A silicone slip-on top is also available in a variety of colors (art. no. BE9164-BE9167).

### Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located on the **transmitters**.

- 1 Open the transmitter cover and move any radio key switch to the up (on position) to change the radio key. See **Changing the radio key** for the relevant transmitter.
- 2 Press and hold the test button located on the top of the flash receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press the test button/s on the transmitter within 30 seconds to send the new radio key.
- 4 All Visit LEDs on the receiver blink 5 times to show that the radio key has been changed. Please note that all Visit units must be set to the same radio key.

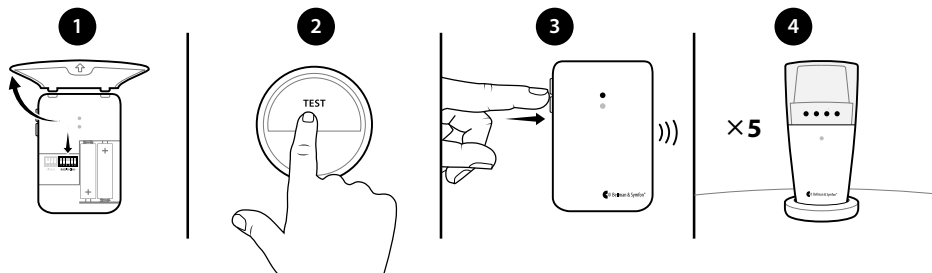## General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

### Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Be aware that alarms and notifications will be missed if the power cable is detached.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Flashes can trigger epileptic seizures for people with photosensitive epilepsy (PSE).
- Use only power adapters that are specified in this booklet.
- Protect cables from any potential source of damage.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.

### Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit [bellman.com](https://bellman.com) for contact info.
- The programming pin on the back of the flash receiver is for authorized service personnel only. Unauthorized use may damage the device.
- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this booklet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

	Indicates the manufacturer's serial number so that a specific device can be identified.		This symbol indicates the need for the user to consult the instructions for use.
	Indicates the device manufacturer.		To identify a place where information may be found, especially in an emergency.
	This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.		Indicates the temperature limits to which the device can be safely exposed.
			Indicates the range of humidity to which the device can be safely exposed.

Troubleshooting

If	Try this
The receiver seems to be turned off	▪ Check that the power supply is connected correctly.
The receiver does not respond when a transmitter is activated, but works when I use the test button	▪ Check the transmitter connections. ▪ Move it closer to the transmitter to make sure it's within range. ▪ Check that the receiver is set to the same radio key as the other units in the Visit system, see <b>Changing the radio key</b> on our website.
The receiver is activated for no apparent reason	▪ There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see <b>Changing the radio key</b> on our website.
The flash is too bright	▪ Redirect the light by rotating the top or use a silicone slip-on top accessory to dim the light.

EN

Website:

# Technical specifications

Dimensions	70 × 140 mm, 2.7 × 5.5" (Ø × H)
Weight	308 g, 10.8 oz.
Mains power	7.5 V DC / 1500 mA Only use the supplied power adapter. This product is also designed for IT power distribution systems with phase to phase voltage of up to 230V. The outlet shall be installed near the equipment and shall be easily accessible.
Power consumption	Active: 750 mA, idle position: 8 mA
Connections	<ul style="list-style-type: none"><li>▪ 2.5 mm external trigger jack</li><li>▪ 3.5 mm bed shaker outlet, vibration power: 2.0 – 4.0 VDC (max. 500 mA)</li><li>▪ RJ-11 telephone modular jack, 26 – 120 V RMS, 15 – 100 Hz</li><li>▪ Barrel jack for the external power adapter</li><li>▪ Programming pin for service purposes. Output: 3.3 V/max. 150mA.</li></ul>
Flash	Built-in ~30 Candela Xenon light <b>Warning!</b> Flashes can cause epileptic seizures
Radio frequency	315 MHz
Coverage	30 – 80 m depending on the characteristics of the building.
Operation	Temperature: 0° to 35° C, 32° to 95° F Humidity: 15% to 90%, non-condensing
Configuration	This receiver can be used with all Visit transmitters.
Accessories	<ul style="list-style-type: none"><li>▪ BE1270 Bed Shaker</li><li>▪ BE9075 Wall Mount Bracket</li><li>▪ BE9105 Telephone Cable</li></ul>
In the box	<ul style="list-style-type: none"><li>▪ BE1441 Visit Flash Receiver</li><li>▪ 7.5 V DC / 1500 mA power adapter</li></ul>

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
## Appendix (EN)

## Testing the connection


A Visit transmitter set to the same radio key as the flash receiver is required to test the system. You will need a bed shaker accessory to test the vibration.

- 1 Press the transmitter test button(s).
- 2 The flash receiver will:
  - Start to flash
  - Light up the corresponding Visit LED
  - If a bed shaker accessory is connected, it will start to vibrate.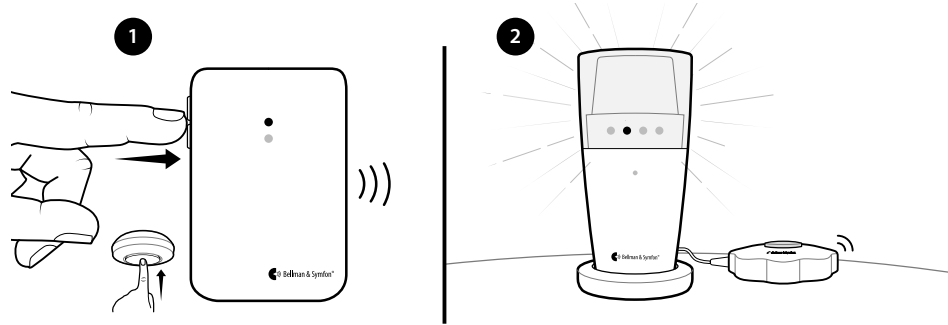

## Testing the landline telephone

- 1 Connect the flash receiver to an analogue telephone socket via the telephone Input. You will need a telephone flex cable accessory and an adapter plug accessory.
- 2 Use for instance your mobile phone to call your landline telephone. The flash receiver will start to flash and light up the yellow Visit LED. If a bed shaker accessory is connected, it will start to vibrate. To repeat the last indication, press the flash receiver test button.

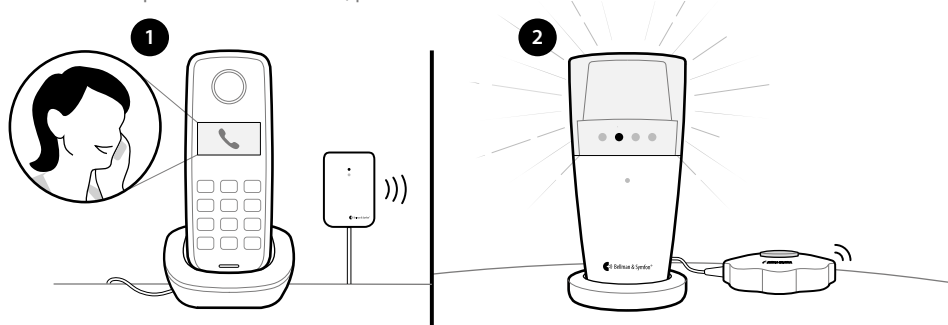## Advanced programming

By using advanced programming, you can customize the signal pattern from a specific transmitter and event, displaying the LED colour and vibration pattern of your choice. The advanced programming overrides the radio key and pairs the units via the serial number.

### **i** Please note

- Smoke- and CO alarms cannot be programmed for safety reasons.
- The transmitter must be activated as it is intended to be used in the system to generate the right signal. This means that you can't always use the transmitter test button (see **Default signal pattern** for the relevant transmitter).

### Here is how you program the receiver:

- 1 Press and hold the mute/test button on the receiver. The green and yellow Visit LEDs will start to blink alternately. While still holding down the button, activate the desired transmitter as intended. Release the button.
- 2 Scroll through the different **Visit LED options** by pressing the mute/test button on the receiver. Select the desired Visit LED colour by holding down the mute/test button until the power LED goes out and lights up again.
- 3 Scroll through the different **vibration options** by pressing the test button on the receiver (bed shaker required). Select the desired vibration pattern by holding down the mute/test button until the power LED goes out and lights up again.
- 4 The receiver will now show the new Visit LED colour and vibration pattern. Press the mute/test button briefly to end the demonstration. After a short while, it will return to normal mode.

### Deleting the advanced programming

Follow the procedure below to delete the advanced programming.

- 1 Hold down the mute/test button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 2 Press the mute/test button on the receiver 3 times in quick succession.
- 3 All Visit LEDs will light up for ~2 seconds to show that it has been deleted.

**Product disposal and recycling information**

According to local laws and regulations, your product should be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The recycling of your product will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

Bellman & Symfon Asia Ltd.  
1401 & 1408, Laifung International Centre  
33 Jie Fang Road South  
Guangzhou, China  
Phone: +86 20 8318 2950

# DESIGN FOR EARS™

## Manufacturer

Bellman & Symfon Group AB  
Södra Långebergsgatan 30  
436 32 Askim Sweden  
Phone +46 31 68 28 20  
E-mail [info@bellman.com](mailto:info@bellman.com)  
[bellman.com](http://bellman.com)

Revision: BE1441\_194MAN1.0  
Date of issue: 2024-12-04  
TM and © 2024 Bellman & Symfon AB.  
All rights reserved.