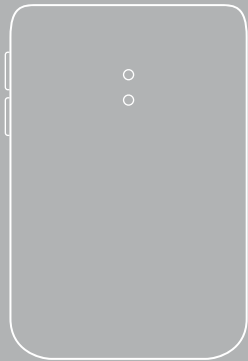
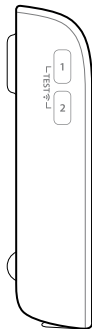
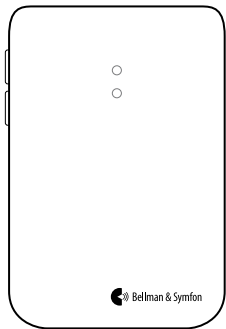
BE1411 Visit door transmitter

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185 Appendix (EN)

DE

App

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Appendix (EN)

186 Installation using the External microphone

187 Hardwiring your doorbell

188 Testing the signal

190 Default signal pattern

190 Changing the signal pattern

191 Using electromagnetic detection

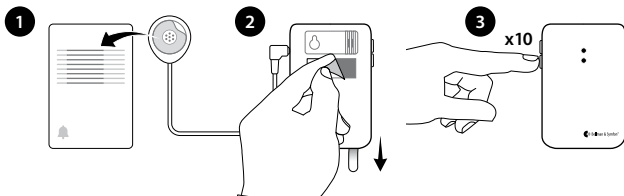
192 Changing the radio key

Alternative installation – single sound source

Using the external microphone accessory




- 1 Connect the external microphone (sold separately) to the transmitter. Remove the protective film and attach it to the doorbell's sound source.
- 2 Pull the battery tab to start the transmitter. Remove the protective film from the Velcro and mount the transmitter on the wall.
- 3 Within 2 min, press the lower button 10 times to turn off the internal microphone. The LEDs will blink 3 times in red to confirm.

Note: If you exceed 2 minutes, you need to restart the transmitter by removing the batteries and putting them back in again.

Installation – hardwiring your doorbell

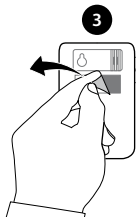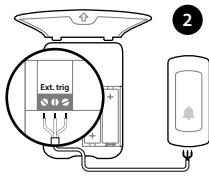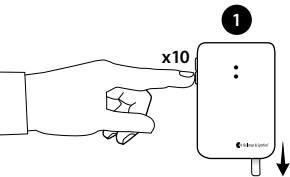- 1 Pull the battery tab to start the transmitter. Within 2 min, press the lower button 10 times to turn off the internal mic. The LEDs will blink 3 times to confirm.
- 2 Remove the front cover to access the screw terminal. There are 3 connection points:
 - Use **1** and **2** to connect an **active** switch, like a relay with current (2–30VDC, polarity independent or 2–24VAC, 5 - 150Hz).
 - Use **2** and **3** to connect a **passive** switch, like a push button.
- 3 Put the front cover back, remove the protective film from the Velcro and mount the door transmitter on the wall.



Testing the signal

Using the test buttons

- 1 Press both test buttons simultaneously on the transmitter. The top LED lights up in green to show that a radio signal is being transmitted.
- 2 The green Visit LED on the receiver lights up to show that the signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

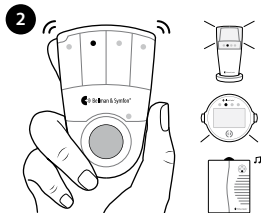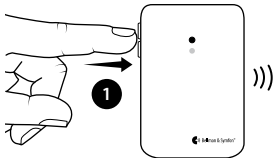Note: If you have changed the signal pattern, it will react in accordance with the tables on page 191 to 192.



Testing the signal

Using the doorbell or intercom

- 1 Press the button on the doorbell or intercom. The LED on the transmitter lights up in green to show that the sound is detected.
- 2 The green Visit LED on the receiver lights up to show that the radio signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

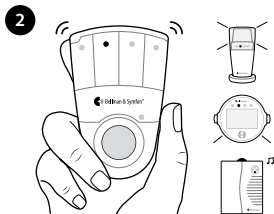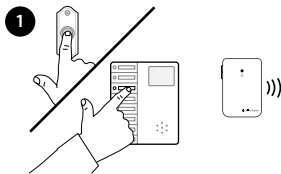Note: If you have changed the signal pattern, it will react in accordance with the tables on page 191 to 192.

Default signal pattern

When the transmitter or a connected accessory is activated, the following happens:

- 1 The LED on the transmitter lights up in green to show that it is signalling the receiver.
- 2 The Visit LED on the receiver lights up in green and it starts to sound, flash or vibrate with a certain pace, called signal pattern. The transmitter determine the signal pattern and the default is as follows:

Transmitter		Visit receiver			
Source	LED	LED	Sound	Vibration	Flash
Internal mic.	Green, top	Green light	1x door chime, low	Slow	Yes
External mic.	Green, bottom	Green blinks	2x door chime, low	Slow	Yes
Doorbell con.	Green, top	3x green blinks	1x door chime, high	Slow	Yes

Changing the signal pattern

The transmitter controls the signal pattern. Open the transmitter front cover and move the signal switches according to the tables to change it:

Transmitter

Receiver signal pattern

Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1 x door chime, low	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	Green blink	2 x door chime, high	Slow ■□□□	Yes
	Connected doorbell	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Int. mic. / learned signal 1 / test	2 x green blinks	2 x door chime low	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Connected doorbell	Green blinks	2 x door chime, high	Slow ■□□□	Yes
	Int. mic. / learned signal 1 / test	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	2 x green blinks	2 x door chime, low	Slow ■□□□	Yes
	Connected doorbell	3 x orange blinks	Baby melody	Fast ■■■■■■	Yes
	Int. mic. / learned signal 1 / test	Green blinks	2 x door chime, high	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	Green light	1 x door chime, low	Slow ■□□□	Yes
	Connected doorbell	Orange blinks	Baby melody	Fast ■■■■■■	Yes

Transmitter


Receiver signal pattern

Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1 x door chime, low	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	3 x green blinks	1 x door chime, high	Slow	■□□□ Yes
	Connected doorbell	2 x green blinks	2 x door chime, low	Slow	■□□□ Yes
	Int. mic. / learned signal 1 / test	Green light	1 x door chime low	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	Green blinks	2 x door chime, high	Slow	■□□□ Yes
	Connected doorbell	Yellow blinks	2 x ring signal, high	Medium	■□□□ Yes
	Int. mic. / learned signal 1 / test	3 x green blinks	1 x door chime, high	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	Green light	1 x door chime, low	Slow	■□□□ Yes
	Connected doorbell	Red + orange blinks	Emergency siren	Long	■□□□ Yes
	Int. mic. / learned signal 1 / test	Green blinks	2 x door chime, high	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	2 x green blinks	2 x door chime, low	Slow	■□□□ Yes
	Connected doorbell	Red blinks	Fire alarm	Long	■□□□ Yes

Using electromagnetic detection

The door transmitter can be set to detect electromagnetic fields emitted by electromechanical doorbells. Here is how it's done:

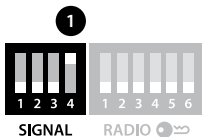- **Activating electromagnetic detection**
Move the 4th signal-switch (1) to the Up=On position.
- **Deactivating electromagnetic detection**
Move the 4th signal-switch to the Down=Off position.

Changing the radio key

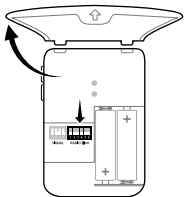
If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located under the transmitter cover.

Here is how you change the radio key:

- 1 Carefully open the front cover on the transmitter. Move any radio key switch to the up (on position) to change the radio key.
- 2 Press and hold the test/function button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press both test buttons simultaneously on the transmitter within 30 s to send the new radio key.
- 4 All Visit LEDs on the receiver blink 5 times to show that the radio key has been changed. It then returns to normal mode.

Please note that all Visit units must be set to the same radio key in order to operate as a group.


1

2

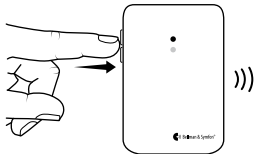3

4

×5

App

Manufacturer

Bellman & Symfon Group AB
Södra Långebergsgatan 30
436 32 Askim Sweden
Phone +46 31 68 28 20
E-mail info@bellman.com
bellman.com

Revision: BE1411_209MAN2.0

Date of issue: 2022-09-07

TM and © 2022 Bellman & Symfon AB.

All rights reserved.