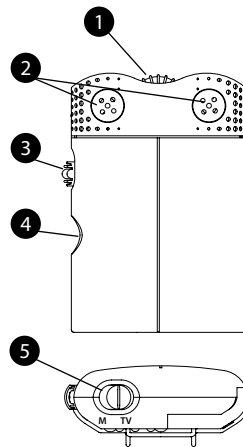
EN

- 1 Volume control
- 2 Microphones
- 3 On/Off switch
- 4 Headphone jack
- 5 Speech/ TV switch

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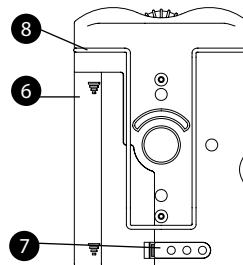
FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

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Bellman & Symfon North America Inc.
 5509 Business Dr, Unit B
 Wilmington, NC 28405
 Unites States
 Phone: +1 (910) 239 5796
 Toll-free: + (833) 235 5626
 E-mail: support.bsn@bellman.com

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Revision: BE1053_010MAN007
 Date of issue: 2021-06-08
 TM and © 2021 Bellman & Symfon AB.
 All rights reserved.

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Response

BE1053

Bellman Response

EN

Thank you for choosing products from Bellman & Symfon!

Getting started

1 Install the battery

Slide the Battery lock (7) on the rear of the unit to the right, then open the battery compartment by pulling the Battery cover (6) downwards.

Insert a 1.5V battery (LR03, AAA) into battery compartment. Close the Battery cover (6). Slide the Battery lock (7) back again.

2 Connect the earphones or headphones into the Headphone jack (4).

3 Turn on/off the unit

Turn on: slide the On/Off switch(3) to “I” position.

Turn off: slide the On/Off switch(3) to “O” position.

4 Volume control

Increase the volume: rotate the Volume control(1) clockwise to increase the volume to a convenient level.

Decrease the volume: rotate the Volume control (1) counter clockwise to decrease the volume to a convenient level.

5 Speech/TV mode

Speech mode: for listening to speech through the microphones, slide the Speech/TV switch (5) to the “M” position.

TV mode : for listening to the TV, connect the BE9022 TV listening kit to the Headphone jack (4), then slide the Speech/TV switch (5) to the “TV” position.

The BE9022 TV listening kit is optional accessory and can be purchased additionally if needed.

In the box

- BE1053 Response ■ AAA battery
- BE9124 Earphones or BE9122 Headphones
- User manual

Troubleshooting

Problem	Possible Reason	Solution
No sound	<ul style="list-style-type: none"> ■ The unit is not turned on ■ Wrong setting on Speech/TV mode ■ Low battery 	<ul style="list-style-type: none"> ■ Slide the On/Off switch (3) to “I” position. ■ Set the right mode as per “5 Speech/TV mode” in “Getting Started”. ■ Change the battery
Not enough amplification	<ul style="list-style-type: none"> ■ Volume setting in low position 	<ul style="list-style-type: none"> ■ Adjust the Volume control (1)
Noisy/noncontinuous sound	<ul style="list-style-type: none"> ■ Low battery 	<ul style="list-style-type: none"> ■ Change the battery

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