

# Domino Personal listening system

A presentation



### Domino

- Uses digital signal processing to deliver clear and distinct speech
- Features encryption technology so there is no risk of eavesdropping
- Is easy to use and there are no complicated menus to navigate
- Runs on rechargeable batteries that last up to 12 hours of extensive use





### Domino

- Uses digital signal processing to deliver clear and distinct speech
- Features state-of-the-art encryption so there is no risk of eavesdropping
- Is easy to use and there are no complicated menus to navigate
- ls equipped with rechargeable batteries that last up to 12 hours





# **New improvements**

- Key lock
- Easier access to the Advanced settings
- **→** Volume memory
- Avoid unintentional startup
- Improved sound





# **Listening situations**

1-on-1 conversation





**Watching TV** 

**Small meeting** 





Family gathering







**Outdoors** 







In class

**Busy restaurant** 








In the car



### Meet the audio family

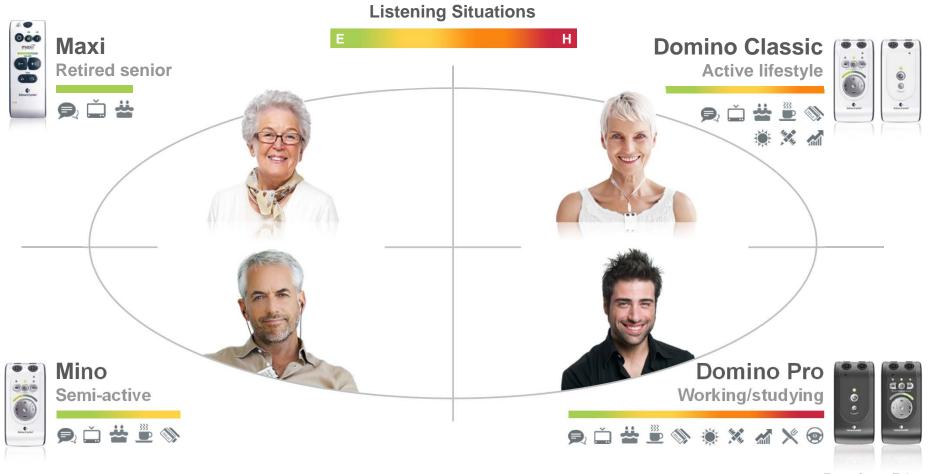



### A solution for every need





### The Domino range

### **Domino Pro**

Our most <u>versatile</u> system



### **Domino Classic**


Our most affordable system







### How it works

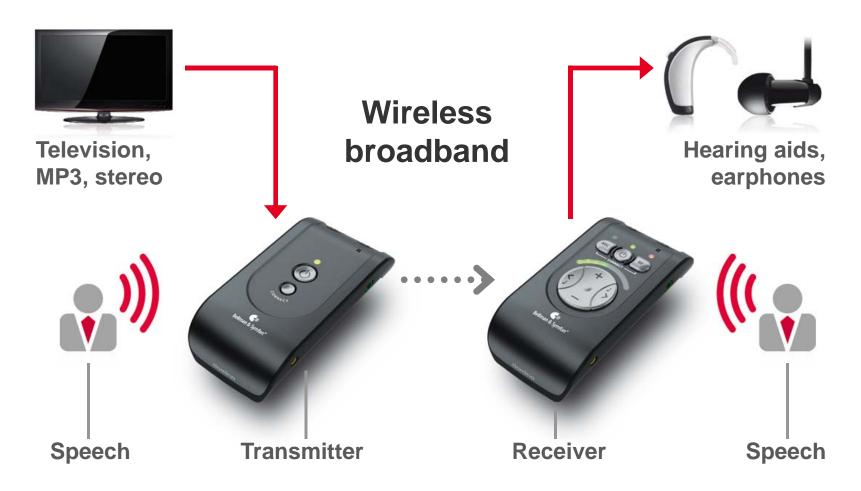



Natural sound with incredible detail

Digital signal processing

10 band noise cancellation

10 band dynamic compression

Adaptive feedback cancellation

Sound with natural colour





# Worry free communication



- Wireless Secure Encryption, WISE
- 128-bit digital encryption
- Impossible to eavesdrop
- Great for confidential business meetings





# Intuitive user interface


- One button one function
- No complicated menus to fumble through
- Soft grip material





# Long lasting power

- Rechargeable Li-ion battery, like any modern cell phone
- Up to 12h operationLess than 2.5h to charge
- **USB** universal charger

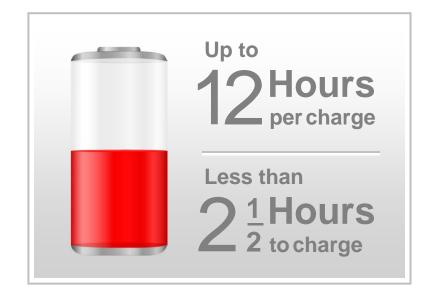



# Light and pocket size



2½ oz – no heavy pockets





# TV and music at your own level



- **→** TV
- Computer
- Radio
- **●** MP3
- **●** DVD





# Great while travelling

- Travel Smart 2.4 GHz.
  Works perfectly
  anywhere in the world
- USB powered universal charger. Mains for US, UK, AU and EU
- Travel cases included





# **Boosts** your hearing aids

Onnect a neck loop and Domino further amplifies speech

Practical remote for easy adjustment of volume, tone and microphones

Amplifies the TV sound in your hearing aid





# Directional microphones on both units

- Zoom function through Adaptive Beam Forming technology
- Suppresses noise from behind and side
- Amplifies sound coming from the front





# Advanced settings menu



#### **Balance control**

Compensates for example for hearing loss in one ear.



#### **Tone control blocking**

Makes it easier for the end user to use Domino.



#### **Basic attenuation**

Trim the attenuation to a specific hearing accessory.



#### **Tone control preset**

Select preset tone for the end user



#### Stereo/Mono

Effects the line-in signal. For neck loop and stetoclips.



#### **Reset to factory settings**

Resets all settings to the standard factory settings.



#### **Bass reduction**

Reduces low-frequency interference (fans, road noise)



#### **Software version**

Primarily for service purposes.



# Ideal listening situations



One-on-one conversations



In school During class



Listening to the television



At work Meeting/seminar



Small meetings e.g. a café



In stores
Talking to a clerk




**Busy restaurant**Pro preferred



In a car Pro preferred




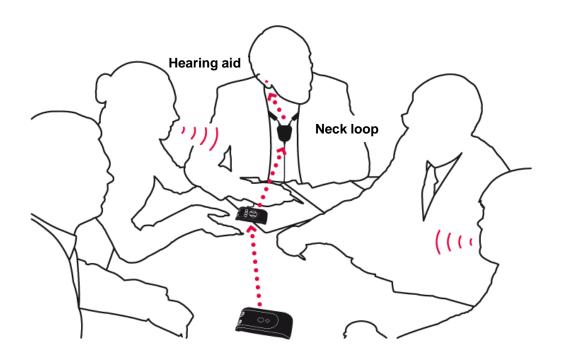

### At work

When more than one person speaks at the same time it can be real be hard to distinguish what is being said.


Domino helps you to hear better in these situations and you can easily place the transmitter close to a distant speaker.

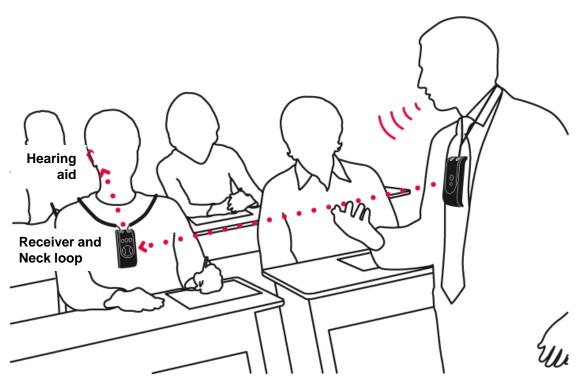

### In class

Even if you are sitting in the back, you can hear the teacher as he was sitting next to you. Clear and noise free.


If you want to communicate with a schoolmate beside you, just press the "Push To Listen" button to pick up sound from your close environment.

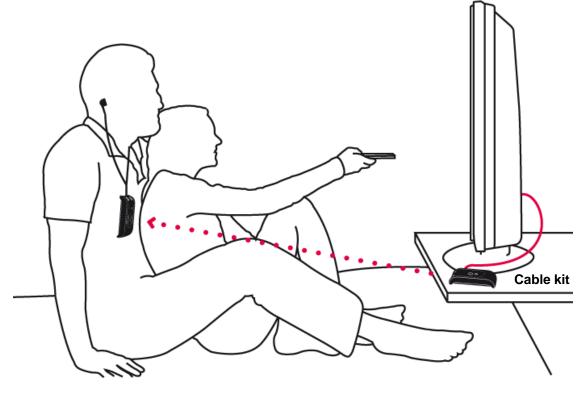

### By the TV

Connect your transmitter to the TV and sit back and enjoy perfect sound in full stereo. No need to crank up the volume to the max.


When you want to discuss the film, you can mix the sound from the TV with the internal microphones on the receiver. "Push To Listen"

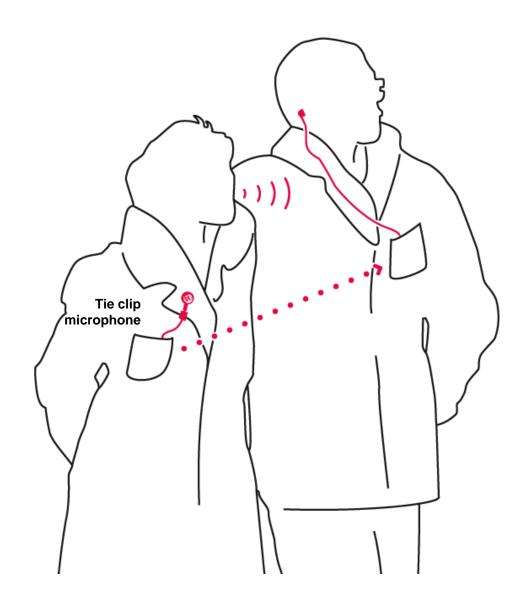

### **Outdoors**

When you are out for a walk, speech is often directed away from you. This in combination with traffic noise makes it very difficult to follow a conversation.

Domino suppresses the noise and overcomes the distance from the speaker effectively.

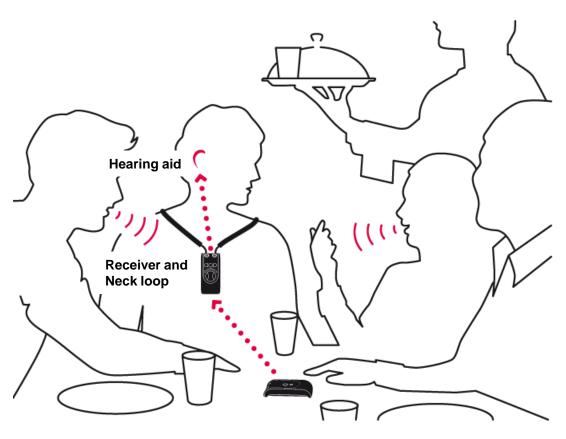


### **Busy restaurant**

In busy places, with music playing and a lot of people talking at the same time, it's easy to feel left out, not understanding what everyone else is laughing about.

Change to the zoom microphone and the background noise will be reduced dramatically so you can focus on the conversation.

oreferre

88

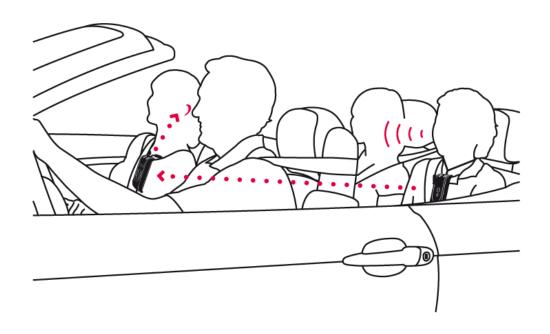

### In the car

A car is one of the hardest sound environments to conduct a conversation in. Besides the noise from the road, wind and engine, the passengers are normally facing away from you.

Select the Zoom microphone and discover the difference.

oreferre

Ser











- Power toggle button
  Power and charge status indication
- Select internal mic mode
  Toggle to switch between Omni
  and Directional mode
- Select remote mic mode
  Toggle to switch between Omni
  and Directional mode
- Press both buttons to pair a receiver to a transmitter





Use + and - buttons to control Volume
9 steps trimmed for optimum usefulness





- Use + and buttons to control Volume 9 steps trimmed for optimum usefulness
- Use < and > buttons to control Tone 9 steps for exact control







- Use + and buttons to control Volume9 steps trimmed for optimum usefulness
- Use < and > buttons to control Tone9 steps for exact control
- Press and hold Mic while in RF mode to select Push-To-Listen
  Use < and > to trim "mix" between Line in signal and int. mic





# Domino **The Transmitter**

- Power toggle button Power and charge status indication
- **Connect mode button** Press during the connection phase For security reasons the transmitter will not accept connection requests unless this button is pressed





### Domino Pro - in the box





### **Domino Classic - in the box**





### Customization



Ext. microphone



**Cable kit** 



Headphones



**Classic receiver** 



**Earphones** 



**Neck loop** 



Neck loop with short cable



Pro receiver



# Neck loop features


High output power with dual channel output

Fabric tube for good comfort

- Safety break-away neck strap
- Pliable cable material
- Tie-clip for optimum positioning
- Soft spray-coated housing





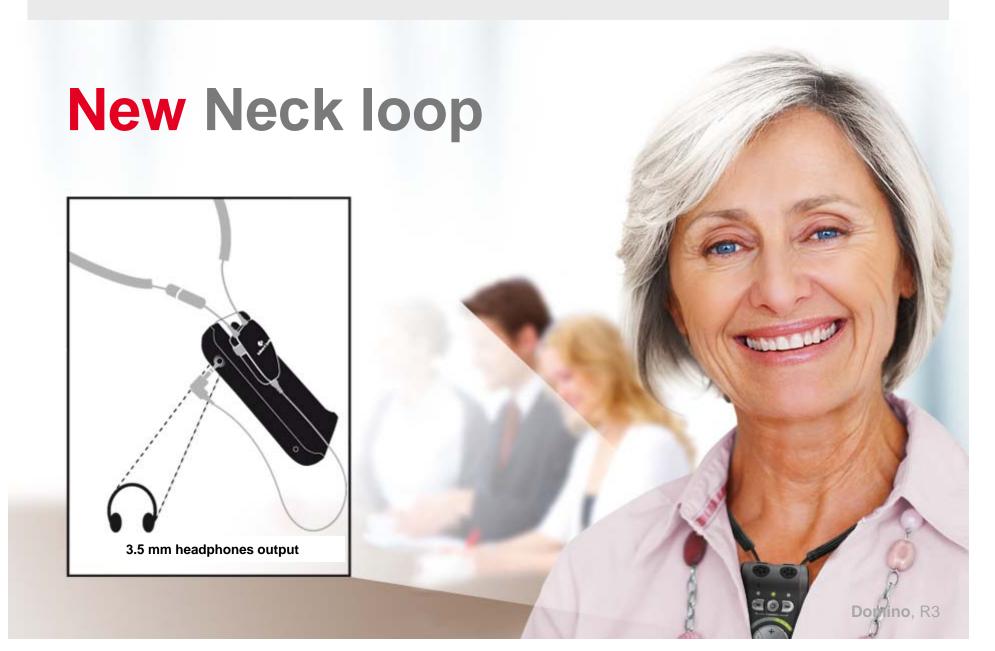



# Addressing real life needs (recap)



### **High Definition Audio**

→ Natural sounding speech with incredible detail



### 128-bit encryption

→ Completely secure from eavesdropping



### **Travel Smart 2.4 GHz**

→ Works perfectly anywhere in the world



# Addressing real life needs (cont.)



### **Easy Connect**

→ Connects easily to your TV, MP3, Stereo etc.



### Single touch action

→ No complicated menus - focus on hearing





#### **Remote Zoom**

→ Equipped with effective directional microphones



