For the safety of your children

Please be aware that this product may include small parts. Ensure that the package and small parts are kept out of reach from your child.

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BEL033_020MAN1.0

bellman.com

TM and © 2023 Bellman & Symfon AB. All rights reserved. All product designs are patented.

 **Bellman & Symfon®**
DESIGN FOR EARS

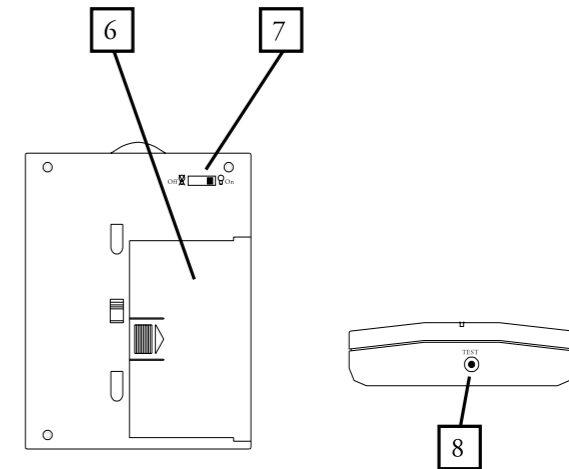
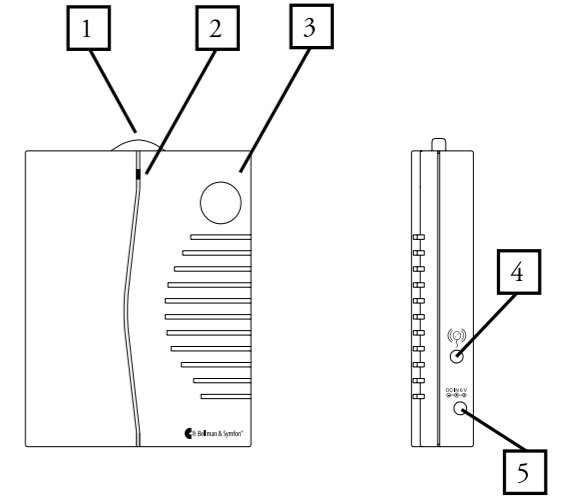
Portable Receiver User manual

bellman.com

The Visit product range

This product is part of the Visit alerting system that helps people with hearing loss to notice signals in their home. It consists of a set of transmitters and receivers that all work together, so you can always add a new product.

Portable receiver

Model BE1033

EN

ES

FR

User manual, Bellman Visit Portable Receiver

EN

Bellman Visit Receiver is part of the Visit System. The receiver receives signals from different Visit transmitters and signals when the door bell or telephone rings. You can carry the receiver in the apartment or garden and place it where you are for the time being.

Getting started...

- 1 Open the battery panel on the back of the receiver and insert four 1.5 V batteries of the LR14 type.
- 2 Press in the test button on the bottom of receiver for a few seconds. The indicator lamp on the receiver will shine green and the receiver will emit an audible sound. Adjust the volume with the large red volume control.
- 3 Check that the receiver receives signals from your Bellman Visit receiver. You can do this by pressing the test button on the respective transmitter so that the receiver gives a signal.

Miscellaneous

Indicator lamp

On the front of the receiver there is an indicator lamp which shines green when the receiver is activated by a transmitter. This shows that the receiver functions. The indicator lamp flashes red when the batteries need changing.

When setting the radio channel the indicator lamp shines alternately red and green. The receiver confirms the changed radio channel by allowing the indicator lamp to shine both red and green simultaneously while emitting a sound signal.

Batteries

Bellman Visit Portable Receiver is powered by four 1.5V alkaline batteries of the LR14 type, which last for one to two years. They are inserted under the battery panel on the back of the receiver.

When the batteries begin to get low the indicator lamp flashes red. It is also possible to use a mains adapter instead of batteries. The mains adapter is connected to the input marked 7.5V DC on the side of the receiver.

Range

The range between the receiver and transmitters is approx. 30 metres but is influenced by the thickness of the walls and wall materials in the house or apartment. Thick concrete and metal walls limit the range.

Radio channel

On delivery all units are set to the same radio channel. If you change a channel in a transmitter you must also change the channel in the receiver, which is done as follows:

Hold the receiver in front of the transmitters; press in the test button on the receiver for approx. five seconds until the indicator lamp on the receiver shines alternatively red and green; now press in the transmitter's test button so that the receiver's indicator lamp shines red and green simultaneously while emitting a sound signal. This confirms that the channel is changed.

Adjustments

You can adjust the volume with the large red volume control on the top of the receiver. The sound signal on the front can be switched off with the sliding switch located on the back of the receiver.

Trouble shooting

A fault is not always serious. Follow the trouble shooting chart below before calling for service.

Symptom	Action
The receiver does not receive signals from the door or telephone transmitter.	<ul style="list-style-type: none"> ▪ Press in the receiver's test button for a few seconds. The test lamp should then shine green and the receiver should emit a sound signal. If it shines red or not at all the batteries in the receiver must be changed.
The receiver signals although no one has rung the door bell or telephone.	<ul style="list-style-type: none"> ▪ Change the radio channel first in all Visit transmitters and then in the receiver.

EN

Product description

- | | | | |
|---|-------------------|---|---------------------------|
| 1 | Volume control | 5 | Connection, mains adapter |
| 2 | Indicator lamp | 6 | Battery panel |
| 3 | Flash signal | 7 | Sound signal on/off |
| 4 | Bed-shaker output | 8 | Test button |

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FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

IC compliance statement

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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Note of Disposal

WEEE Directive (2012/19/EU)

The crossed-out bin symbol on a product and/or packing indicates that this product should not be discarded with general domestic waste, but must be collected separately at the end of its service life. Please observe the legal regulations on electrical waste segregation and disposal applicable in your country.