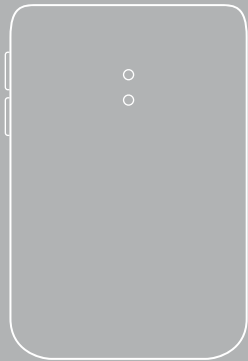
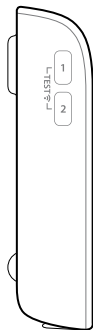
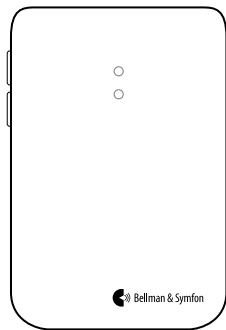
BE1411 Visit door transmitter

---

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---

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---

**44** User manual

---

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---

**84** Käyttöopas

---

**104** Mode d'emploi

---

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---

**144** Brukermanual

---

**164** Bruksanvisning

---

**185** Appendix (EN)

---

DE

DK

EN

ES

FI

FR

NL

NO

SE

App

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## Read this first

This booklet guides you on how to use and maintain your new transmitter. Ensure you read this booklet carefully, including the **Warnings** section. This will help you get the most out of your product. If you have additional questions, contact your hearing care professional.

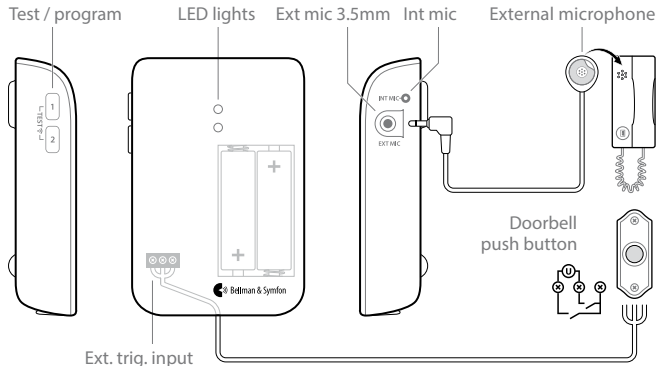
**Intended purpose** The intended purpose of the Visit system is to alert deaf and hard of hearing people of important signals in their home.

**Intended user** A person with mild to severe hearing loss or deafness.

**Intended user group** The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.

**Principle of operation** The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.


## Overview

EN


## Basic installation – single sound source

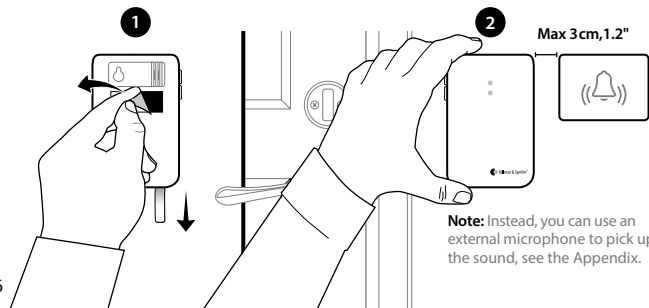- 1 Remove the battery pull tab to start the unit. Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 2 Mount the door transmitter to the left of the doorbell's sound source, as close as possible. You can also use the supplied screw and plug.

## Installation – intercom and electronic doorbell

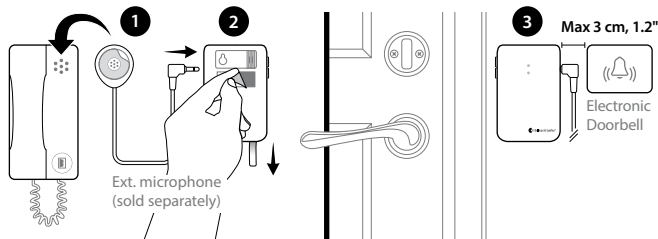- 1 Remove the protective film from the external microphone (sold separately) and attach it to the intercom's speaker. Connect it to the transmitter.
- 2 Pull the battery tab to start the transmitter. Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 3 Mount the door transmitter to the left of the doorbell's sound source, as close as possible. Note that the intercom and doorbell must be at least 25 cm, 10" apart to avoid sound interference.


## Installation – intercom and mechanical doorbell

If you have an intercom and a mechanical doorbell, you may need two door transmitters, i.e. one for each sound source.

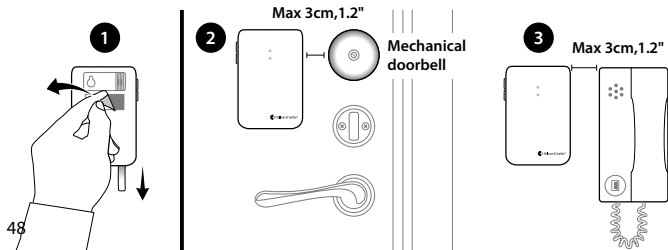
- 1 Pull the battery tab to start each door transmitter. Clean the wall surface with the wet wipe and remove the protective film from the Velcro.
- 2 Mount one of the door transmitters to the left of the doorbell and the other to the left of the intercom's speaker, as close to the sound sources as possible.

## Optimizing the doorbell detection


The next step is to teach the transmitter to recognize your doorbell.

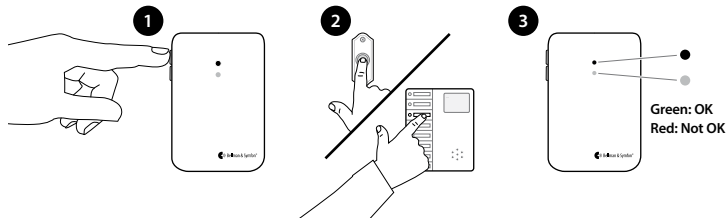- 1 Press and hold button 1 until the top LED start to blink. Release it to start the recording.
- 2 **For door buzzers:** Press and hold the doorbell for at least 8 seconds.  
**For door chimes:** Press the doorbell at least 8 times.
- 3 Press button 1 on the transmitter to stop the recording. If the LED lights up in red, you need to repeat steps 1 and 2. Following the same steps, you can use button 2 to record the sound of your intercom.

## Testing the signal

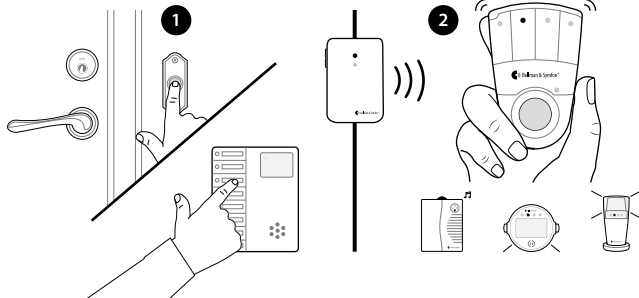- 1 Ring the doorbell or intercom.
- 2 The LED on the Visit receiver lights up in green and the receiver starts to sound, flash or vibrate, depending on the model. If it doesn't react, see **Troubleshooting**.

## General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

### Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years.
- Note that alarms and notifications can be missed if batteries run out.
- Use only battery types that are specified in this booklet.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the user guide, may cause harmful interference to radio communication.

- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- Protect cables from any potential source of damage.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

### **i** Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it
- Do not use the device in areas where electronic equipment is prohibited.

- The device may only be repaired by an authorized service center.
- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- Without permission granted by the NCC, no company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

### **Operating conditions**

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

## Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

## Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

## Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as

fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

## Configuration options

This transmitter can be configured with the following Visit receivers and accessories:

### Compatible Visit receivers

- BE1450 Visit portable receiver
- BE1441 Visit flash receiver
- BE1442 Visit flash receiver
- BE1580 Visit alarm clock receiver
- BE1470 Visit pager receiver
- BE1560 Visit wrist receiver

### Available accessories

- BE9199 External microphone 2.5 m
- BE9200 External microphone 0.75 m

For detailed product information, see the corresponding user manual.

## Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below. The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit **bellman.com** for contact information.

---

Radio Equipment Directive (RED)

---

Medical Device Regulation (MDR)

---

Waste Electrical & Electronic Equipment (WEEE)

---

RoHS Directive

---

REACH Regulation

---

EC Battery Directive

---

## Regulatory symbols

With this symbol, Bellman & Symfon confirms that the product meets the Medical Device Regulation EU 2017/745.

This symbol indicates the manufacturer's serial number so that a specific medical device can be identified. It's available on the product and gift box.

This symbol indicates the manufacturer's catalogue number so that the medical device can be identified. It's available on the product and gift box.

This symbol indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.

This symbol indicates that the user should consult the instruction for use.

This symbol indicates that it is important for the user to pay attention to the relevant warning notices in the user guide.

This symbol indicates important information for handling and product safety.

Temperature during transport and storage:  $-10^{\circ}$  to  $50^{\circ}$  C,  $14^{\circ}$  –  $122^{\circ}$  F  
Temperature during operation:  $0^{\circ}$  to  $35^{\circ}$  C,  $32^{\circ}$  to  $95^{\circ}$  F

Humidity during transportation and storage:  $<90\%$ , non-condensing  
Humidity during operation:  $15\%$  –  $90\%$ , non-condensing

Atmospheric pressure during operation, transportation and storage:  
 $700$  hpa to  $1060$  hpa

**Operating conditions**

This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in the user guide.

With this CE symbol, Bellman & Symfon confirms that the product meets EU standards for health, safety, and environmental protection as well as the Radio Equipment Directive 2014/53/EU.

This symbol indicates that the product shall not be treated as household waste. Please hand over your old or unused product to the applicable collection point for the recycling of electrical and electronic equipment or bring your old product to your hearing care professional for appropriate disposal.

## ISO Certification of legal manufacturer

Bellman is certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.  
SS-EN ISO 9001 Cert. No.: CN19/42071, SS-EN ISO 13485 Cert. No.: CN19/42070

### Certification Body

SGS United Kingdom Ltd, Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

# Troubleshooting

If	Try this
The LEDs blink in orange every minute	<ul style="list-style-type: none"><li>▪ Change the batteries. Only use 1.5 V AA (LR6) lithium or alkaline batteries.</li></ul>
The LEDs blink in orange every second	<ul style="list-style-type: none"><li>▪ There are other competing sound sources around the door transmitter. Switch them off or turn them down.</li><li>▪ Disconnect the External microphone to make sure it is not faulty.</li></ul>
The LED on the transmitter lights up when I press the doorbell or intercom - but the receiver is not activated	<ul style="list-style-type: none"><li>▪ Check the transmitter batteries and the receiver batteries and connections.</li><li>▪ Move the receiver closer to the transmitter to make sure it's within radio range.</li><li>▪ Check that the door transmitter and the receiver are set to the same radio key. For more information, see <b>Changing the the radio key</b> in the Appendix.</li></ul>

If	Try this
The LED on the transmitter doesn't light up when I ring the doorbell	<ul style="list-style-type: none"><li>▪ Make sure you have mounted the transmitter to the left of the doorbell sound source, no more than 3 cm apart.</li><li>▪ If you are using an external microphone, make sure its connected. If you are using an intercom, make sure that the distance between the intercom and transmitter is at least 25 cm or that the internal microphone is turned off.</li><li>▪ Try again to teach the transmitter to recognize the sound of your doorbell, see <b>Optimizing the doorbell detection</b>.</li><li>▪ If the doorbell signal varies a lot, change to magnetic detection, see <b>Electromagnetic detection</b> in the Appendix.</li></ul>
The receiver is activated for no reason	<ul style="list-style-type: none"><li>▪ There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see <b>Changing the radio key</b> in the Appendix.</li></ul>

## Technical specifications

<b>Dimensions</b>	100 × 65 × 27 mm, 4.0 × 2.6 × 1.1"
<b>Weight</b>	100 – 118 g, 3.5 – 4.2 oz. depending on batteries
<b>Power and battery</b>	Battery: 2 × 1.5 V AA (LR6) lithium or alkaline Power consumption: Active < 70 mA, Idle position < 15 µA Operating time: Alkaline ~ 5 years, Lithium ~ 10 years
<b>Activation</b>	Via the test buttons, the internal or external microphone, via electromagnetic detection or the external trigger input.
<b>Radio frequency</b>	868.30MHz
<b>Max output power</b>	4.7dbM
<b>Coverage</b>	50 - 250 m depending on the characteristics of the building.
<b>No. of radio keys</b>	64 as standard. Up to 256 with optional software.
<b>In the box</b>	<ul style="list-style-type: none"><li>▪ BE1411 Visit door transmitter</li><li>▪ 1.5V AA alkaline batteries</li><li>▪ Velcro, screws and plugs</li></ul>

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## Appendix (EN)

---

**186** Installation using the External microphone

---

**187** Hardwiring your doorbell

---

**188** Testing the signal

---

**190** Default signal pattern

---

**190** Changing the signal pattern

---

**191** Using electromagnetic detection

---

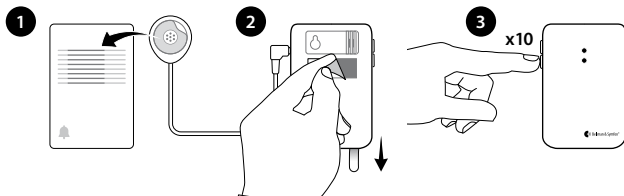
**192** Changing the radio key

---

## Alternative installation – single sound source


### Using the external microphone accessory

- 1 Connect the external microphone (sold separately) to the transmitter. Remove the protective film and attach it to the doorbell's sound source.
- 2 Pull the battery tab to start the transmitter. Remove the protective film from the Velcro and mount the transmitter on the wall.
- 3 Within 2 min, press the lower button 10 times to turn off the internal microphone. The LEDs will blink 3 times in red to confirm.  
**Note:** If you exceed 2 minutes, you need to restart the transmitter by removing the batteries and putting them back in again.

## Installation – hardwiring your doorbell

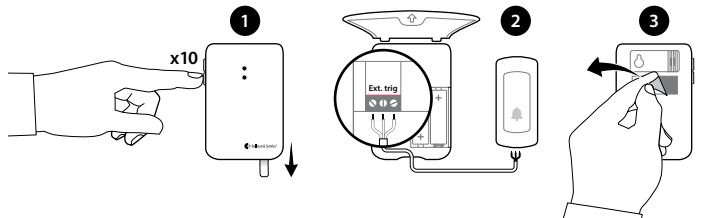- 1 Pull the battery tab to start the transmitter. Within 2 min, press the lower button 10 times to turn off the internal mic. The LEDs will blink 3 times to confirm.
- 2 Remove the front cover to access the screw terminal. There are 3 connection points:
  - Use **1** and **2** to connect an **active** switch, like a relay with current (2–30VDC, polarity independent or 2–24VAC, 5 - 150Hz).
  - Use **2** and **3** to connect a **passive** switch, like a push button.
- 3 Put the front cover back, remove the protective film from the Velcro and mount the door transmitter on the wall.


## Testing the signal

### Using the test buttons

- 1 Press both test buttons simultaneously on the transmitter. The top LED lights up in green to show that a radio signal is being transmitted.
- 2 The green Visit LED on the receiver lights up to show that the signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

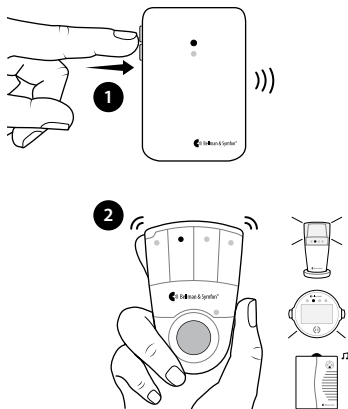**Note:** If you have changed the signal pattern, it will react in accordance with the tables on page 191 to 192.


## Testing the signal

### Using the doorbell or intercom

- 1 Press the button on the doorbell or intercom. The LED on the transmitter lights up in green to show that the sound is detected.
- 2 The green Visit LED on the receiver lights up to show that the radio signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

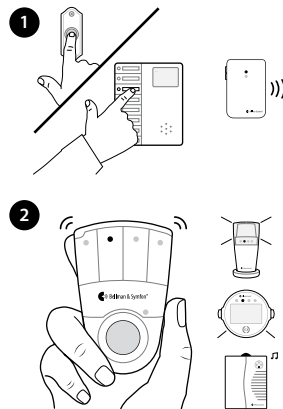**Note:** If you have changed the signal pattern, it will react in accordance with the tables on page 191 to 192.

## Default signal pattern

When the transmitter or a connected accessory is activated, the following happens:

- 1 The LED on the transmitter lights up in green to show that it is signalling the receiver.
- 2 The Visit LED on the receiver lights up in green and it starts to sound, flash or vibrate with a certain pace, called signal pattern. The transmitter determine the signal pattern and the default is as follows:

Transmitter		Visit receiver			
Source	LED	LED	Sound	Vibration	Flash
Internal mic.	Green, top	Green light	1x door chime, low	Slow	Yes
External mic.	Green, bottom	Green blinks	2x door chime, low	Slow	Yes
Doorbell con.	Green, top	3x green blinks	1x door chime, high	Slow	Yes

## Changing the signal pattern

The transmitter controls the signal pattern. Open the transmitter front cover and move the signal switches according to the tables to change it:

### Transmitter

Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1 x door chime, low	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	Green blink	2 x door chime, high	Slow ■□□□	Yes
	Connected doorbell	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Int. mic. / learned signal 1 / test	2 x green blinks	2 x door chime low	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Connected doorbell	Green blinks	2 x door chime, high	Slow ■□□□	Yes
	Int. mic. / learned signal 1 / test	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	2 x green blinks	2 x door chime, low	Slow ■□□□	Yes
	Connected doorbell	3 x orange blinks	Baby melody	Fast ■■■■■■	Yes
	Int. mic. / learned signal 1 / test	Green blinks	2 x door chime, high	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	Green light	1 x door chime, low	Slow ■□□□	Yes
	Connected doorbell	Orange blinks	Baby melody	Fast ■■■■■■	Yes

## Transmitter

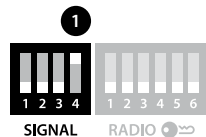
## Receiver signal pattern

Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1 x door chime, low	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	3 x green blinks	1 x door chime, high	Slow	■□□□ Yes
	Connected doorbell	2 x green blinks	2 x door chime, low	Slow	■□□□ Yes
	Int. mic. / learned signal 1 / test	Green light	1 x door chime low	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	Green blinks	2 x door chime, high	Slow	■□□□ Yes
	Connected doorbell	Yellow blinks	2 x ring signal, high	Medium	■□□□ Yes
	Int. mic. / learned signal 1 / test	3 x green blinks	1 x door chime, high	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	Green light	1 x door chime, low	Slow	■□□□ Yes
	Connected doorbell	Red + orange blinks	Emergency siren	Long	■□□□ Yes
	Int. mic. / learned signal 1 / test	Green blinks	2 x door chime, high	Slow	■□□□ Yes
	Ext. mic. / learned signal 2	2 x green blinks	2 x door chime, low	Slow	■□□□ Yes
	Connected doorbell	Red blinks	Fire alarm	Long	■□□□ Yes

## Using electromagnetic detection

The door transmitter can be set to detect electromagnetic fields emitted by electromechanical doorbells. Here is how it's done:

- Activating electromagnetic detection**  
 Move the 4th signal-switch (1) to the Up=On position.
- Deactivating electromagnetic detection**  
 Move the 4th signal-switch to the Down=Off position.





## Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located under the transmitter cover.

### Here is how you change the radio key:

- 1 Carefully open the front cover on the transmitter. Move any radio key switch to the up (on) position to change the radio key.
- 2 Press and hold the test/function button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press both test buttons simultaneously on the transmitter within 30 s to send the new radio key.
- 4 All Visit LEDs on the receiver blink 5 times to show that the radio key has been changed. It then returns to normal mode.

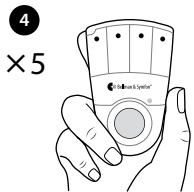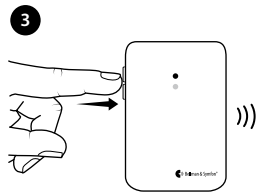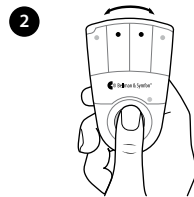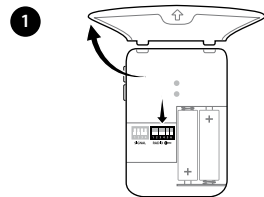Please note that all Visit units must be set to the same radio key in order to operate as a group.

## Manufacturer

Bellman & Symfon Group AB  
Södra Långebergsgatan 30  
436 32 Askim Sweden  
Phone +46 31 68 28 20  
E-mail [info@bellman.com](mailto:info@bellman.com)  
[bellman.com](http://bellman.com)

Revision: BE1411\_209MAN1.0

Date of issue: 2022-08-31

TM and © 2022 Bellman & Symfon AB.

All rights reserved.