Bellman & Symfon®

Bellman & Symfon®


BE9159-P02 Neck Loop,  
long cable

BE9161-P02 Neck Loop,  
short cable

# Neck Loop

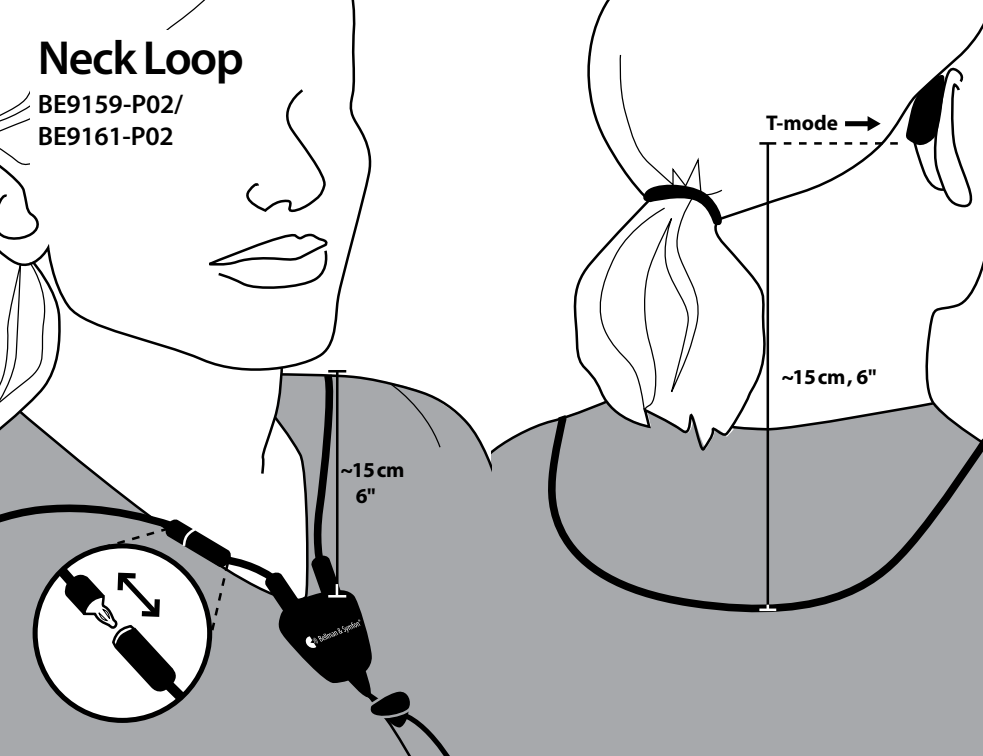BE9159-P02/  
BE9161-P02


~15 cm  
6"

T-mode →


~15 cm, 6"


Polhemus & Synchro

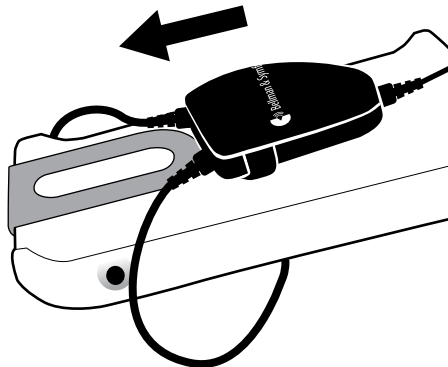- EN** Use with Domino or Mino  
**DE** Verwendung mit Domino oder Mino  
**DK** Brug med Domino eller Mino  
**ES** Utilízelo con Domino o Mino  
**FI** Käyttö Dominon tai Minon kanssa  
**FR** Utiliser avec Domino ou Mino

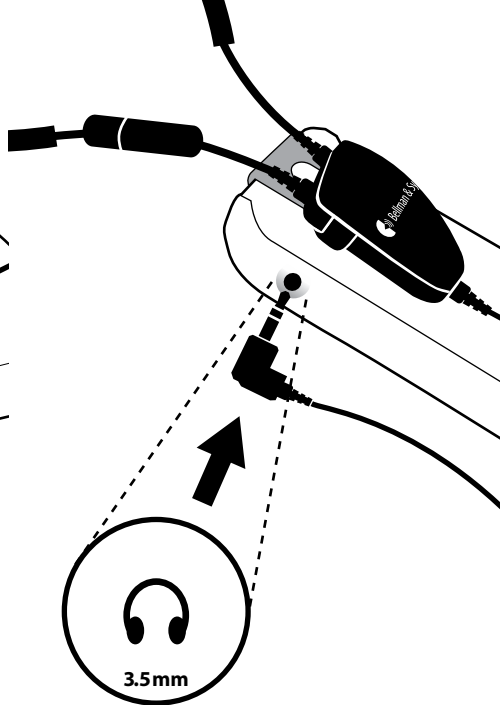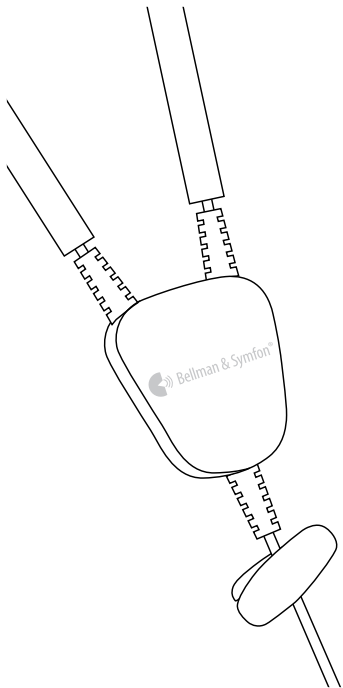- IT** Utilizzi con Domino o Mino  
**NL** Gebruiken met Domino of Mino  
**NO** Brukes med Domino eller Mino  
**PL** Proszę używać z Domino lub Mino  
**SE** Använd med Domino eller Mino  
**SK** Používanie s Domino alebo Mino

## BE9159-P02 Neck Loop, long cable BE9161-P02 Neck Loop, short cable

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SK

## User information

- This Neck Loop from Bellman & Symfon is to be used with hearing aids equipped with Telecoil.
- It is optimized for the Maxi, Maxi Pro, Sonido, Mino personal amplifiers and the Domino listening system.
- The Neck Loop can also be used with smart devices, TVs, radios or other equipment that features a 3.5 mm stereo headphone jack intended for low impedance headphones.
- It is discreet and comfortable, and can easily be worn under your shirt. It also incorporates a breakaway neck strap for increased safety.
- **Important!** If you have a pacemaker, we recommend you check with your general practitioner or cardiologist before using a Neck Loop.

## Read this first

This booklet guides you on how to use and maintain your new Neck Loop. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

## About the BE9159-P02 / BE9161-P02 Neck Loop

### Intended purpose

The intended purpose of the audio product family is to amplify the volume and improve the speech intelligibility during conversations and TV-listening. It can also be used with other sound sources.

### Intended user group

The intended user group consists of people of all ages experiencing mild to severe hearing loss who need sound amplification in different situations.

### Intended user

The intended user is a person with mild to severe hearing loss who needs sound amplification.

### Principle of operation

The audio product family consists of several amplifiers and sound transmitters which have been specifically developed to provide sound enhancement even in demanding situations. Depending on the assigned function of the specific amplifier or sound transmitter, different microphones can be used to pick up direct sound or enhance the ambient sound.

This device will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.

## General Warnings

This section contains important information about safety, handling and operating conditions. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

### Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Protect cables from any potential source of damage.
- If you have a pacemaker, we recommend you check with your general practitioner or cardiologist before using a Neck Loop.

### Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- Do not drop your device. Dropping onto a hard surface can damage it.

### Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

### Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

## Warranty conditions

Bellman & Symfon guarantees this product for 90 days from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

## Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation (EU) 2017/745. The full text of the declaration of conformity can be obtained from the Bellman & Symfon website [bellman.com](http://bellman.com).

## Symbol explanation

- 
- |   |   |
|---|---|
|  | Indicates the item is a medical device.   |
|  | Indicates a carrier that contains unique device identifier information.   |
|  | Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.   |
|  | Indicates the manufacturer's serial number so that a specific medical device can be identified.   |
|  | Indicates the manufacturer's catalogue number so that the medical device can be identified.   |
|  | Indicates the medical device manufacturer.  |
|  | This symbol indicates the need for the user to consult the instructions for use.  |
|  | This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences. |
|  | To identify a place where information may be found, especially in an emergency.   |
|  | Indicates the temperature limits to which the medical device can be safely exposed.   |
|  | Indicates the range of humidity to which the medical device can be safely exposed.  |
-

## Technical specifications

<b>Neck loop diameter</b>	22 cm
<b>Weight</b>	BE9159-P02: 62g BE9161-P02: 58g
<b>Cable length</b>	BE9159-P02: 90cm BE9161-P02: 15cm
<b>Connectors</b>	3.5mm tele plug (stereo) gold plated connector 90 degrees angle (Break-away connector on the neck loop).
<b>Load impedance</b>	2×5 Ω
<b>Magnetic output</b>	1500mA/m @ 15cm distance and 2×50mW input signal
<b>Operation</b>	Temperature: 0 to 35 °C Humidity: 15% to 90% (non condensing)

### Configuration

This Neck Loop can be configured with the following personal amplifiers and listening systems:

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All rights reserved.
Bellman & Symfon UK Limited
Audiocare AG
Bellman & Symfon Group AB
Bellman & Symfon AB.