
BE2030  
MINO PERSONAL AMPLIFIER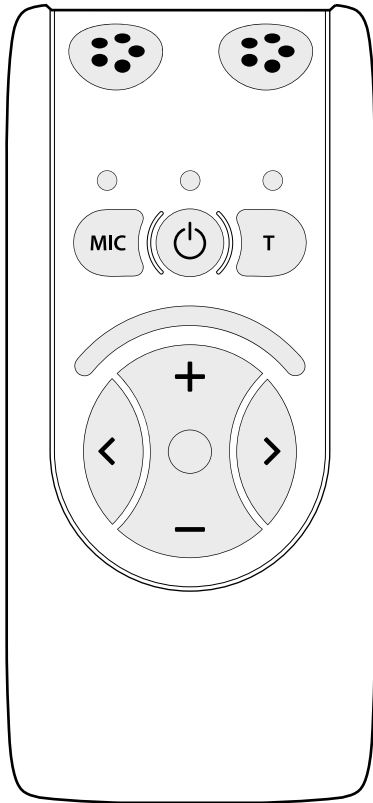

## BE2030 Mino personal amplifier

04	User manual
18	Bedienungsanleitung
32	Brugervejledning
46	Manual de usuario
60	Käyttöopas
74	Mode d'emploi
88	Gebruiksaanwijzing
102	Brukermanual
116	Bruksanvisning
130	Appendix (EN)

EN

DE

DK

ES

FI

FR

NL

NO

SE

App

## Les dette først

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### Sertifiseringsorgan

SGS United Kingdom Ltd

Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

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
## Settings

Bellman Audio Mino has more settings than those you can access in user mode. By going into the Settings Menu you can adjust settings that you do not normally need to change or which are adjusted when first used and never need to be changed again.

### Settings Menu

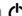How to get into the Settings Menu.

- Start the unit by pressing the On/Off button  (4).
- Hold down the On/Off button  (4) and press down the Volume button  (7) within 1 second until the **T** LED (9) starts to blink.
- When the **T** LED (9) starts to blink, you can release both the buttons.
- Press and hold down the **T** button (12) within 2 seconds of the moment the **T** LED (9) starts to blink.
- Release the **T** button (12) immediately once the **T** LED (9) has gone out.


You are now in Balance Control mode.

### Overview of the Settings Menu

In the Settings Menu you can switch between the different settings by pressing the **T** button (12) or the

**MIC** button (3). Each press of the **T** button moves you clockwise on to the next setting in the list and the **MIC** button takes you anti-clockwise to the previous setting in the list.

When you have finished adjusting your settings, give a short press on the On/Off button  (4) to return to normal user mode.

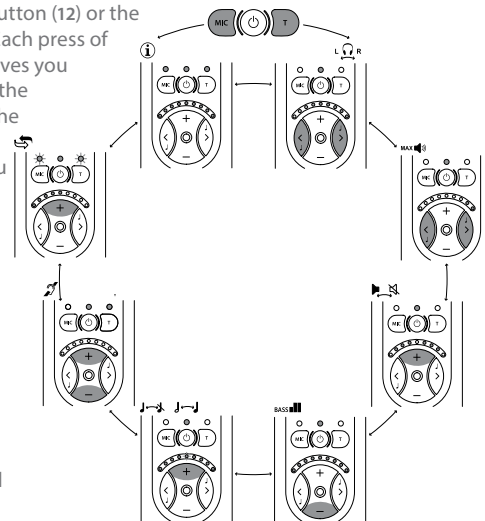App

## Balance control

The balance between the right and left-hand channels can be set as preferred in 5 dB steps.

Press the **◀** button (6) to adjust the balance on the left and the **▶** button (6) to adjust the balance on the right.

The standard setting is centered balance.

## Basic attenuation

Different headphones have different sensitivity levels and you can compensate for this with basic attenuation. What you actually set is the attenuation you require at the maximum power output you want to have (which is therefore 0 dB attenuation). This setting is useful when you want to adjust the maximum volume in user mode for a pair of headphones or in-ear phones with a higher sensitivity than the BE9122 Bellman Stereo Headphones or, for example, if you want to limit the volume for a child.

Use the **▶** button (6) to reduce the basic attenuation (increase the volume) and the **◀** button (6) to increase the basic attenuation (reduce the volume). The basic attenuation is adjusted in increments of 3 dB.

The standard setting is 0 dB attenuation, which corresponds to the maximum output signal.

## Stereo/Mono

The line-in signal can be rendered in either Stereo or in Mono. The intelligibility of speech can be increased in certain listening situations, such as when watching TV, if the signal is in Mono.

Press the **+** (7) button to select Stereo.

Press the **-** (7) button to select Mono.

The standard setting is Stereo.

## Bass reduction

Low-frequency interference such as fans and road noise can be tiring if you are exposed to it for a long period. By activating bass reduction, low-frequency interference and sounds are reduced.

Press the **+** (7) button to select Bass Reduction.

Press the **-** (7) button to deactivate Bass Reduction and get level frequency response.

The standard setting is deactivated Bass Reduction.

## Adjusting the Tone Control

The Tone Control Blocking function has been developed in case you want to make it easier for the user to use Mino. This setting can be used to choose whether or not the user should be able to adjust the Tone Control. If you choose to switch off the tone control buttons, you can preset the Tone Control that will be used in user mode.

## Activation and deactivation of Tone Control in User Mode

Press the **+** (7) button to activate the Tone Control buttons. Press the **-** (7) button to deactivate them.

The standard setting is Tone Control activated.

## Presetting the Tone Control

Press the **◀** button (6) and the **▶** button (6) to preset the Tone Control to the level that will be used in user mode. The standard setting is Tone Control centered (level frequency response)

*Please note that if the Tone Control is activated, the preset value will be changed as soon as the Tone Control is changed in user mode.*

## Blocking the T position

The **T** position can be blocked if you know that the **T** position will not be used. This setting is for those who will never use the **T** position and want to avoid selecting this position by mistake.

Press the **+** (7) button to activate the **T** position.

Press the **-** (7) button to deactivate the **T** position.

The standard setting is activated **T** position.

## Resetting to the factory setting

You can use this to reset all settings to the standard settings the product was delivered with from the factory.

Press and hold down the **+** button (7) until all the LEDs on the display (5) are lit. After resetting is completed, the display will blink for a short period.

## Version information

The product version information is presented here via a pattern of lit and blinking LEDs. This internal information is only intended as an aid for servicing and support.

## Testing the internal microphones

- 1 Switch on the Mino and make sure that the Battery status indicator (1) lights up green.
- 2 Connect the headphones to the Headphones output (11) and scrape carefully on the round Microphone (8) with your finger and check that there is a scraping sound in the headphones.
- 3 Press the **MIC** button (3) and make sure that the Microphone indicator (2) changes colour and lights up orange.
- 4 Carefully scrape the small microphone hole on the top (1) and the round microphone (8) with your finger and check that there is a scraping sound in the headphones.
- 5 Press the buttons (6, 7) and check that the Volume and Tone LED indicator changes.

## Testing the external sound

- 1 Press the T button (12) and make sure that the External Sound LED (9) lights up green.
- 2 Hold the unit against an electrical watch with mechanical hands (with built-in battery) and check that it ticks.
- 3 Connect an external sound source to the External Sound Source Input (10) and check that the sound from the sound source is heard in the headphones. The External Sound Source Input (10) can be tested, for example, by connecting the TV or a radio to the input via a BE9126 audio cable. The unit automatically switches to an external sound source when a connector is connected to the External Sound Source Input (10).

## Troubleshooting

If	Try
The sound is louder in one ear than in the other.	▪ Adjust the Balance in the <b>Settings Menu</b> .
The sound volume is low even though the volume is set to max.	▪ Adjust the Basic Attenuation in the <b>Settings Menu</b> .
The sound is only in Mono even though it is connected via a stereo cable to a sound source which provides stereo sound.	▪ Change to Stereo in the <b>Settings Menu</b> .
The bass reproduction is weak.	▪ Switch off Bass Reduction in the <b>Settings Menu</b> .
The Tone Control cannot be adjusted.	▪ Use the <b>Settings Menu</b> to activate the Tone Control buttons or to adjust the sound.
It is not possible to select T position.	▪ Use the <b>Settings Menu</b> to activate the T position.
The unit is acting strangely.	▪ Reset the unit to the factory settings in the <b>Settings Menu</b> .
Humming occurs when connecting the stereo cable while charging.	▪ Only use the supplied charger.
Interference is heard when used in some environments	▪ Move the product away from the suspected source of radio frequency energy, for example a transmitter.

# DESIGN FOR EARS™

## Manufacturer

Bellman & Symfon Group AB  
Södra Långebergsgatan 30  
436 32 Askim, Sweden  
Phone: +46 31 68 28 20  
E-mail: [info@bellman.com](mailto:info@bellman.com)  
[bellman.com](http://bellman.com)

Revision: BE2030\_117MAN2.0

Date of issue: 2024-02-19

TM and © 2024 Bellman & Symfon AB. All rights reserved.