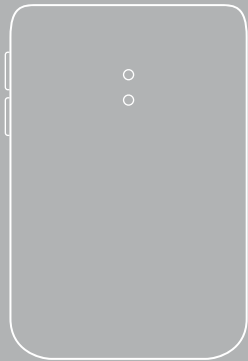
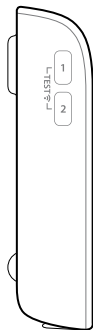
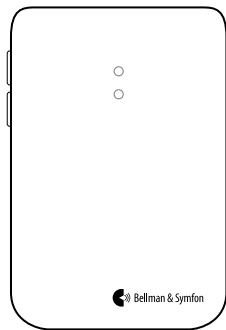
BE1411 Visit door transmitter

---

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---

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---

**44** User manual

---

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---

**84** Käyttöopas

---

**104** Mode d'emploi

---

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---

**144** Brukermanual

---

**164** Bruksanvisning

---

**185** Appendix (EN)

---

DE

DK

EN

ES

FI

FR

NL

NO

SE

App

## Les dette først

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### Sertifiseringsorgan

SGS United Kingdom Ltd, Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

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## Appendix (EN)

---

**186** Installation using the External microphone

---

**187** Hardwiring your doorbell

---

**188** Testing the signal

---

**190** Default signal pattern

---

**190** Changing the signal pattern

---

**191** Using electromagnetic detection

---

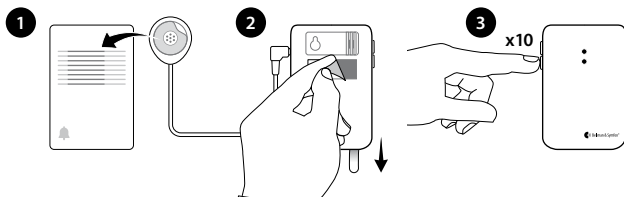
**192** Changing the radio key

---

## Alternative installation – single sound source


### Using the external microphone accessory

- 1 Connect the external microphone (sold separately) to the transmitter. Remove the protective film and attach it to the doorbell's sound source.
- 2 Pull the battery tab to start the transmitter. Remove the protective film from the Velcro and mount the transmitter on the wall.
- 3 Within 2 min, press the lower button 10 times to turn off the internal microphone. The LEDs will blink 3 times in red to confirm.  
**Note:** If you exceed 2 minutes, you need to restart the transmitter by removing the batteries and putting them back in again.

## Installation – hardwiring your doorbell

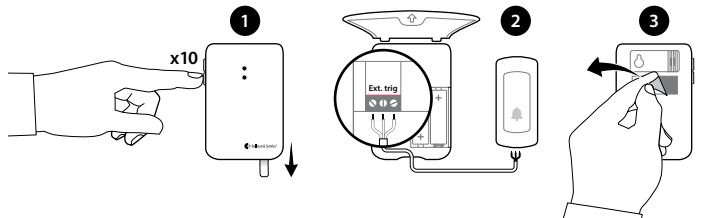- 1 Pull the battery tab to start the transmitter. Within 2 min, press the lower button 10 times to turn off the internal mic. The LEDs will blink 3 times to confirm.
- 2 Remove the front cover to access the screw terminal. There are 3 connection points:
  - Use **1** and **2** to connect an **active** switch, like a relay with current (2–30VDC, polarity independent or 2–24VAC, 5 - 150Hz).
  - Use **2** and **3** to connect a **passive** switch, like a push button.
- 3 Put the front cover back, remove the protective film from the Velcro and mount the door transmitter on the wall.


## Testing the signal

### Using the test buttons

- 1 Press both test buttons simultaneously on the transmitter. The top LED lights up in green to show that a radio signal is being transmitted.
- 2 The green Visit LED on the receiver lights up to show that the signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

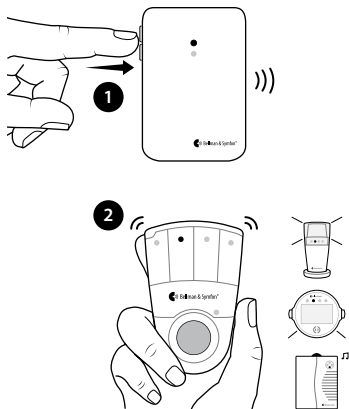**Note:** If you have changed the signal pattern, it will react in accordance with the tables on page 191 to 192.


## Testing the signal

### Using the doorbell or intercom

- 1 Press the button on the doorbell or intercom. The LED on the transmitter lights up in green to show that the sound is detected.
- 2 The green Visit LED on the receiver lights up to show that the radio signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

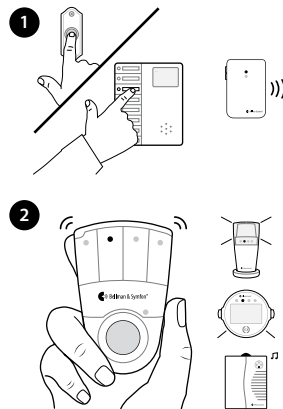**Note:** If you have changed the signal pattern, it will react in accordance with the tables on page 191 to 192.

## Default signal pattern

When the transmitter or a connected accessory is activated, the following happens:

- 1 The LED on the transmitter lights up in green to show that it is signalling the receiver.
- 2 The Visit LED on the receiver lights up in green and it starts to sound, flash or vibrate with a certain pace, called signal pattern. The transmitter determine the signal pattern and the default is as follows:

| Transmitter   |               | Visit receiver  |                     |           |       |
|---------------|---------------|-----------------|---------------------|-----------|-------|
| Source        | LED           | LED             | Sound               | Vibration | Flash |
| Internal mic. | Green, top    | Green light     | 1x door chime, low  | Slow      | Yes   |
| External mic. | Green, bottom | Green blinks    | 2x door chime, low  | Slow      | Yes   |
| Doorbell con. | Green, top    | 3x green blinks | 1x door chime, high | Slow      | Yes   |

## Changing the signal pattern

The transmitter controls the signal pattern. Open the transmitter front cover and move the signal switches according to the tables to change it:

### Transmitter

| Switch  | Source                              | LED               | Sound                | Vibration   | Flash |
|---|-------------------------------------|-------------------|----------------------|-------------|-------|
|  | Int. mic. / learned signal 1 / test | Green light       | 1 x door chime, low  | Slow ■□□□   | Yes   |
|   | Ext. mic. / learned signal 2        | Green blink       | 2 x door chime, high | Slow ■□□□   | Yes   |
|   | Connected doorbell                  | 3 x green blinks  | 1 x door chime, high | Slow ■□□□   | Yes   |
|  | Int. mic. / learned signal 1 / test | 2 x green blinks  | 2 x door chime low   | Slow ■□□□   | Yes   |
|   | Ext. mic. / learned signal 2        | 3 x green blinks  | 1 x door chime, high | Slow ■□□□   | Yes   |
|   | Connected doorbell                  | Green blinks      | 2 x door chime, high | Slow ■□□□   | Yes   |
|  | Int. mic. / learned signal 1 / test | 3 x green blinks  | 1 x door chime, high | Slow ■□□□   | Yes   |
|   | Ext. mic. / learned signal 2        | 2 x green blinks  | 2 x door chime, low  | Slow ■□□□   | Yes   |
|   | Connected doorbell                  | 3 x orange blinks | Baby melody          | Fast ■■■■■■ | Yes   |
|  | Int. mic. / learned signal 1 / test | Green blinks      | 2 x door chime, high | Slow ■□□□   | Yes   |
|   | Ext. mic. / learned signal 2        | Green light       | 1 x door chime, low  | Slow ■□□□   | Yes   |
|   | Connected doorbell                  | Orange blinks     | Baby melody          | Fast ■■■■■■ | Yes   |

## Transmitter

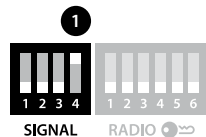
## Receiver signal pattern

| Switch  | Source                              | LED                 | Sound                 | Vibration | Flash    |
|---|-------------------------------------|---------------------|-----------------------|-----------|----------|
|  | Int. mic. / learned signal 1 / test | Green light         | 1 x door chime, low   | Slow      | ■□□□ Yes |
|   | Ext. mic. / learned signal 2        | 3 x green blinks    | 1 x door chime, high  | Slow      | ■□□□ Yes |
|   | Connected doorbell                  | 2 x green blinks    | 2 x door chime, low   | Slow      | ■□□□ Yes |
|  | Int. mic. / learned signal 1 / test | Green light         | 1 x door chime low    | Slow      | ■□□□ Yes |
|   | Ext. mic. / learned signal 2        | Green blinks        | 2 x door chime, high  | Slow      | ■□□□ Yes |
|   | Connected doorbell                  | Yellow blinks       | 2 x ring signal, high | Medium    | ■□□□ Yes |
|  | Int. mic. / learned signal 1 / test | 3 x green blinks    | 1 x door chime, high  | Slow      | ■□□□ Yes |
|   | Ext. mic. / learned signal 2        | Green light         | 1 x door chime, low   | Slow      | ■□□□ Yes |
|   | Connected doorbell                  | Red + orange blinks | Emergency siren       | Long      | ■□□□ Yes |
|  | Int. mic. / learned signal 1 / test | Green blinks        | 2 x door chime, high  | Slow      | ■□□□ Yes |
|   | Ext. mic. / learned signal 2        | 2 x green blinks    | 2 x door chime, low   | Slow      | ■□□□ Yes |
|   | Connected doorbell                  | Red blinks          | Fire alarm            | Long      | ■□□□ Yes |

## Using electromagnetic detection

The door transmitter can be set to detect electromagnetic fields emitted by electromechanical doorbells. Here is how it's done:

- Activating electromagnetic detection**  
 Move the 4th signal-switch (1) to the Up=On position.
- Deactivating electromagnetic detection**  
 Move the 4th signal-switch to the Down=Off position.





## Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located under the transmitter cover.

### Here is how you change the radio key:

- 1 Carefully open the front cover on the transmitter. Move any radio key switch to the up (on) position to change the radio key.
- 2 Press and hold the test/function button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press both test buttons simultaneously on the transmitter within 30 s to send the new radio key.
- 4 All Visit LEDs on the receiver blink 5 times to show that the radio key has been changed. It then returns to normal mode.

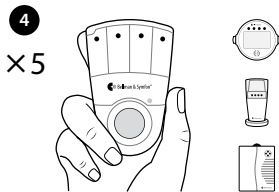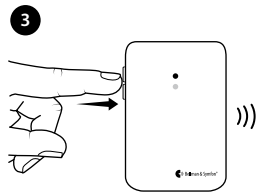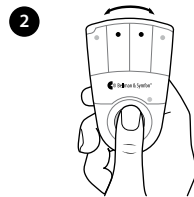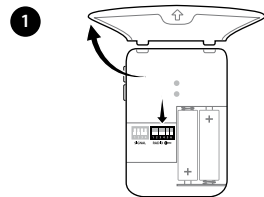Please note that all Visit units must be set to the same radio key in order to operate as a group.

## Manufacturer

Bellman & Symfon Group AB  
Södra Långebergsgatan 30  
436 32 Askim Sweden  
Phone +46 31 68 28 20  
E-mail [info@bellman.com](mailto:info@bellman.com)  
[bellman.com](http://bellman.com)

Revision: BE1411\_209MAN1.0

Date of issue: 2022-08-31

TM and © 2022 Bellman & Symfon AB.

All rights reserved.