

Bellman & Symfon®

BE1420

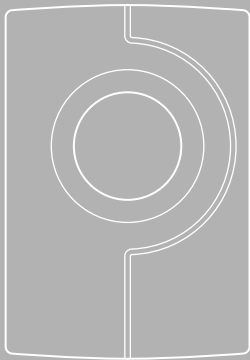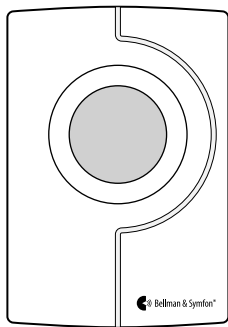Visit push button transmitter

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28 User manual

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52 Käyttöopas

64 Mode d'emploi

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88 Brukermanual

100 Bruksanvisning

112 Appendix

DE

DK

EN

ES

FI

FR

NL

NO

DK

App

Les dette først

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
Sertifiseringsorgan

SGS United Kingdom Ltd, Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

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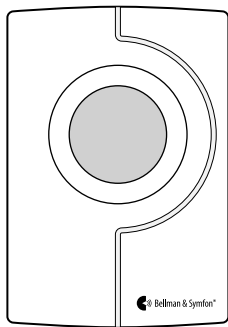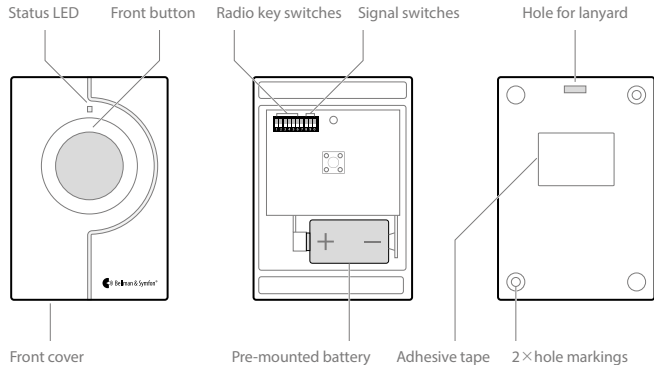
Overview




Using it as a caller button

The transmitter can be worn around your neck and be used as a wireless caller button.

Here is how you use it:

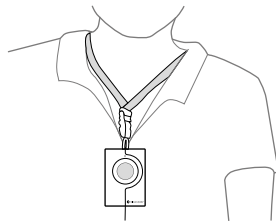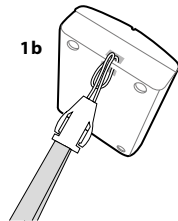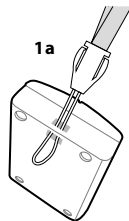- 1 Attach the lanyard to the transmitter.
- 2 Hang the transmitter around your neck.


Using it as a doorbell

The transmitter can also be used as a wireless doorbell. Here is how you set it up:

Mounting with screws

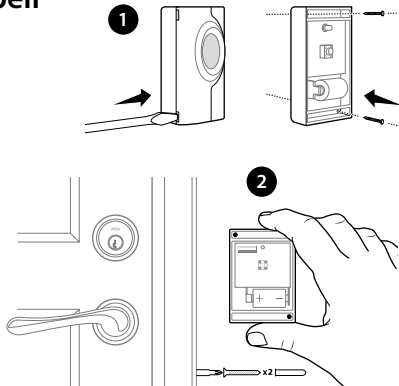- 1 Carefully remove the transmitter front cover and make two holes on the markings.
- 2 Fix the unit to the wall using the supplied screws and put the cover back on.


Using it as a doorbell

You can instead choose to use the supplied adhesive tape. Here is how it is done:

Mounting with adhesive tape

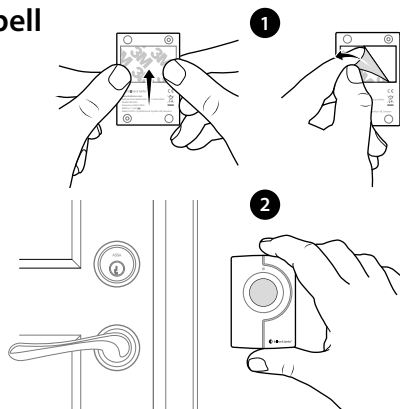- 1 Attach the adhesive tape to the back of the transmitter. Clean the wall with the wet wipe and remove the protective film from the tape.
- 2 Mount the unit in a weather protected area by the front door.



Testing the signal

Using the test buttons

- 1 Press the front button on the transmitter. The LED lights up in green to show that a radio signal is being transmitted.
- 2 The green Visit LED on the receiver lights up to show that the signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

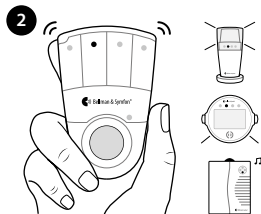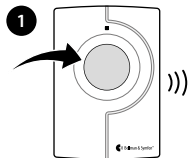Note: If you have changed the signal pattern, it will react in accordance with the tables on page 120 to 121.

Default signal pattern

When the push button transmitter is activated, the following happens:

- 1 The LED on the transmitter lights up in green to show that it is signalling the receiver.
- 2 The Visit LED on the receiver lights up in green and it starts to sound, flash or vibrate with a certain pace, called signal pattern. The transmitter and the connected accessories determine the signal pattern. The default is as follows:

Transmitter

LED
Green, top

Visit receiver

LED	Sound	Vibration	Flash
Green light	1x door chime, low	Slow	Yes

Changing the signal pattern

Changing the signal pattern is easy. Just open the transmitter front cover and move signal switches no. 8, 9 and 0 according to the table below:

Transmitter Receiver signal pattern

Switch	LED	Sound	Vibration	Flash
	Green light	1 x door chime, low	Slow ■□□□	Yes
	2 x green blinks	2 x door chime low	Slow ■□□□	Yes
	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Green blinks	2 x door chime, high	Slow ■□□□	Yes

Transmitter Receiver signal pattern

Switch	LED	Sound	Vibration	Flash
	Orange light	Baby melody	Fast ■■■■■■	Yes
	Orange blinks	Baby melody	Fast ■■■■■■	Yes
	Yellow light	1 x ring signal, low	Medium ■□■□	Yes
	Yellow blinks	2 x ring signal, high	Medium ■□■□	Yes

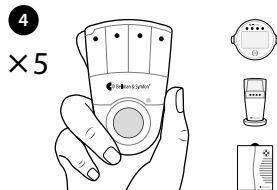
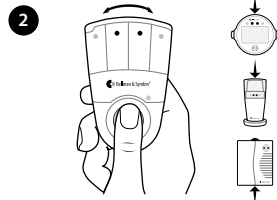
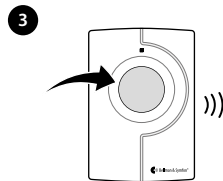
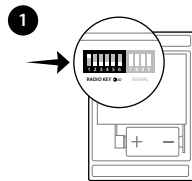
Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located under the transmitter cover.

Here is how you change the radio key:

- 1 Carefully remove the front cover on the push button transmitter. Move any radio key switch to the up (on position) to change the radio key.
- 2 Press and hold the test/function button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press the button on the transmitter within 30 s to send the new radio key.
- 4 All Visit LEDs on the receiver blink 5 times to show that the radio key has been changed. It then returns to normal mode.

Please note that all Visit units must be set to the same radio key in order to operate as a group.

Troubleshooting

If	Try this
The transmitter LED lights up in yellow when I press the button	<ul style="list-style-type: none">▪ The battery is nearly depleted. Replace it with an alkaline PX28A or a lithium PX28L type battery.
The transmitter LED doesn't light up when I press the button	<ul style="list-style-type: none">▪ Check that the battery is positioned correctly.▪ Replace the battery with an alkaline PX28A or a lithium PX28L type battery.
The transmitter LED lights up in green but the receiver is not activated	<ul style="list-style-type: none">▪ Check the receiver batteries and connections.▪ Move the receiver closer to the transmitter to make sure it's within radio range.▪ Check that the units are set to the same radio key, see Changing the radio key.

If	Try this
The receiver is activated for no apparent reason	<ul style="list-style-type: none">▪ There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see Changing the radio key.

Manufacturer

Bellman & Symfon Group AB
Södra Långebergsgatan 30
436 32 Askim Sweden
Phone +46 31 68 28 20
E-mail info@bellman.com
bellman.com

Revision: BE1420_082MAN2.0

Date of issue: 2022-06-30

TM and © 2022 Bellman & Symfon AB.

All rights reserved.