

Mino

Personal amplifier



Discover the joy of better hearing

Mino is a personal amplifier that will help you enjoy the company of your near and dear ones. It brings out the words so you can hear them loud and clear, even when there is a lot going on in the background. This means you don't have to make so much of an effort to hear.

Use Mino for example when the family gathers around the dinner table, in the store, or to listen to the radio and TV. Mino delivers clear and distinct speech and works both with and without hearing aids.





**” Now I am not
so tired after the
family dinner ”**



At home

Even in quiet surroundings, it can sometimes be difficult to distinguish the details of a conversation.

Just place Mino on the table in front of the person you are talking to and it will take care of the rest. You can choose to listen with headphones, earphones or Stetoclips. Mino is also an excellent complement to hearing aids when there is a lot going on in the background. It will filter out the background noise and clarify speech further before it reaches the hearing aid.

▼ **Headphones**

▼ **Earphones**

▼ **Stetoclips**



” Now we no longer talk past each other ”

At social events

Family parties are often noisy events with high sound levels and a lot of people talking at the same time. On such occasions it can be particularly challenging to follow a conversation. After tiring of having to ask people to repeat themselves, it is easy to end up silent.

With Mino you can hear the words even in busy environments. Just point it to the person talking and switch to directional microphone, and it will make the words stand out from the background noise.

▼ Neck loop

A neck loop is used to connect Mino to a hearing aid or a cochlear implant.





By the TV

If you have difficulty hearing, it is easy to turn the volume on the TV and radio to the max. This is not always appreciated by the other family members.

Now there's no reason to turn up the volume to hear better. Connect Mino to the TV or radio with the accompanying cable and you can enjoy great sound in stereo while sparing your family's ears. If you want to comment on the show, just press the Microphone button. Then you can join the conversation while still listening to the TV.

▼ **Cable kit, 16 ft (5 m)**



▼ **External microphone**





In theaters

A visit to a movie or concert is a pleasant break from everyday life. But it is easy to miss out on the experience if you have to make an effort to hear.

Many public venues, such as theaters, concert halls, and churches, are fitted with loop systems. In these venues, Mino captures the sound from the orchestra, movie or speaker with the built-in Telecoil. With a simple press of a button, you can switch to the internal microphone when you want to talk to the person next to you.

▼ T-symbol

Look for a sign with a T-symbol next time you visit your favorite movie theater or concert hall.





At the pharmacy


For people who are hard of hearing, something as simple as a visit to the pharmacy can become a real challenge. When the pharmacist's voice blends in with the background it is easy to miss out on important information. On the other hand, you don't want your private conversation to be overheard by others.

Place Mino on the counter to hear the advisor loud and clear, without her having to raise the voice. You can also bring it to your next doctor's appointment.

▼ **MicroSet™**

MicroSet™ is an accessory specially developed for Mino. Attach the lapel microphone to your shirt and put Mino in the pocket. Now your hands are free for other things.





**”Now I don’t need
to worry about
getting it wrong”**

In the car

A car is one of the toughest environments to conduct a conversation in. Besides the noise from the traffic, the passengers are normally facing away from you. This can make it very hard to hear what’s being said.

Just hang Mino around the neck of your travel companion and it will make the words stand out from the traffic noise. If you are sitting further away from each other you can connect a lapel microphone that you attach to your travel companion’s shirt with a clip.

▼ Travelcase


Do you travel occasionally?
Order the travel bag in durable canvas, specially designed to protect Mino and the accessories.



Pocket size and easy to use **Up and running within minutes**

Mino is small and light like a cell phone and fits discreetly in your pocket. It is intuitively easy to use and everything is accessible with the single press of a button.

Mino is also equipped with a rechargeable battery that lasts for a full 18 hours. That's enough for a whole day out, a movie marathon, or a day trip with your car.

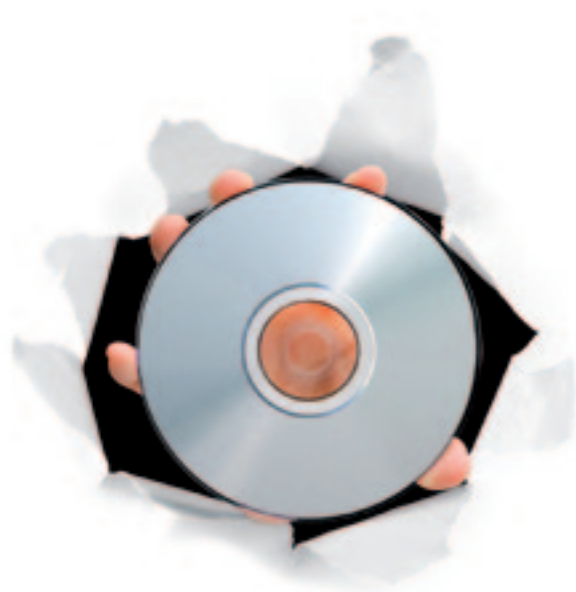

Directional microphone **Just point and listen**

Mino is equipped with both an omnidirectional- and a directional microphone. With the directional microphone, you can 'point' to the person talking and reduce unwanted noise around you. The directional microphone is especially useful in busy environments, such as restaurants, train terminals, and cafés, where it would otherwise be difficult to distinguish speech.



Clear and distinct sound **Listen without effort**

Mino is equipped with the latest digital technology and delivers clear and distinct speech of the same quality as modern CD and DVD players. It efficiently amplifies speech while filtering out unwanted background noise. This means, you can take part in the conversation without having to put so much effort into trying to hear things.



Using Mino with hearing aids

Mino clarifies speech and filters out disturbing background noise before it reaches the hearing aid or cochlear implant. This means you don't have to make so much of an effort to get the words right. Here is how it works.



1. On the table

Mino captures the sound, filters out the background noise and clarifies the speech.

2. Around the neck

The neck loop transmits the speech wirelessly to the hearing aid or cochlear implant.

3. In the ear

The hearing aid or cochlear implant refines the sound further before it reaches the ear.

Requires a hearing aid with Telecoil



Features

- Directional microphone ✓
- Omnidirectional microphone ✓
- Digital signal processing ✓
- Digital noise reduction ✓
- Tone control ✓
- Volume control ✓
- Balance control ✓
- Built-in Telecoil ✓
- Rechargeable battery ✓
- Input for external sound source ✓
- Advanced mix function ✓
 - Mixes the sound from the built in microphones with the signal from the:
 - Neck loop (MT mode)
 - Line input
 - External microphone
- Advanced settings menu ✓

In the box

- Mino personal amplifier
- Neck strap
- Belt clip
- User manual
- Universal charger

Content may vary with territory



Dedicated to your hearing

Bellman & Symfon of Sweden is dedicated to improve the quality of life for people with hearing problems. Our people and partners are devoted to this mission and we work closely with hearing healthcare professionals and leading experts to protect your hearing and make better hearing possible for everyone.

European office

Bellman & Symfon Europe AB
Södra Långebergsgatan 30
421 32 Västra Frölunda
Sweden
Phone +46 31 68 28 20
Fax +46 31 68 28 90
E-mail info@bellman.se

Asia Pacific office

Bellman & Symfon Asia Ltd.
1115-16 , Main Tower, GD Int'l Hotel
339 Huanshi Dong Rd
Guangzhou, China, 510095
Phone +86 20 8318 2950
Fax +86 20 8333 5528
E-mail info.cn@bellman.com