

# Visit Flash Receiver

BE1441-868/BE1442-868



## Features

**Bright flashes**  
30 Candela Xenon lights

**Clear signal identification**  
Through coloured LED lights

**Rotating top**  
Easy to direct the flashlight

**Multiple connections**  
Outlet for phone and bed shaker

**Battery backup (BE1442-868)**  
Works during power loss

**MD**

CE-marked medical  
device under EU  
MDR 2017/745

## Completely silent alerting.

The Flash Receiver from Bellman & Symfon is part of the Visit system that helps you to keep tabs on your home. It alerts you with bright flashes when the doorbell or telephone rings, when your baby needs you or if a fire should occur.

### Powerful and portable


The Flash Receiver is very easy to set up and use. Just connect it to the mains and place it on a table or shelf. You can also mount it on the wall using a wall bracket accessory.

The Flash Receiver produces bright flashes that are noticeable even in large areas and uses colored lights to differentiate the Visit signals. In addition, the rotating top makes it easy to direct the light.

The Flash Receiver features separate outlets for landline- and mobile telephones. Additionally, you can connect a Bed Shaker and wake up to vibrations if something should happen while you are asleep. The BE1442-868 model is also equipped with a battery backup.



# Technical specifications

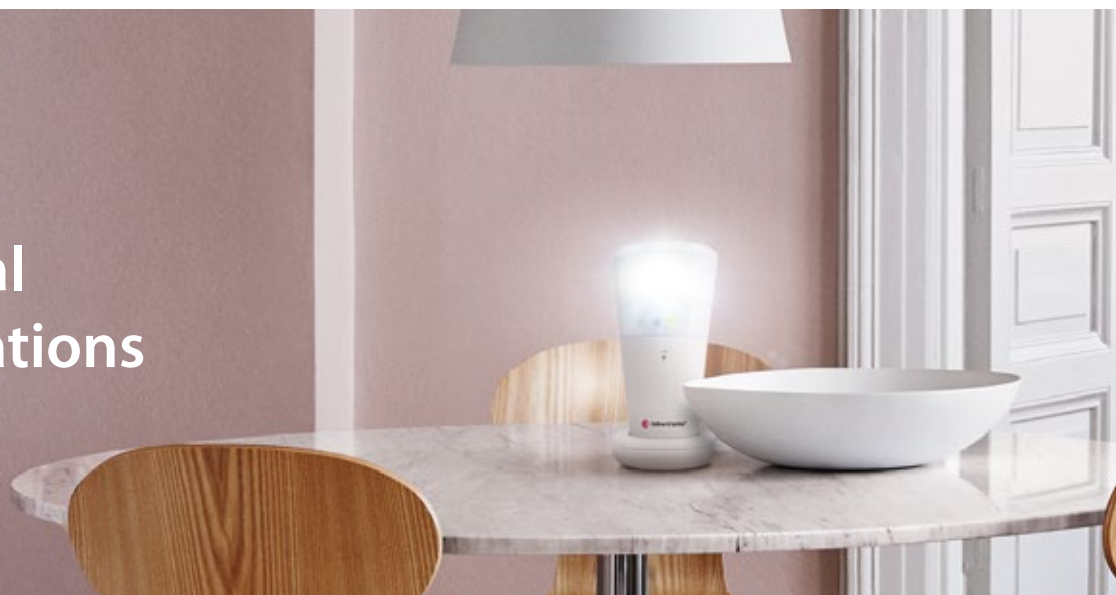


## Dimensions and weight

|          |                                                            |
|----------|------------------------------------------------------------|
| Height   | BE1441: <b>140 mm, 5.5"</b><br>BE1442: <b>155 mm, 6.1"</b> |
| Diameter | BE1441: <b>70 mm, 2.7"</b><br>BE1442: <b>78 mm, 3.1"</b>   |
| Weight   | <b>308 g, 10.8 oz.</b>                                     |



## Buttons and controls

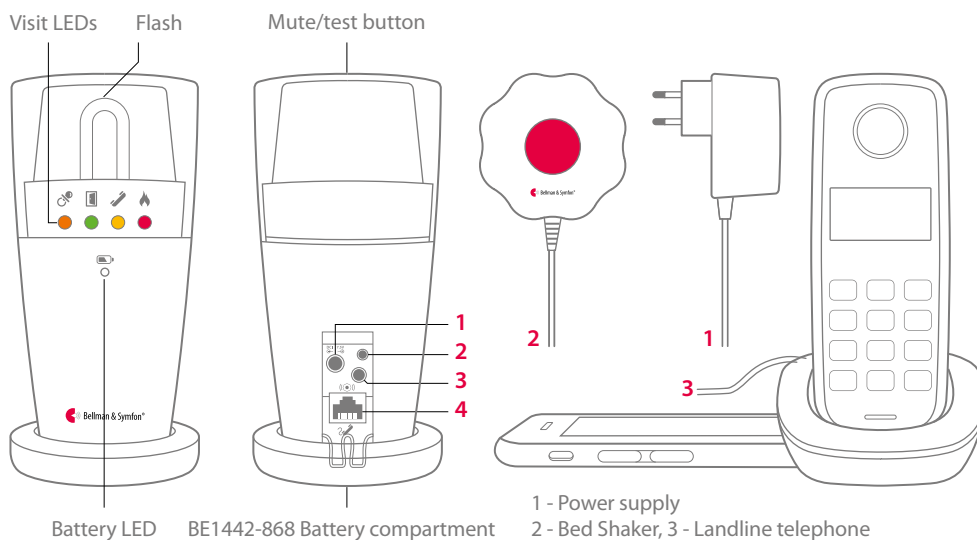

## Features

- Models  
B1441-868 and BE1442-868 that includes battery backup
- Alerting signal  
Built-in flash light ~30 Candela Xenon light  
Warning! Flashes can cause epileptic attacks.
- Activation
  - Via the Visit transmitters
  - Via the analogue telephone network
  - Via mobile phones/tablets (BE9250 required)

## Frequency and coverage

- Radio frequency: 868.3 MHz
- Coverage: 50 – 250 m, 55 – 273 yards depending on the characteristics of the building

## Approvals



This product is a medical device in conformity with Regulation (EU) 2017/745 (Medical Device Regulation).

## Power and battery

- Mains power  
7.5 V DC / 1500 mA with power supply unit
- Power consumption  
Active: 750 mA, idle position: 8 mA
- Backup batteries (BE1442 only)  
4×1.2 V AAA (HR03) NiMH rechargeable batteries  
Operation time: ~48 h when fully charged

## In the box

- BE1441-868 Visit Flash Receiver or  
BE1442-868 Visit Flash Receiver with  
battery backup
- 7.5 V DC / 1500 mA power adapter
- 4×1.2 V AAA NiMH batteries  
(BE1442-868 model only)

## Environmental requirements

- Operating temperature: 0 to 35° C, 32 to 95° F
- Relative humidity: 15% to 90%  
(non condensing)

## Visit indicators

The Visit LEDs normally indicates the following:

- Orange LED, pacifier symbol  
The baby monitor is activated
- Green LED, door symbol  
The door transmitter is activated
- Yellow LED, telephone symbol  
The telephone transmitter is activated
- Red LED, fire symbol  
The smoke alarm is activated
- Orange and Red LEDs blink alternately  
The CO alarm is activated

## Accessories

- BE9075 Wall mount bracket
- BE1270-P02 Bed Shaker
- BE9105 Telephone Cable

© 2025 Bellman & Symfon AB. All rights reserved.

Bellman, Bellman & Symfon and the Bellman & Symfon logo are registered trademarks of Bellman & Symfon AB. This leaflet is subject to change without prior notice.

BE1441-868-1442-868\_186DSH1.0-EN

[bellman.com](https://bellman.com)