
- EN** Medical device information
- DE** Informationen zu Medizinprodukten
- FI** Lääketieteellisiä laitteita koskevat tiedot
- NL** Informatie over medische hulpmiddelen
- NO** Informasjon om medisinsk utstyr
- SE** Information om medicinteknisk utrustning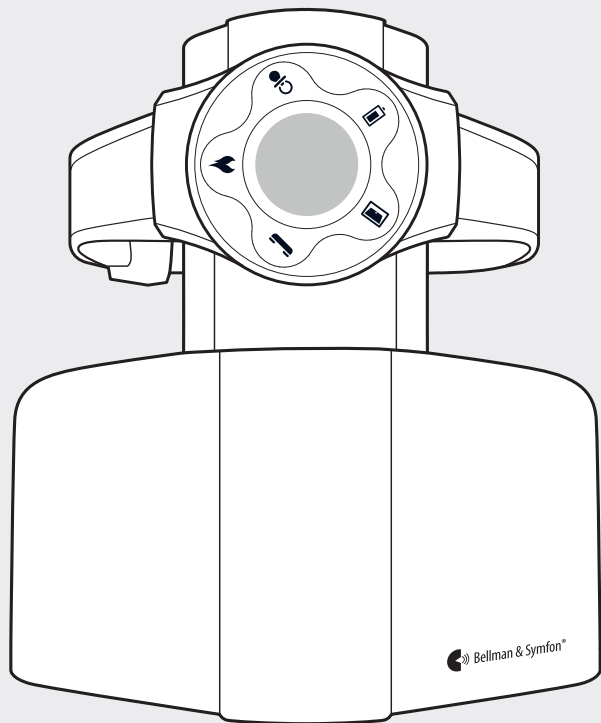

Visit wrist receiver and charger

-
- 03 Medical device information
-
- 09 Informationen zu Medizinprodukten
-
- 15 Lääketieteellisiä laitteita koskevat tiedot
-
- 21 Informatie over medische hulpmiddelen
-
- 27 Informasjon om medisinsk utstyr
-
- 33 Information om medicinteknisk utrustning
-

EN

DE

FI

NL

NO

SE

Read this first

Thank you for choosing a product from Bellman & Symfon – the world leader in alerting systems based in Gothenburg, Sweden. This leaflet contains important medical device information. Please read it carefully to make sure that you understand and get the best out of your Bellman & Symfon product. For more information about features and benefits, contact your hearing care professional.

About the BE8102 Visit wrist receiver and charger

Intended purpose

This wrist receiver is part of the Visit alerting system. The intended purpose of the system is to alert deaf and hard of hearing people of important signals in their home. Depending on the system setup, the wrist receiver alerts the user with vibrations and colored lights when for instance the doorbell or telephone rings, the baby needs attention, or the smoke alarm goes off. The intended purpose of the accompanying charger is to recharge the wrist receiver and to power an optional BE1270 Bed shaker accessory, used for night-time alerts.

Intended user group

The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.

Intended user

The intended user is a person with mild to severe hearing loss or deafness.

Principle of operation

The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

 This device will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.

Important safety information

This section contains important information about safety, handling and operating conditions. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

Hazard warnings

 Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.

 Be aware that alarms and notifications can be missed if batteries run out.

 Keep this device out of reach of children under 3 years.

 Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

 Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.

 Use only power adapters and battery types that are specified in this leaflet.

 Protect cables from any potential source of damage.

 This device is designed for indoor use only. Do not expose the device to moisture.

 Protect the device from shocks during storage and transport.

 Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.

 The wrist receiver and charger can only be used with rechargeable NIMH batteries! Non-rechargeable batteries will start to leak if recharged! The acid in the batteries will damage the electronics in the wrist receiver and the charger. The resulting damage will not be covered by any guarantee. Only use Varta V40H batteries for the wrist receiver and Lexel 600AAA batteries for the charger.

Information on product safety

 Failure to follow these instructions could result in damage to the device and void the warranty.

 Do not expose batteries to fire or to direct sunlight.

 Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35°C, 32°– 95° F.

 If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.

 Do not drop your device. Dropping onto a hard surface can damage it.

 The device may only be repaired by an authorized service center.

 If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit bellman.com for contact info.

 Do not use the device in areas where electronic equipment is prohibited.

 Without permission granted by the NCC, no company, enterprise or user is allowed

to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.

 The low power radio frequency devices shall not influence aircraft security and interfere legal communications; if found, the user shall cease operating immediately until no interference is achieved. The said legal communications means radio communications is operated in compliance with the Telecommunications Act. The low power radio frequency devices must be susceptible with the interference from legal communications or ISM radio wave radiated devices.

 The wrist receiver will not vibrate when it's in the charger. A bed shaker must be connected to the charger to provide a vibratory alert. Other indicators will act as normal. Apart from a bed shaker, it is also possible to connect other products such as some Bellman & Symfon alarm clock models to the vibrator output.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

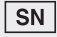
Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory.

Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Regulatory symbols

	With this symbol, Bellman & Symfon confirms that the product meets the Medical Device Regulation EU 2017/745.
	This symbol indicates the manufacturer's serial number so that a specific medical device can be identified. It's available on the product and gift box.
	This symbol indicates the manufacturer's catalogue number so that the medical device can be identified. It's available on the product and gift box.
	This symbol indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.
	This symbol indicates that the user should consult the instruction guide and this leaflet.
	This symbol indicates that it is important for the user to pay attention to the relevant warning notices in the user guides.
	This symbol indicates important information for handling and product safety.
	Temperature during transport and storage: -10° to 50° C, 14° to 122° F Temperature during operation: 0° to 35° C, 32° to 95° F
	Humidity during transportation and storage: <90%, non-condensing Humidity during operation: 15% to 90%, non-condensing
	Atmospheric pressure during operation, transportation and storage: 700 hpa to 1060 hpa
Operating conditions	This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in the user guide or this leaflet.
	With this CE symbol, Bellman & Symfon confirms that the product meets EU standards for health, safety, and environmental protection as well as the Radio Equipment Directive 2014/53/EU.
	This symbol indicates that the product shall not be treated as household waste. Please hand over your old or unused product to the applicable collection point for the recycling of electrical and electronic equipment or bring your old product to your hearing care professional for appropriate disposal. By ensuring this product is disposed of correctly, you will help prevent potential negative effects on the environment and human health.

Technical specifications

Dimensions and weight	Receiver: 38 × 12 × 49 mm / 1.5 × 0.5 × 1.9", 30 g / 1.1 oz. (incl. batteries) Charger: 95 × 100 × 117 mm / 3.7 × 3.9 × 4.6", 300 g / 10.6 oz. (including batteries and power adapter)
Mains power	7.5 V DC/1500 mA with power supply unit
Battery power	Receiver: 1 x 1.2 V V40H rechargeable battery Charger: 4 x 1.2 V NiMH rechargeable batteries
Power consumption	Receiver: Active: 100 mA, Idle position: 3 mA Charger: Active 650 mA, Idle position 70 mA
Operation and charging time	Receiver: ~30 h from fully charged, Charging time: ~10 h from fully depleted Charger: Backup battery charging time: ~24 h
Connections	Equipped with a bed shaker jack
Radio frequency	314.91MHz, 433.92MHz or 868.30MHz depending on region
Coverage	50 – 250 m, 55 – 273 yd. depending on the radio frequency and the characteristics of the building. Coverage is reduced if walls and large objects block the signal.
In the box	<ul style="list-style-type: none">▪ BE1560 Visit wrist receiver▪ BE1570 Charger▪ Power adapter▪ Extra wrist band

Configuration options

This BE1560 wrist receiver can be configured with the following Visit transmitters and original accessories:

Compatible Visit transmitters	Available accessories
<ul style="list-style-type: none">▪ BE1411 Visit door transmitter▪ BE1420 Visit pushbutton transmitter▪ BE1431 Visit telephone transmitter▪ BE1432 Visit mobile phone transmitter▪ BE1481 Visit smoke alarm transmitter▪ BE1551 Visit smoke alarm transmitter▪ BE1555 Visit CO alarm transmitter▪ BE1491 Visit baby monitor	<ul style="list-style-type: none">▪ BE1270 Bed shaker

 For detailed product information, see the corresponding user manual.

Compliance information

Europe

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below. The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit bellman.com for contact information.

Radio Equipment Directive (RED)

Medical Device Regulation (MDR)

EC General Product Safety Directive

Electromagnetic Compatibility Directive (EMC)

Restriction of Hazardous Substances Directive (RoHS)

REACH Regulation

Waste Electrical & Electronic Equipment (WEEE)

EC Battery Directive

ISO Certification of legal manufacturer

Bellman is certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.

SS-EN ISO 9001 Certification Number: CN19/42071

SS-EN ISO 13485 Certification Number: CN19/42070

Certification Body

SGS United Kingdom Ltd

Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

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TM and © 2021 Bellman & Symfon AB. All rights reserved.
SGS United Kingdom Ltd, Rossmore Business Park Ellesmere Port Cheshire CH65 3EN Storbritannien
Bellman & Symfon Group AB