

Bellman & Symfon®

BE1420

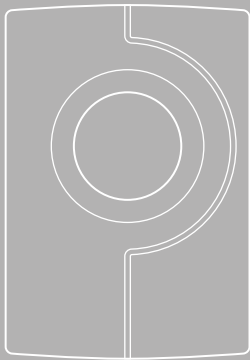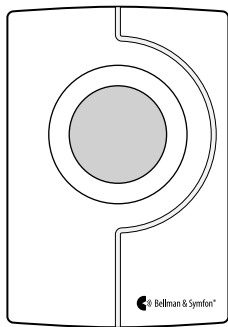Visit push button transmitter

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28 User manual

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52 Käyttöopas

64 Mode d'emploi

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88 Brukermanual

100 Bruksanvisning

112 Appendix

DE

DK

EN

ES

FI

FR

NL

NO

DK

App

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Read this first

This booklet guides you on how to use and maintain your new push button transmitter. Ensure you read this booklet carefully, including the **Warnings** section. This will help you get the most out of your product. If you have additional questions, contact your hearing care professional.

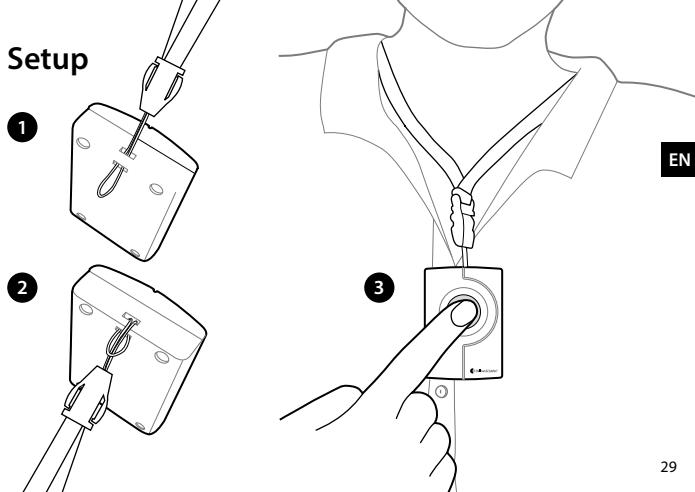
Intended purpose The intended purpose of the Visit system is to alert deaf and hard of hearing people of important signals in their home.

Intended user The intended user is a person with mild to severe hearing loss who needs sound amplification.

Intended user group The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.

Principle of operation The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

Setup

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years.
- Note that alarms and notifications can be missed if batteries run out.
- Only use battery types that are specified in this leaflet.
- This device is for indoor use and outdoor use in a protected location. It will not withstand water or rain.
- Protect the device from shocks during storage and transport.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.

- This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the user guide, may cause harmful interference to radio communication.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.

- The device may only be repaired by an authorized service center.
- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- Without permission granted by the NCC, no company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as

fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Configuration options

This transmitter can be configured with the following Visit receivers:

Compatible Visit receivers

- BE1450 Visit portable receiver
- BE1441 Visit flash receiver
- BE1442 Visit flash receiver
- BE1580 Visit alarm clock receiver
- BE1470 Visit pager receiver
- BE8102 Visit wrist receiver

For detailed product information, see the corresponding user manual.

Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below. The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit **bellman.com** for contact information.

Radio Equipment Directive (RED)

RoHS Directive

Medical Device Regulation (MDR)

REACH Regulation

Waste Electrical & Electronic Equipment (WEEE)

EC Battery Directive

Regulatory symbols

MD

With this symbol, Bellman & Symfon confirms that the product meets the Medical Device Regulation EU 2017/745.

SN

This symbol indicates the manufacturer's serial number so that a specific medical device can be identified. It's available on the product and gift box.

REF

This symbol indicates the manufacturer's catalogue number so that the medical device can be identified. It's available on the product and gift box.

This symbol indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.

This symbol indicates that the user should consult the instructions for use.

This symbol indicates that it is important for the user to pay attention to the relevant warning notices in the user guide.

This symbol indicates important information for handling and product safety.

Temperature during transport and storage: -10° to 50° C, 14° – 122° F
Temperature during operation: -10° to 40° C, 14° to 104° F

Humidity during transportation and storage: $<90\%$, non-condensing
Humidity during operation: 15% – 90% , non-condensing

Atmospheric pressure during operation, transportation and storage:
 700 hpa to 1060 hpa

Operating conditions

This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in the instructions for use.

With this CE symbol, Bellman & Symfon confirms that the product meets EU standards for health, safety, and environmental protection as well as the Radio Equipment Directive 2014/53/EU.

With this UKCA symbol, Bellman & Symfon confirms that the product meets Great Britain (England, Wales and Scotland) standards for health, safety, and environmental protection.

This symbol indicates that the product shall not be treated as household waste. Please hand over your old or unused product to the applicable collection point for the recycling of electrical and electronic equipment or bring your old product to your hearing care professional for appropriate disposal.

ISO Certification of legal manufacturer

Bellman is certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.
SS-EN ISO 9001 Cert. No.: CN19/42071, SS-EN ISO 13485 Cert. No.: CN19/42070

Certification Body

SGS United Kingdom Ltd, Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

Technical specifications

Dimensions	66 × 48 × 23 mm, 2.6 × 1.9 × 0.9"
Weight	50 g, 1.8 oz. including battery
Power and battery	Battery: 1 × 6V PX28G/PX28M alkaline or 6V PX28L lithium Power consumption: Active 35 mA, idle position 0.05 µA Operating time: Alkaline ~ 2 years, Lithium ~ 4 years
Activation	Via the front button
Radio frequency	868.30MHz
Coverage	50-250 m (55-273 yd) depending on the characteristics of the building.
Number of radio keys	64 as standard. Up to 256 with optional software.
In the box	<ul style="list-style-type: none">▪ BE1420 Visit pushbutton transmitter▪ PX28G/PX28M 6V alkaline battery▪ Lanyard with safety clasp▪ Adhesive tape, screws and plugs

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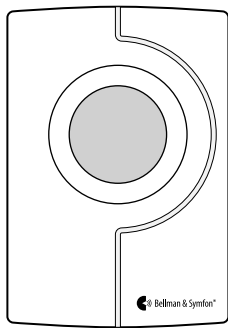
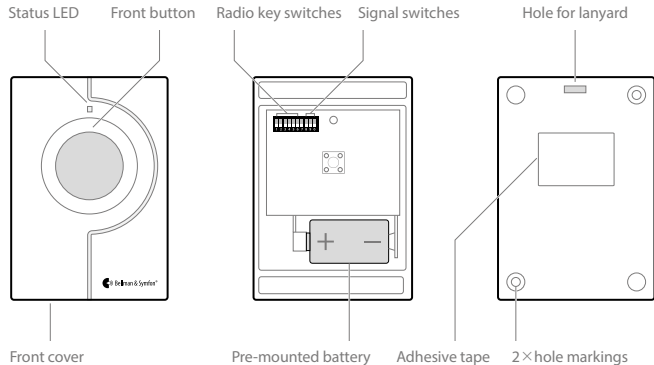
Overview




Using it as a caller button

The transmitter can be worn around your neck and be used as a wireless caller button.

Here is how you use it:

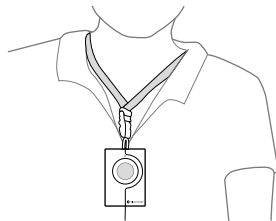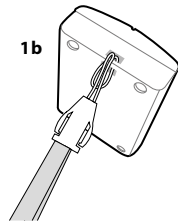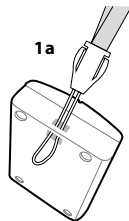- 1 Attach the lanyard to the transmitter.
- 2 Hang the transmitter around your neck.


Using it as a doorbell

The transmitter can also be used as a wireless doorbell. Here is how you set it up:

Mounting with screws

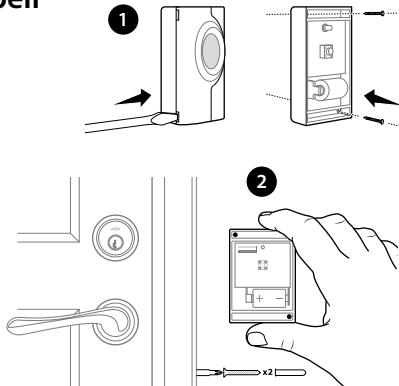- 1 Carefully remove the transmitter front cover and make two holes on the markings.
- 2 Fix the unit to the wall using the supplied screws and put the cover back on.


Using it as a doorbell

You can instead choose to use the supplied adhesive tape. Here is how it is done:

Mounting with adhesive tape

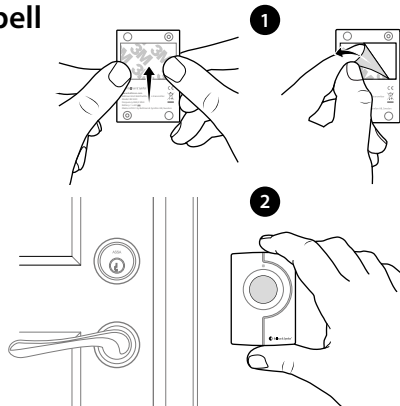- 1 Attach the adhesive tape to the back of the transmitter. Clean the wall with the wet wipe and remove the protective film from the tape.
- 2 Mount the unit in a weather protected area by the front door.


Testing the signal

Using the test buttons

- 1 Press the front button on the transmitter. The LED lights up in green to show that a radio signal is being transmitted.
- 2 The green Visit LED on the receiver lights up to show that the signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

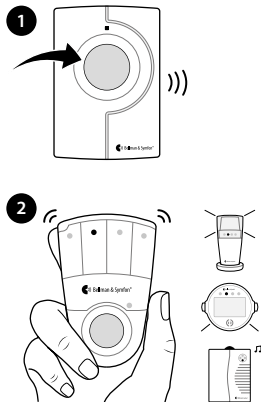Note: If you have changed the signal pattern, it will react in accordance with the tables on page 120 to 121.

Default signal pattern

When the push button transmitter is activated, the following happens:

- 1 The LED on the transmitter lights up in green to show that it is signalling the receiver.
- 2 The Visit LED on the receiver lights up in green and it starts to sound, flash or vibrate with a certain pace, called signal pattern. The transmitter and the connected accessories determine the signal pattern. The default is as follows:

Transmitter

LED
Green, top

Visit receiver

LED	Sound	Vibration	Flash
Green light	1x door chime, low	Slow	Yes

Changing the signal pattern

Changing the signal pattern is easy. Just open the transmitter front cover and move signal switches no. 8, 9 and 0 according to the table below:

Transmitter Receiver signal pattern

Switch	LED	Sound	Vibration	Flash
	Green light	1 x door chime, low	Slow ■□□□	Yes
	2 x green blinks	2 x door chime low	Slow ■□□□	Yes
	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Green blinks	2 x door chime, high	Slow ■□□□	Yes

Transmitter Receiver signal pattern

Switch	LED	Sound	Vibration	Flash
	Orange light	Baby melody	Fast ■■■■■■	Yes
	Orange blinks	Baby melody	Fast ■■■■■■	Yes
	Yellow light	1 x ring signal, low	Medium ■□■□	Yes
	Yellow blinks	2 x ring signal, high	Medium ■□■□	Yes

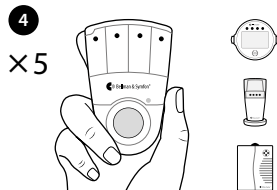
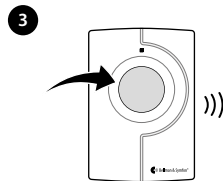
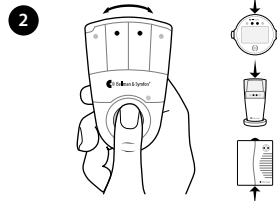
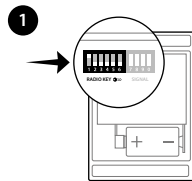
Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located under the transmitter cover.

Here is how you change the radio key:

- 1 Carefully remove the front cover on the push button transmitter. Move any radio key switch to the up (on position) to change the radio key.
- 2 Press and hold the test/function button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press the button on the transmitter within 30 s to send the new radio key.
- 4 All Visit LEDs on the receiver blink 5 times to show that the radio key has been changed. It then returns to normal mode.

Please note that all Visit units must be set to the same radio key in order to operate as a group.

Troubleshooting

If	Try this
The transmitter LED lights up in yellow when I press the button	<ul style="list-style-type: none">▪ The battery is nearly depleted. Replace it with an alkaline PX28A or a lithium PX28L type battery.
The transmitter LED doesn't light up when I press the button	<ul style="list-style-type: none">▪ Check that the battery is positioned correctly.▪ Replace the battery with an alkaline PX28A or a lithium PX28L type battery.
The transmitter LED lights up in green but the receiver is not activated	<ul style="list-style-type: none">▪ Check the receiver batteries and connections.▪ Move the receiver closer to the transmitter to make sure it's within radio range.▪ Check that the units are set to the same radio key, see Changing the radio key.

If	Try this
The receiver is activated for no apparent reason	<ul style="list-style-type: none">▪ There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see Changing the radio key.

Manufacturer

Bellman & Symfon Group AB
Södra Långebergsgatan 30
436 32 Askim Sweden
Phone +46 31 68 28 20
E-mail info@bellman.com
bellman.com

Revision: BE1420_082MAN2.0

Date of issue: 2022-06-30

TM and © 2022 Bellman & Symfon AB.

All rights reserved.