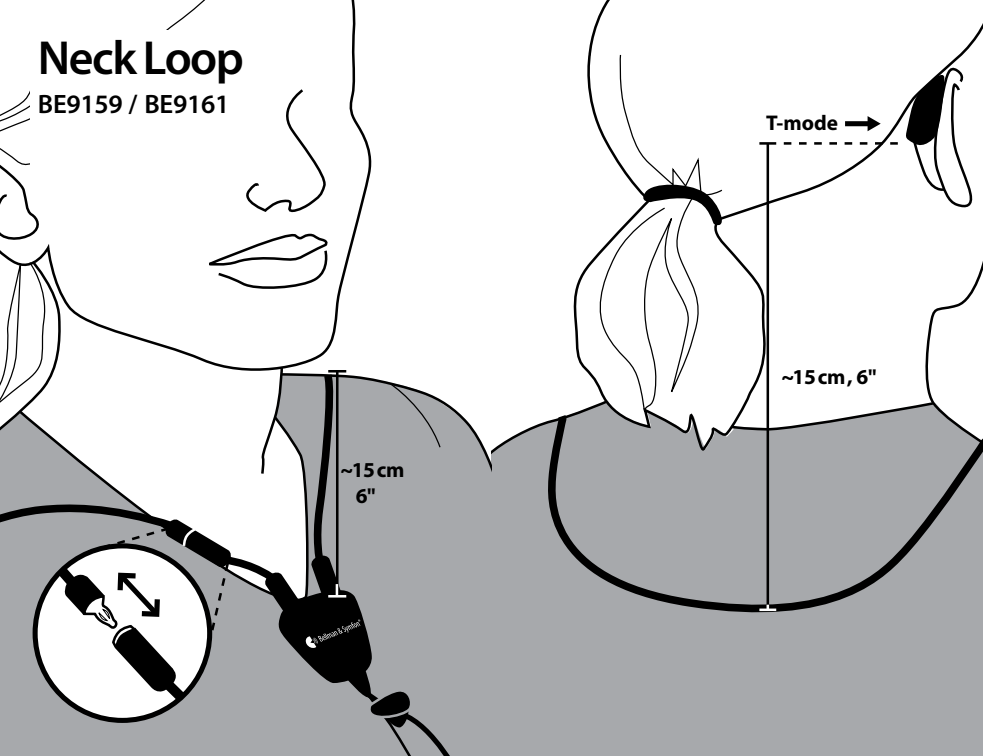
BE9159 Neck Loop,
long cable

BE9161 Neck Loop,
short cable




Neck Loop

BE9159 / BE9161

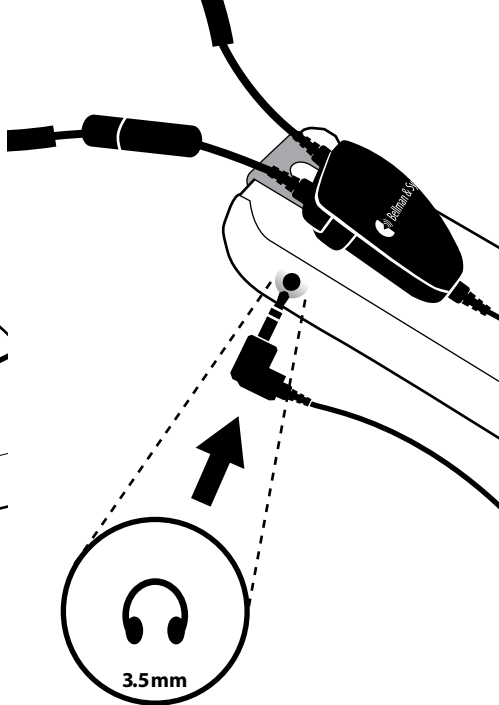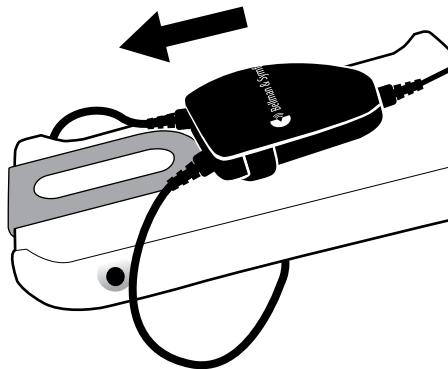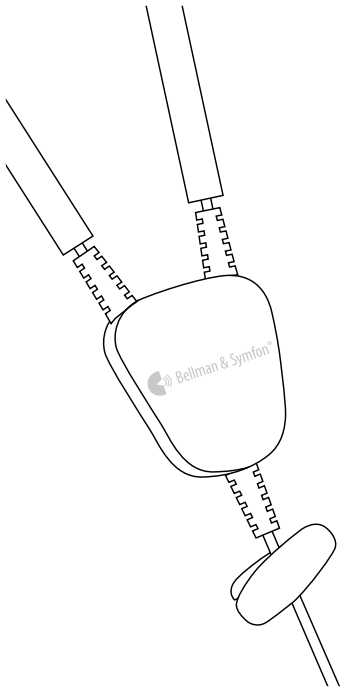- EN** Use with Domino or Mino
ES Utilízelo con Domino o Mino
FR Utiliser avec Domino ou Mino
JP Domino または Mino と併用してください。
CN_{SI} 与Domino或 Mino 搭配使用
CN_{TR} 與 Domino 或 Mino 搭配使用

BE9159 Neck Loop, long cable
BE9161 Neck Loop, short cable

08 User manual

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EN

ES

FR

CN_{SI}

CN_{TR}

JP

User information

- This Neck Loop from Bellman & Symfon is to be used with hearing aids equipped with Telecoil.
- It is optimized for the Maxi, Maxi Pro, Sonido, Mino personal amplifiers and the Domino listening system.
- The Neck Loop can also be used with smart devices, TVs, radios or other equipment that features a 3.5 mm stereo headphone jack intended for low impedance headphones.
- It is discreet and comfortable, and can easily be worn under your shirt. It also incorporates a breakaway neck strap for increased safety.
- **Important!** If you have a pacemaker, we recommend you check with your general practitioner or cardiologist before using a Neck Loop.

Read this first

This booklet guides you on how to use and maintain your new Neck Loop. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

About the BE9159 /BE9161 Neck Loop

Intended user group

People of all ages.

Intended user

A user that wants to control tuning and listening volume.

Principle of operation

The audio product family consists of several amplifiers and sound transmitters which have been specifically developed to provide sound enhancement even in demanding situations. Depending on the assigned function of the specific amplifier or sound transmitter, different microphones can be used to pick up direct sound or enhance the ambient sound.

This device will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.

General Warnings

This section contains important information about safety, handling and operating conditions. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Protect cables from any potential source of damage.
- If you have a pacemaker, we recommend you check with your general practitioner or cardiologist before using a Neck Loop.

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not use the device in areas where electronic equipment is prohibited.

- The device may only be repaired by an authorized service center.
- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- Do not drop your device. Dropping onto a hard surface can damage it.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product for 90 days from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes

incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

Indicates the manufacturer's serial number so that a specific device can be identified.

Indicates the device manufacturer.

This symbol indicates the need for the user to consult the instructions for use.

This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

To identify a place where information may be found, especially in an emergency.

Indicates the temperature limits to which the device can be safely exposed.

Indicates the range of humidity to which the device can be safely exposed.

Technical specifications

Neck loop diameter	22 cm, 9"
Weight	BE9159: 62g, 2.2 oz BE9161: 58g, 2 oz
Cable length	BE9159: 90cm, 3" BE9161: 15cm, 6"
Connectors	3.5mm tele plug (stereo) gold plated connector 90 degrees angle. (Break-away connector on the neck loop).
Load impedance	2 × 5 Ω
Magnetic output	1500mA/m @ 15cm, 6" distance and 2 × 50mW input signal
Operation	Temperature: 0 to 35 °C, 32 to 95 °F Humidity: 15% to 90%, non condensing
Configuration	This Neck Loop can be configured with the following personal amplifiers and listening systems:
Compatible personal amplifiers	■ BE2020 Maxi Personal Amplifier ■ BE2021 Maxi Pro Personal Amplifier ■ BE2030 Mino Personal Amplifier ■ BE2210 Domino Pro Receiver
Compatible listening systems	■ BE8005 Domino Pro Listening System
In the box	BE9159 / BE9161 Neck Loop

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All rights reserved.
Bellman & Symfon North America Inc.
Bellman & Symfon Asia Ltd.
Bellman & Symfon Group AB
Bellman & Symfon AB