BE2020 Maxi personal amplifier

- 4 Bedienungsanleitung
- 12 Brugervejledning
- 20 User manual
- 28 Manual de usuario
- 36 Käyttöopas
- 44 Mode d'emploi
- 52 Gebruiksaanwijzing
- 60 Brukermanual
- 68 Bruksanvisning
- 76 Appendix (EN)

DE

DK

EN

ES

FI

FR

NL

NO

SE

App

Les dette først

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SGS United Kingdom Ltd

Inward Way

Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

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Advanced settings

Maxi has a few advanced settings that can be used to adapt it to the user's individual requirements. Some of the advanced settings are aimed at limiting the user to accidentally operate the product in an undesired way. The following settings can be adjusted:

Balance

The balance between the right and left channels can be set, e.g. for headphones. This setting can be used to compensate for a hearing loss in one ear. The default setting is centered balance.

Basic attenuation

The basic attenuation reduces the gain and basically shifts the user volume setting downwards. This setting is useful when you want to adjust the maximum volume for a pair of headphones or in-ear phones with a higher sensitivity or when you want to limit the maximum volume a user can set. The default setting is 0 dB attenuation (full gain).


Blocking the T position

The Telecoil selection button can be blocked if you know that the T position will not be used. This setting is useful for those who will never use the telecoil and want to avoid selecting it by mistake. The default position is not blocked.

Adjusting advanced settings

To enter the advanced settings menu:

- 1 Open the battery cover on the back of Maxi by pressing the battery lock in the direction of the arrow.
- 2 Use e.g. a paper clip to press the advanced settings button for about 5 seconds, see the illustration. One of the LEDs in the volume/tone indicator (5) will start to blink.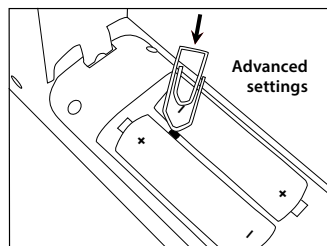

To adjust the L/R balance:

Press (← on the volume controls (6) to move the sound towards the left channel and +)) to move it towards the right channel. The current setting is shown in the volume/tone indicator (5). When you are done, press the advanced settings button to enter Basic attenuation.

To adjust the basic attenuation:

Press (← on the volume controls (6) to increase the attenuation and +)) to reduce it. The current attenuation relative to the maximum volume is shown in the volume/tone indicator (5). As you increase the attenuation, more and more LEDs will light up, each corresponding to a 3dB volume reduction. When you are done, press the advanced settings button to enter T position blocking.

To block the T position:

Press (← on the volume controls (6) to block the telecoil and +)) to turn it back on. When the telecoil is blocked, the telecoil indicator (10) is off and when the telecoil is activated, the telecoil indicator (10) blinks in green.

When you are done, you have two options:


- 1 To save your settings, press the advanced settings button for about 5 seconds.
- 2 To start all over, press the settings button briefly and Maxi will return to the L/R balance settings.

Locking the tone controls

You can lock the tone controls to prevent the tone from being accidentally changed by the user.

- 1 First, make sure that Maxi is switched off. Then, hold down the (A) -button on the tone controls (7) and start Maxi by pressing the On/Off button (3). Keep the (A) -button down until Maxi has stopped blinking.
- 2 The blinking LED in the volume/tone indicator (5) shows the current setting. Use the tone controls (7) to adjust it.
- 3 If all other volume/tone indicators are lit, the tone control is activated. If all other volume/tone indicators are off, the tone control is locked. Press +)) on the volume controls (6) to activate the tone controls and (← to lock them.
- 4 To save your settings, switch Maxi off using the On/Off button (3).

Maxi frequency response

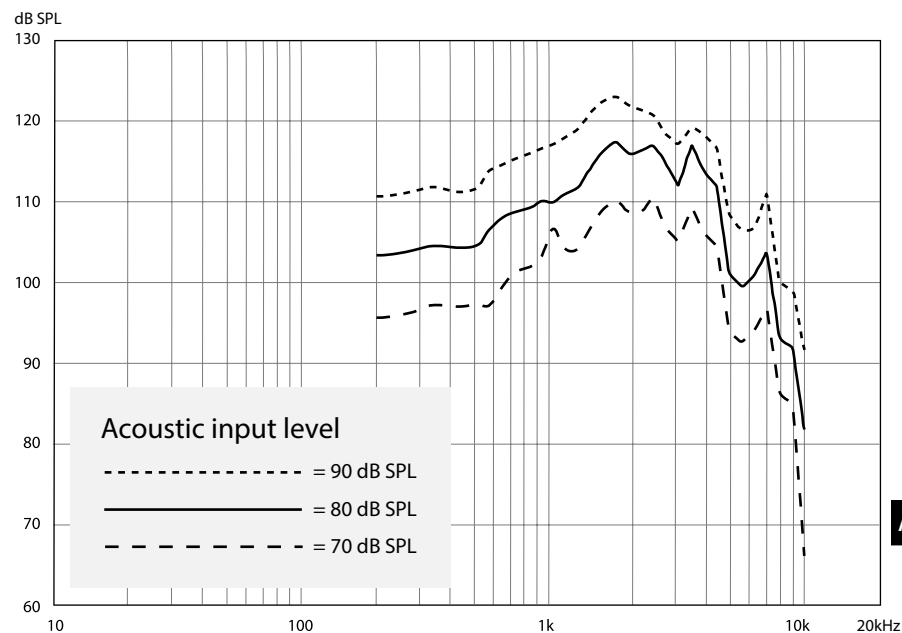DESIGN FOR EARS™

Manufacturer

Bellman & Symfon Group AB
Södra Långebergsgatan 30
436 32 Askim Sweden
Phone +46 31 68 28 20
E-mail info@bellman.com
bellman.com

Revision: BE2020_005MAN1.0

Date of issue: 2022-08-29

TM and © 2022 Bellman & Symfon AB.

All rights reserved.