
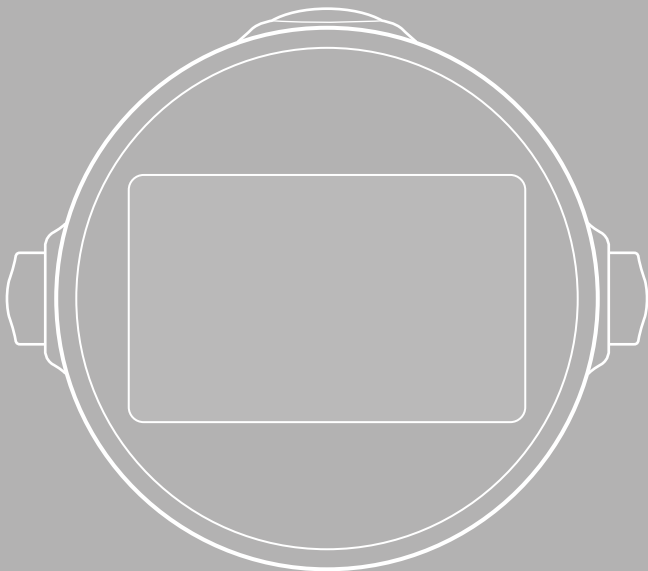
BE1580-868/BE1370-C
Alarm Clock Visit / Alarm Clock Pro

- (EN)** Model overview
- (DE)** Model Übersicht
- (DK)** Modeloversigt
- (ES)** Visión general del modelo
- (FI)** Mallikatsaus
- (FR)** Vue d'ensemble du modèle


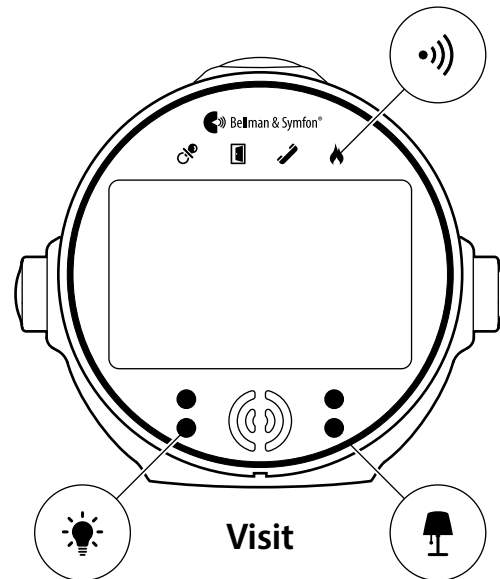
- (IT)** Panoramica del modello
- (NL)** Modeloverzicht
- (NO)** Modeloversikt
- (PL)** Przegląd modelu
- (SE)** Modellöversikt
- (SK)** Prehľad modelu


- (EN)** Remove the battery tab
- (DE)** Sicherungsstreifen herausziehen
- (DK)** Aftagning af batteridæksel
- (ES)** Retirar la lengüeta de las pilas
- (FI)** Poista pariston suojaliuska
- (FR)** Retirer la languette de la batterie

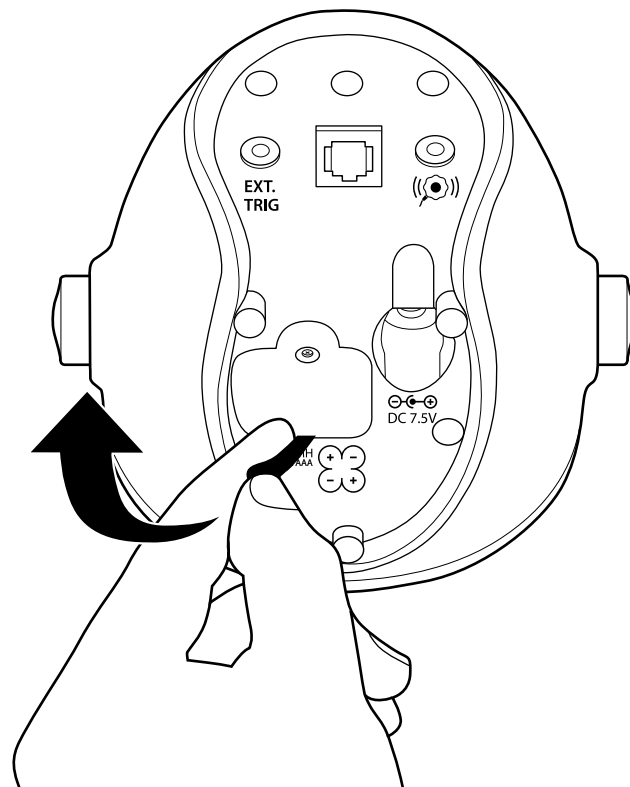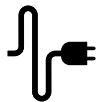- (IT)** Rimuova la scheda della batteria
- (NL)** Het batterijlipje verwijderen
- (NO)** Fjern batterimerket
- (PL)** Proszę wyjąć zakładkę baterii
- (SE)** Dra bort batterilappen
- (SK)** Odstráňte kartu batérie


- | | |
|------------------------------------|------------------------------|
| (EN) Connect the alarm clock | (IT) Colleghi la sveglia |
| (DE) Verbinden Sie den Wecker | (NL) De wekker aansluiten |
| (DK) Tilslutning af vækkeuret | (NO) Koble til vekkerklokken |
| (ES) Conectar el reloj despertador | (PL) Proszę podłączyć budzik |
| (FI) Kytke herätyskello | (SE) Anslut väckarklockan |
| (FR) Brancher le réveil | (SK) Pripojenie budíka |

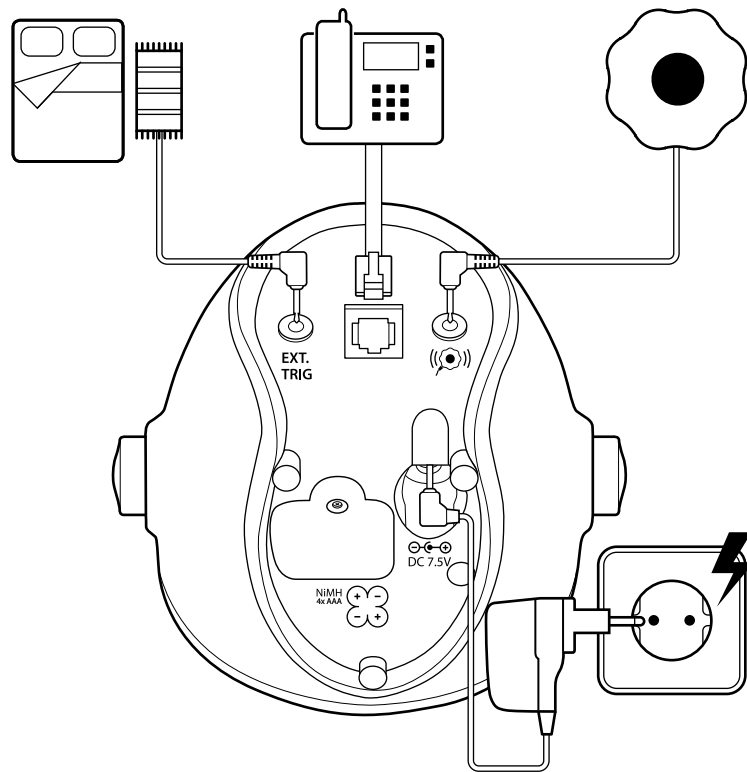
- | | |
|--------------------------------------|---------------------------------|
| (EN) Set the time | (IT) Imposta l'ora |
| (DE) Stellen Sie die Zeit ein | (NL) De tijd instellen |
| (DK) Indstilling af tiden | (NO) Innstill tiden |
| (ES) Ajustar la hora | (PL) Proszę ustawić czas |
| (FI) Aseta aika | (SE) Ställ in tid |
| (FR) Régler l'heure | (SK) Nastavenie času |

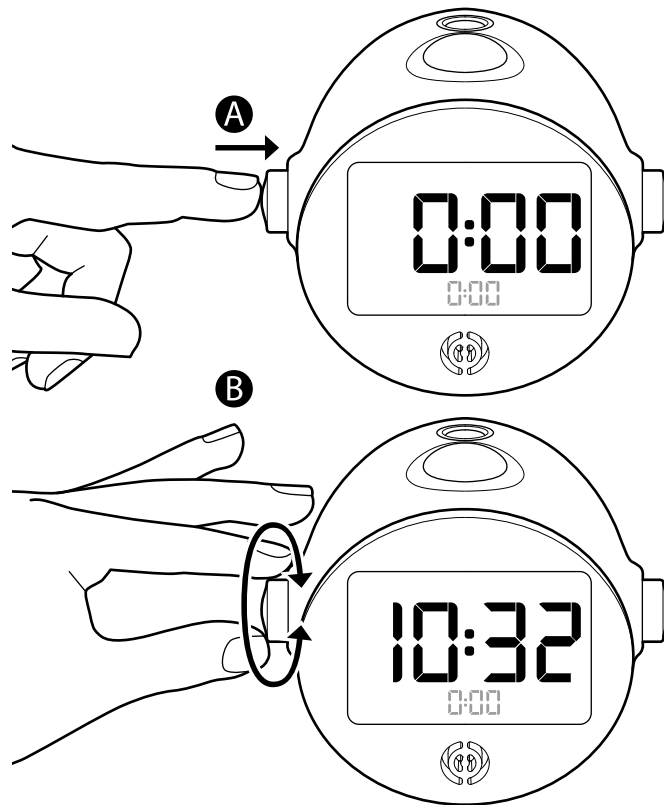
- | | |
|---------------------------------------|----------------------------------|
| (EN) Set the alarm | (IT) Imposta l'allarme |
| (DE) Stellen Sie den Alarm ein | (NL) Het alarm instellen |
| (DK) Indstilling af alarmer | (NO) Innstill alarmer |
| (ES) Ajustar la alarma | (PL) Proszę ustawić alarm |
| (FI) Aseta herätys | (SE) Ställ in alarm |
| (FR) Régler l'alarme | (SK) Set the alarm |

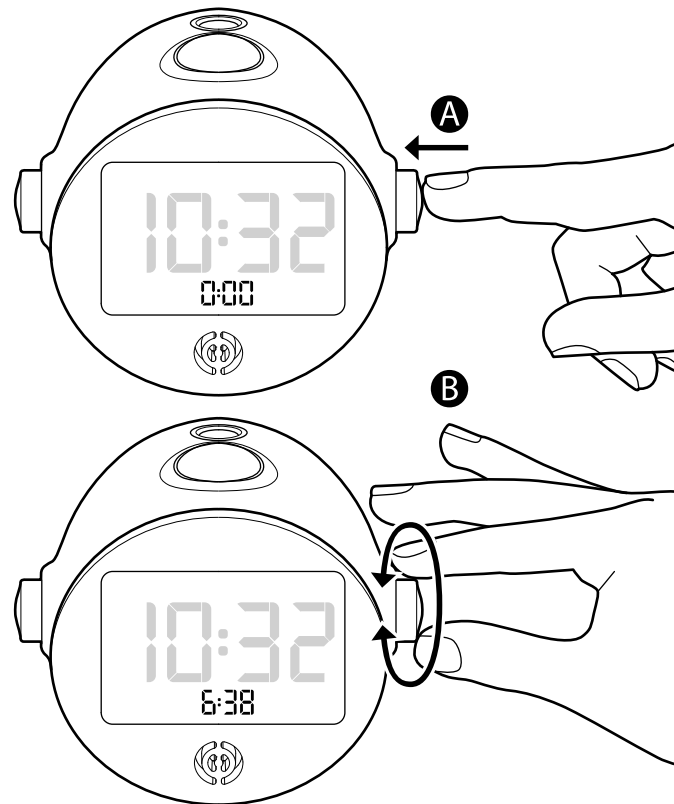
- | | |
|-------------------------------|--------------------------|
| (EN) Activate the alarm | (IT) Attivare l'allarme |
| (DE) Aktivieren Sie den Alarm | (NL) Het alarm activeren |
| (DK) Aktivering af alarmer | (NO) Aktiver alarmer |
| (ES) Activar la alarma | (PL) Aktywacja alarmu |
| (FI) Aktivoi herätys | (SE) Aktivera alarmet |
| (FR) Activer l'alarme | (SK) Aktivácia alarmu |

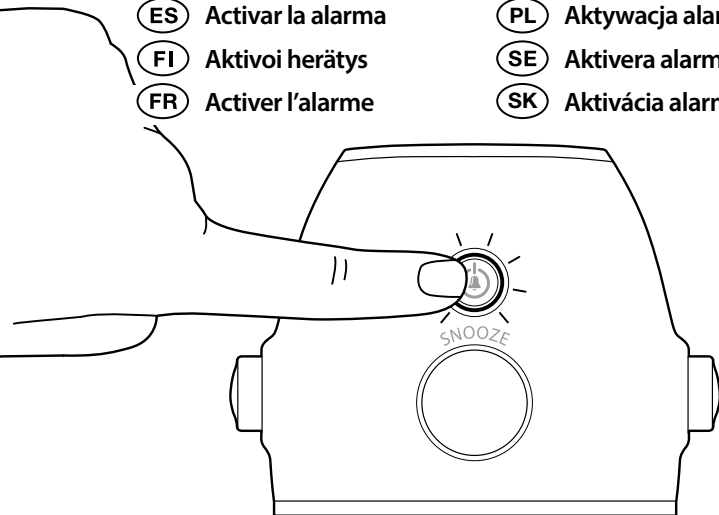SNOOZE

- | | |
|-----------------|-------------------|
| (EN) Snooze | (IT) Sonnacchiare |
| (DE) Schlummern | (NL) Snooze |
| (DK) Slumre | (NO) Slumre |
| (ES) Repetir | (PL) Drzemka |
| (FI) Torkku | (SE) Snooza |
| (FR) Répétition | (SK) Snooze |

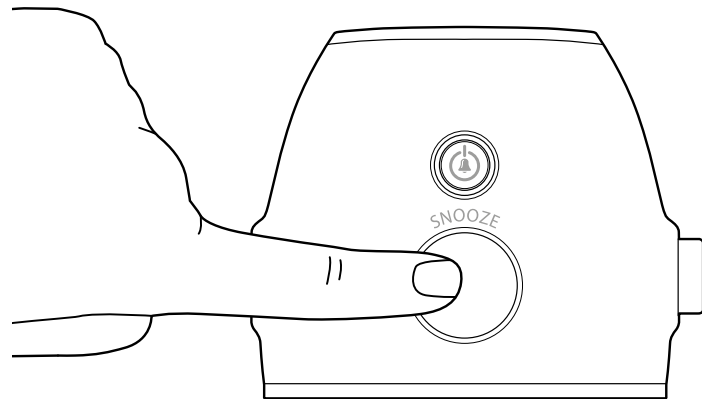(EN) Night light

(DE) Nachtlcht

(DK) Natlys

(ES) Luz nocturna

(FI) Yövalo

(FR) Veilleuse

(IT) Luce notturna

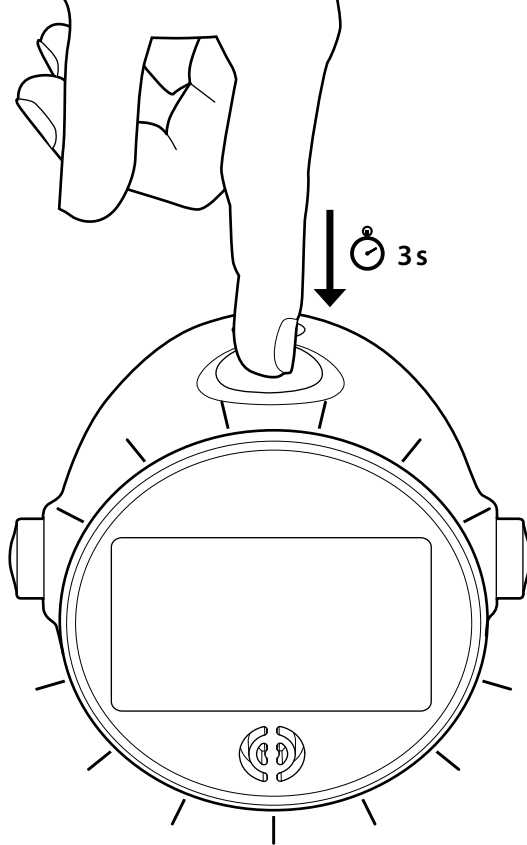
(NL) Nachtlchtje

(NO) Nattlys

(PL) Lampka nocna

(SE) Nattlampa

(SK) Nočné svetlo

EN Time IT Tempo
DE Zeit NL Tijd
DK Tid NO Tid
ES Hora PL Czas
FI Aika SE Tid
FR Heure SK Čas

EN Snooze IT Sonnacchiare
DE Schlummern NL Snooze
DK Slumre NO Slumre
ES Repetición PL Drzemka
FI Torkku SE Snooza
FR Répétition SK Snooze

EN Alarm IT Allarme
DE Alarm NL Alarm
DK Alarme NO Alarm
ES Alarma PL Alarm
FI Herätys SE Alarm
FR Alarme SK Alarm

EN Flash IT Flash
DE LED-Blitz NL Flits
DK Blink NO Lys
ES Destello PL Błysk
FI Vilkku SE Blixtljus
FR Clignotant SK Blesk

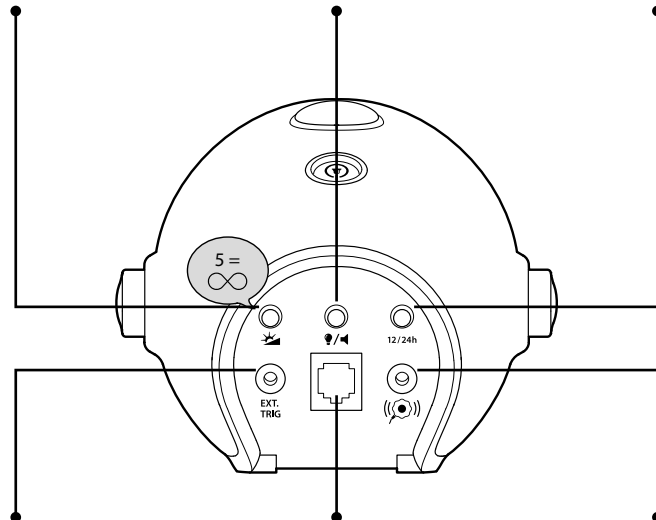
EN Speaker IT Altoparlante
DE Lautsprecher NL Luidspreke
DK Højtaler NO Høyttaler
ES Altavoz PL Głośnik
FI Kaiutin SE Högtalare
FR Haut-parleur SK Reprodukter

EN Night light IT Luce notturna
DE Nachtlcht NL Nachtlchtje
DK Natlys NO Natlyts
ES Luz nocturna PL Lampka nocna
FI Yövalo SE Nattlampa
FR Veilleuse SK Nočné svetlo

EN Brightness IT Luminosità
DE Helligkeit NL Helderheid
DK Lysstyrke NO Lysstyrke
ES Brillo PL Jasność
FI Kirkkaus SE Ljusstyrka
FR Luminosité SK Jas

EN Flash/Sound IT Flash/Suono
DE Blitz/Ton NL Flits/Geluid
DK Blink/Lyd NO Lys/Lyd
ES Destello/Sonido PL Błysk/Dźwięk
FI Vilkku/Ääni SE Blixtljus/Ljud
FR Clignotant/Son SK Flash/Zvuk

EN Time format IT Formato dell'ora
DE Zeitformat NL Indeling
DK Format NO Format
ES Formato PL Format czasu
FI Formaatti SE Format
FR Format SK Formát času

EN Ext. trig. IT Ext. trig.
DE Trigger NL Ext. trig.
DK Ekst. udl. NO Ekst. utl.
ES Activ. ext. PL Zew. wyzw.
FI Ulk. lauk. SE Ext. trig.
FR Déclench. ext. SK Rozšírenie trig.

EN Telephone IT Telefono
DE Telefon NL Telefoon
DK Telefon NO Telefon
ES Teléfono PL Telefon
FI Puhelin SE Telefon
FR Téléphone SK Telefón

EN Bed shaker IT Agitatore Per Letto
DE Vibrationskissen NL Trilalarm
DK Pudevibrator NO Sengevibrator
ES Vibrador Para Almohada PL Pad Wibracyjny
FI Sängyn Ravistin SE Sängvibrator
FR Coussin Vibrant SK Vibracný Vankúš

(EN) Pairing with a Visit transmitter

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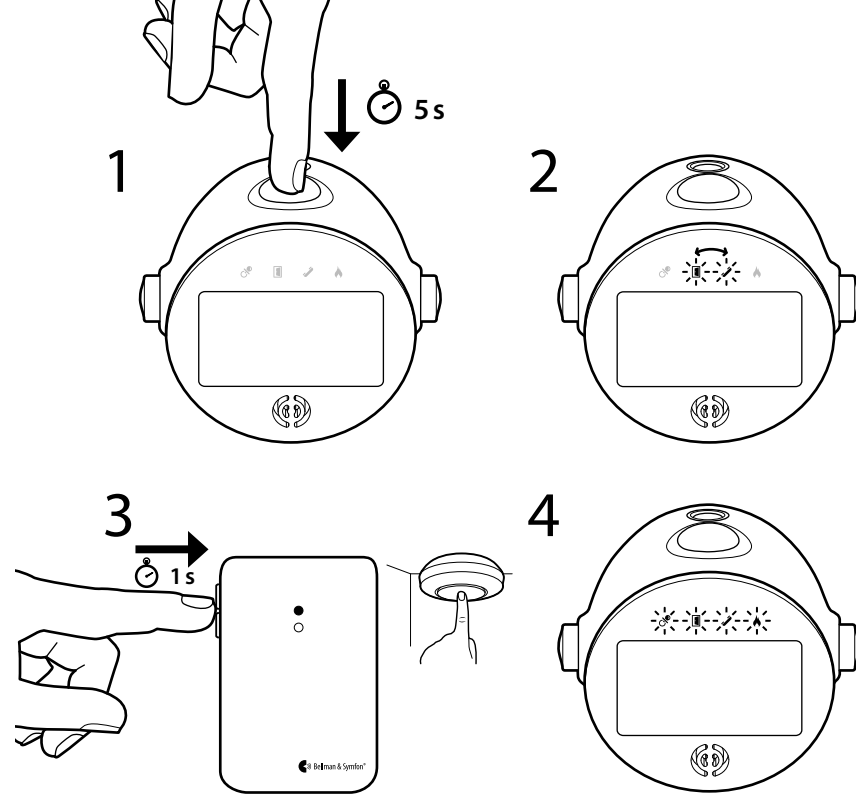
SK

Read this first

This booklet guides you on how to use and maintain your new Visit Alarm Clock. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

For BE1580-868 Visit Alarm Clock

- Intended purpose** The intended purpose of the Visit alerting system is to alert deaf and hard of hearing people of important signals in their home.
- Intended user** The intended user is a person with mild to severe hearing loss or deafness.
- Intended user group** The intended user group consists of people of all ages with mild to severe hearing loss or deafness who need audio, visual or sensory amplification.
- Principle of operation** The Visit system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

For BE1370-C Alarm Clock Pro

- Intended purpose** The intended purpose of the alarm clock is to awake and alert deaf and hard of hearing people of a set alarm.
- Intended user** The intended user is a person with mild to severe hearing loss or deafness.
- Intended user group** The intended user group consists of people of all ages with mild to severe hearing loss or deafness who need audio, visual or sensory amplification.
- Principle of operation** The alarm clock is typically placed on the bedside table and used during the night. When the alarm clock is activated by a set alarm, it awakes the user with sound, flashes and vibrations from the accompanying bed shaker. In addition, it can be configured with a BE9105 Telephone cable to alert for incoming calls and a BE9026-P02 Contact mat to alert the user if someone steps on it.

Power and batteries

⚠ WARNING

Risk of explosion if batteries are replaced with the wrong type or used batteries are disposed of in accordance with local and state regulations.

Batteries are toxic. They must not be swallowed! Keep out of reach of children and pets. If the batteries are swallowed, seek medical attention immediately!

- Rechargeable batteries should be fully charged before you use the device for the first time.
- The batteries are maintained charged as needed; when this happens, the segments in the battery icon will blink. Once they are fully charged, the battery icon disappears.
- When the "crossed-out battery" symbol appears on the display, contact your hearing center or dealer for further assistance.
- Please note that you may miss alarms and notifications if the batteries are drained.

When the battery level has dropped to the point where all three segments are off, the battery frame will blink to indicate that the batteries are almost drained. Insert the power adapter to prevent the clock from turning off.

When the "crossed-out battery" icon lights up, it may be due to the following battery issues:

- Batteries are missing
- Batteries are installed incorrectly
- Batteries are no longer accepting a charge
- Battery leakage is preventing contact
- Batteries are defective
- The wrong type of batteries has been installed

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Flashes can trigger epileptic seizures.
- Be aware that alarms and notifications can be missed if batteries run out.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Use only power adapters and battery types that are specified in this leaflet.
- Protect cables from any potential source of damage.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- Do not expose batteries to fire or to direct sunlight.
- Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35°C, 32°– 95° F.
- Rechargeable batteries should only be changed by Bellman & Symfon or an authorized service center. There is a risk of explosion if the batteries are not replaced correctly.

- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.
- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Compliance information

Hereby Bellman & Symfon declares that, in Europe, these products are in compliance with the essential requirements of the Medical Device Regulation (EU) 2017/745. In addition, BE1580-868 also complies with Radio Equipment Directive 2014/53/EU. The full text of the declaration of conformity can be obtained from the Bellman & Symfon website bellman.com.

Symbol explanation

	Indicates the item is a medical device.
	Indicates a carrier that contains unique device identifier information.
	Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.
	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	Indicates the medical device manufacturer.
	This symbol indicates the need for the user to consult the instructions for use.
	This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
	To identify a place where information may be found, especially in an emergency.
	Indicates the temperature limits to which the medical device can be safely exposed.
	Indicates the range of humidity to which the medical device can be safely exposed.

Technical specifications

Dimensions	121 × 108 × 92 mm, 4.8 × 4.3 × 3.6"
Weight	Visit/Pro model: 350 g, 12.4 oz. (Including batteries)
Mains power	7.5 V DC / 1.5 A
Battery	Battery backup: 4 × 1.2 V AAA NiMH rechargeable batteries Operating time: ~ 24 h when fully charged Charging time: ~ 10 h from fully depleted
Connections	3.5 mm stereo ext. trigger input RJ-11 telephone modular jack, 26 – 120 V RMS, 15 – 100 Hz
Sound	Raises slowly to 100 dBA @ 10 cm, 4"/ 94 dBA @ 30 cm, 12" Multiple frequency signal, 950 Hz – 3000 Hz
Flash	Four high-intensity pulsating LEDs Warning! Flashes can cause epileptic seizures
Bed Shaker	Vibrator power: 2.0 – 4.0 VDC
Radio frequency (Visit model only)	868.30 MHz
Operation	Temperature: 0° to 35° C, 32° to 95° F Humidity: 15% - 90%, non-condensing
Coverage (Visit model only)	50 – 250 m depending on the characteristics of the building. Coverage is reduced if walls and large objects block the signal.
Display	LCD display with adjustable backlight intensity in 5 steps
Night light	Type: Electro Luminescent light Timeout: 1 h on mains power, 15 min. on battery
Alarm and snooze	Alarm timeout: 20 min. Decreasing interval from 9 to 2 min.
Configuration	The Visit model can be used with all Visit transmitters
In the box (Visit model)	■ BE1580-868 Visit Alarm Clock ■ BE1270-P02 Bed Shaker ■ 4 × 1.2 V AAA NiMH batteries ■ 7.5 V DC / 1.5 A power adapter
Accessories (Visit model)	■ BE1270-P02 Bed Shaker ■ BE9105 Telephone Cable ■ BE9026-P02 Contact Mat
In the box (Pro model)	■ BE1370-C Alarm Clock Pro ■ BE1270-P02 Bed Shaker ■ 4 × 1.2 V AAA NiMH batteries ■ 7.5 V DC / 1.5 A power adapter
Accessories (Pro model)	■ BE1270-P02 Bed shaker ■ BE9105 Telephone Cable ■ BE9026-P02 Contact Mat

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All rights reserved.
Bellman & Symfon UK Limited
Audiocare AG
Bellman & Symfon Group AB
TM and © 2024 Bellman & Symfon AB.