# Visit Portable Receiver

BE1450-868







### **Features**

## Powerful sound

Adjustable up to 93 dB

### Clear signal identification

Uses LEDs and sound signatures

### Long lasting power

Up to 3 years battery life

### Bed shaker outlet

For nighttime alerts

# Easy setup

Works right out of the box



CE-marked medical device under EU MDR 2017/745

# Keeps you in the loop.

The Portable Receiver from Bellman & Symfon is part of the Visit system that helps you to keep tabs on your home. It alerts you with lights and sounds when the doorbell or telephone rings, when your baby needs you or if a fire should occur.


The portable receiver requires no installation and is very easy to use. It's completely wireless so you can bring it out on the porch or balcony and has a battery life of up to three years.

### **Clear and distinct alerts**

The portable receiver is capable of producing a sound signal of up to 93 dB and features a large, ergonomic dial that makes it easy to adjust the volume to your liking. It uses colored lights and individual sound signatures to differentiate the Visit signals.

Additionally, you can connect a bed shaker and wake up to vibrations if something should happen while you are asleep.



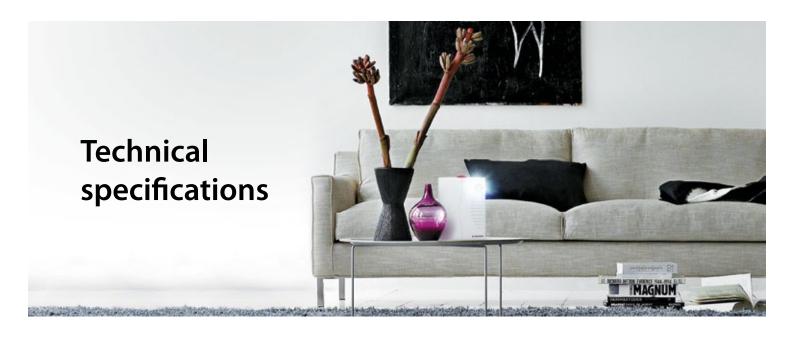

## **Dimensions and weight**

Height **165** mm, 6.5" Width **130** mm, 5.1" Depth **36** mm, 1.4"

Weight **590** g, 20.8 oz. (incl. battery)



# Features

- Adjustable sound signal
  Max 93 dBA @ 1 m, frequency range: 500-1000 Hz
- Clear signal identification
  Colored LEDs and individual sound signatures
- Long lasting power
  Up to 3 years battery life
- Bed shaker voltage: 2.0 4.0 VDC


# Frequency and coverage

- Radio frequency: 868.3 MHz,
- Coverage: Up to 250 meters, depending on the building's characteristics and the combination of transmitters and receivers.

### **Environmental requirements**

- Operating temperature: 0 to 35° C, 32 to 95° F
- Relative humidity: 15% to 90% (non condensing)

### **Buttons and controls**

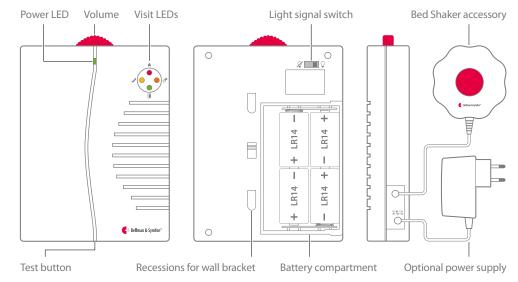

### **Power and battery**

- Battery power
  4×1.5V LR14 alkaline batteries
- Mains power
  7.5 V DC / 1500 mA with optional power supply
- Power consumption Active: 1000 mA, idle position: 0.1 mA
- Operating time
  2 3 years with alkaline batteries

#### In the box

- BE1450-868 Portable Receiver
- 4×1.5V LR14 batteries
- Wall bracket with screws and plugs

### Accessories

- BE1270-P02 Bed Shaker
- Power supply unit (EU)
- Power supply unit (UK)

### **Approvals**



This product is a medical device in conformity with Regulation (EU) 2017/745 (Medical Device Regulation).

#### **Visit indicators**

The Visit LEDs normally indicates the following:

- Orange LED, pacifier symbol
  The Baby Monitor is activated
- Green LED, door symbol
  The Door Transmitter is activated
- Yellow LED, telephone symbol
  The Telephone Transmitter is activated
- Red LED, fire symbol
  The Smoke Alarm is activated
- Orange and Red LEDs blink alternately The CO Alarm is activated

### © 2025 Bellman & Symfon AB. All rights reserved.

Bellman, Bellman & Symfon and the Bellman & Symfon logo are registered trademarks of Bellman & Symfon AB. This leaflet is subject to change without prior notice.

BE1450-868\_001DSH1.0\_EN

bellman.com