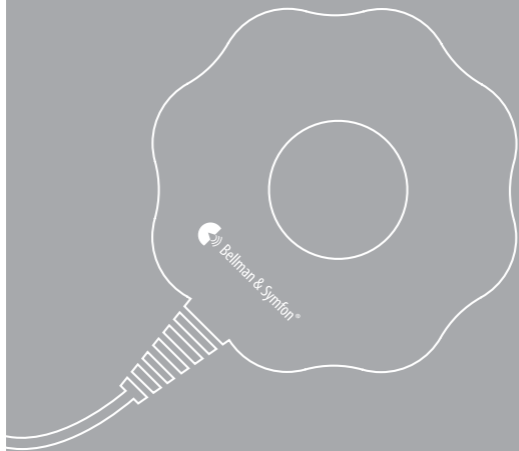
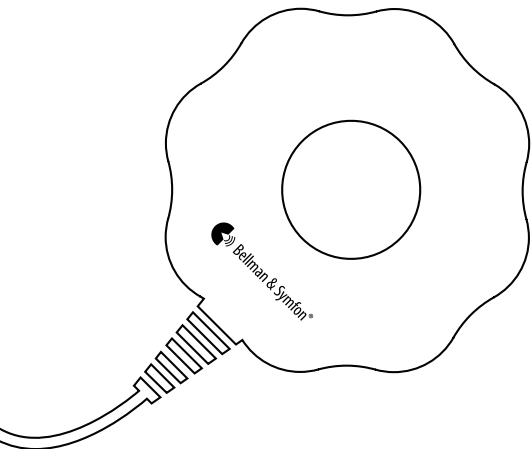
 Bellman & Symfon®

BE1270
Bed Shaker

BE1270 Bed Shaker

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JP

Read this first

This booklet guides you on how to use and maintain your new Bed Shaker. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

About the BE1270 Bed Shaker

Intended user group

The intended user group consists of people of all ages.

Intended user

The intended user is a person with a wish to receive augmented signals.

Principle of operation

Place the Bed Shaker under your mattress or pillow and plug it into any of the Bellman & Symfon receivers or chargers. If any of the Visit transmitters are activated, the Bed Shaker will wake the user with vibrations.

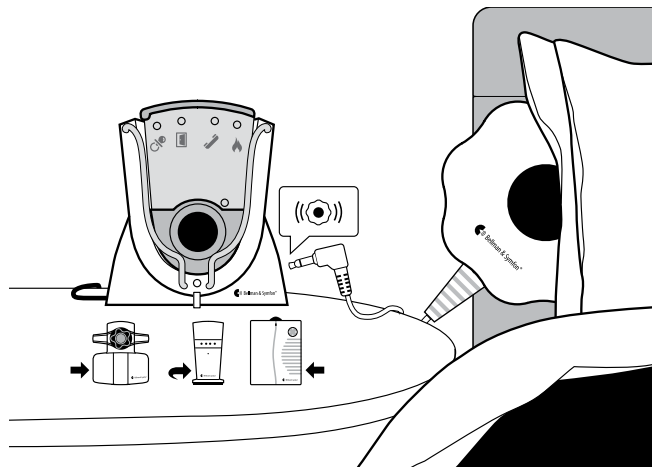
The Visit alerting system will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.

Getting started

The Bed Shaker can be used with all the Visit receivers and chargers, as a means of waking up with help of vibrations.

- 1 Connect the Bed Shaker by inserting the connector in the Bed Shaker jack on the receiver or charger. (See below)
- 2 Press the test button on any of your Visit transmitters and check that the Bed Shaker vibrates.

Note: When you connect the Bed Shaker to the pager charger, the pager must be inserted in order for it to work.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Protect the device from shocks during storage and transport.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Protect cables from any potential source of damage.

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- Do not drop your device. Dropping onto a hard surface can damage it.

- If you encounter problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit bellman.com for contact info.
- When you connect the Bed Shaker to the pager charger, the pager must be inserted in order for it to work.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product for six (6) months from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific

legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

Indicates the manufacturer's serial number so that a specific device can be identified.

Indicates the device manufacturer.

This symbol indicates the need for the user to consult the instructions for use.

This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

To identify a place where information may be found, especially in an emergency.

Indicates the temperature limits to which the device can be safely exposed.

Indicates the range of humidity to which the device can be safely exposed.

Technical specifications

Dimensions	88×88×27 mm, 3.5×3.5×1.1"
Weight	120 g, 4.2 oz
Operating voltage	2.0 - 4.0 V DC from a Visit receiver or charger
Power consumption	250 - 750 mA
Color	White with red rubber anti-skid protector
Cable length	2 meters
Function	The BE1270 Bed Shaker is a vibrator for indoor use that is placed in the bed to attract the attention of the user with vibrations.
Installation	The Bed Shaker is usually placed under a pillow or under the mattress. It is connected to a range of Bellman & Symfon products with vibrator outputs.
Operation	Temp: 0° to 35° C Humidity: 15% to 90%, non-condensing
Configuration	This device can be used with all Visit receivers.
Care and cleaning	Maintenance free. Clean with a soft cloth. Do not use cleaners, solvents, ammonia or alcohol. Not reusable.
In the box	■ BE1270 Bed Shaker

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FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by Bellman & Symfon Group AB could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

IC compliance statement

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: (1) This device may not cause interference.

(2) This device must accept any interference, including interference that may cause undesired operation of the device.

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Product disposal and recycling information

According to local laws and regulations, your product should be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The recycling of your product will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

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GB4943.1-2022

GB/T 9254.1-2021

GB17625.1-2022

Bellman & Symfon North America Inc.

PO Box 1388, Morrisville,
North Carolina 27560, USA

Phone: (910) 239-5796

Customer support: (833) BELLMAN
or (833) 235-5626

Bellman & Symfon Asia Ltd.

1401 & 1408, Laifung International Centre
33 Jie Fang Road South
Guangzhou, China
Phone: +86 20 8318 2950

Manufacturer

Bellman & Symfon Group AB

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436 32 Askim Sweden

Phone +46 31 68 28 20

E-mail info@bellman.com

bellman.com

Revision: BE1270_090MAN1.0

Date of issue: 2024-11-19

TM and © 2024 Bellman & Symfon AB. All
rights reserved.