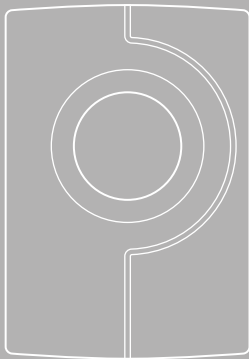
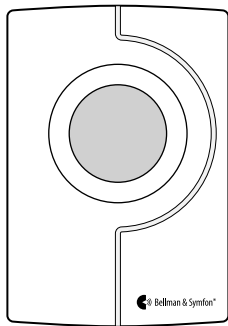
BE1420-868
Visit Pushbutton Transmitter

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EN
DE
DK
ES
FI
FR
IT
NL
NO
PL
SE
SK

Read this first

This booklet guides you on how to use and maintain your new Visit Pushbutton Transmitter. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your hearing care professional.

Intended purpose The intended purpose of the Visit alerting system is to alert deaf and hard of hearing people of important signals in their home.

Intended user A person with mild to severe hearing loss or deafness.

Intended user group The intended user group consists of people of all ages with mild to severe hearing loss or deafness who need audio, visual or sensory amplification.

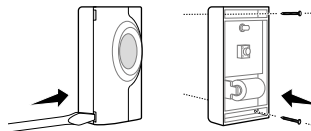
Principle of operation The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

Using it as a doorbell

Mounting with screws

- 1 Remove the transmitter front cover and make two holes on the markings.
- 2 Fix the unit to the wall using the supplied screws and put the cover back on.


Mounting with tape. You can instead choose to use the supplied adhesive tape. Attach it to the back of the transmitter and mount it in a weather protected area.

Using it as a caller button

The transmitter can be worn around your neck and be used as a wireless caller button.

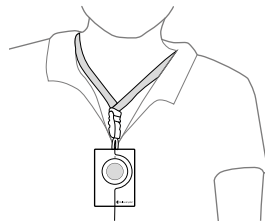Here is how you use it: Attach the lanyard to the top hole on the transmitter and hang it around your neck.


Testing the signal

Using the test buttons

- 1 Press the front button on the transmitter. The LED lights up in green to show that a radio signal is being transmitted.
- 2 The green Visit LED on the receiver lights up to show that the signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

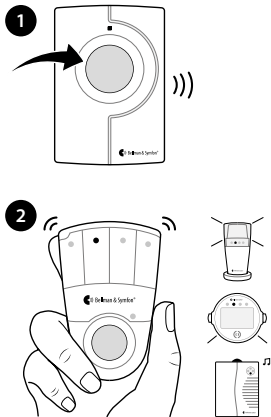Note: If you have changed the signal pattern, it will react in accordance with the signal tables that you can find on our website.

General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet should be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years.
- Note that alarms and notifications can be missed if batteries run out.
- Only use battery types that are specified in this leaflet.
- This device is for indoor use and outdoor use in a protected location. It will not withstand water or rain.
- Protect the device from shocks during storage and transport.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.

- This equipment radiates radio frequency energy. If not installed and used in accordance with the user guide, may cause harmful interference to radio communication.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.

- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes.

This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation

-  Indicates the item is a medical device.
-  Indicates a carrier that contains unique device identifier information.
-  Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.
-  Indicates the manufacturer's serial number so that a specific medical device can be identified.
-  Indicates the manufacturer's catalogue number so that the medical device can be identified.
-  Indicates the medical device manufacturer.

-  This symbol indicates the need for the user to consult the instructions for use.
-  This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
-  To identify a place where information may be found, especially in an emergency.
-  Indicates the temperature limits to which the medical device can be safely exposed.
-  Indicates the range of humidity to which the medical device can be safely exposed.

Troubleshooting

If	Try this
The LED lights up in yellow when I press the button	▪ The battery is nearly depleted. Replace with a new battery (see Technical specifications).
The LED doesn't light up when I press the button	▪ Check that the battery is positioned correctly. ▪ Replace with a new battery (see Technical specifications).
The LED lights up in green but the receiver is not activated	▪ Check the receiver batteries and connections. ▪ Move the receiver closer to the transmitter to make sure it's within radio range. ▪ Check that the units are set to the same radio key, see Changing the radio key on our website.
The receiver is activated for no apparent reason	▪ There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see Changing the radio key on our website.

Technical specifications

Dimensions	66 × 48 × 23 mm, 2.6 × 1.9 × 0.9"
Weight	50 g, 1.8 oz. including battery
Power and battery	Battery: 1 × 6V Alkaline PX28M/IEC-4LR44 or PX28L Lithium Power consumption: Active 35 mA, idle position 0.05 µA Operating time: Alkaline ~ 2 years, Lithium ~ 4 years
Max output power	-0.9dBm
Activation	Via the front button
Radio frequency	868.30MHz
Coverage	50-250 m (55-273 yd) depending on the characteristics of the building.
Operation	Temperature: -10° to 40° C, Humidity: 15% - 90%, non-condensing
Configuration	This transmitter can be used with all Visit receivers.
Number of radio keys	64 as standard. Up to 256 with optional software.
In the box	▪ BE1420-868 Visit Pushbutton Transmitter ▪ Lanyard with safety clasp ▪ Adhesive tape, screws and plugs

Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation (EU) 2017/745 and the Radio Equipment Directive 2014/53/EU. The full text of the declaration of conformity can be obtained from the Bellman & Symfon website **bellman.com**.

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All rights reserved.
Bellman & Symfon UK Limited
Audiocare AG
Bellman & Symfon Group AB
TM and © 2024 Bellman & Symfon AB.