
- EN** Medical device information
- DE** Informationen zu Medizinprodukten
- FI** Lääketieteellisiä laitteita koskevat tiedot
- NL** Informatie over medische hulpmiddelen
- NO** Informasjon om medisinsk utstyr
- SE** Information om medicinteknisk utrustning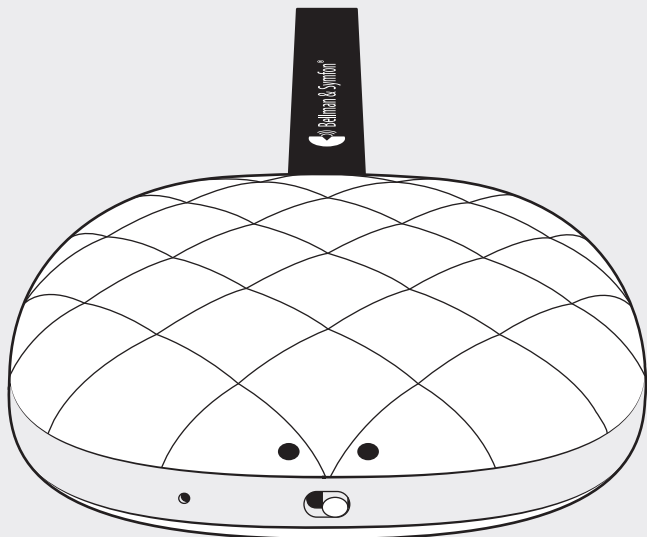

## Vibio bed shaker

- 
- 03 Medical device information
- 
- 07 Informationen zu Medizinprodukten
- 
- 11 Lääketieteellisiä laitteita koskevat tiedot
- 
- 15 Informatie over medische hulpmiddelen
- 
- 19 Informasjon om medisinsk utstyr
- 
- 23 Information om medicinteknisk utrustning
- 

EN

DE

FI

NL

NO

SE

## Read this first

Thank you for choosing a product from Bellman & Symfon – the world leader in alerting systems based in Gothenburg, Sweden. This leaflet contains important medical device information. Please read it carefully to make sure that you understand and get the best out of your Bellman & Symfon product. For more information about features and benefits, contact your hearing care professional.

### About BE1221 Vibio bed shaker

#### Intended purpose

The intended purpose of this wireless bed shaker is to awake deaf and hard of hearing people using vibrations. The bed shaker is to be used together with the **Vibio** mobile phone application.

#### Principle of operation

The Vibio bed shaker is typically placed under the pillow or mattress and the alarm is set in the Vibio mobile phone application. When the alarm goes off, the bed shaker vibrates under the pillow or mattress to awake the user. Once the alarm is set, the bed shaker works independently.

#### Intended user group

The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need sensory amplification.

#### Intended user

The intended user is a person with mild to severe hearing loss or deafness.

 The device will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.

## Important safety information

This section contains important information about safety, handling and operation. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

### Hazard warnings

 Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.

 Keep this device out of reach of children under 3 years.

 Be aware that alarms and notifications can be missed if the batteries run out.

 Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.

 Use only power adapters and battery types that are specified in this leaflet.

 Protect cables from any potential source of damage.

 Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.

 This device is designed for indoor use only. Do not expose the device to moisture.

 Protect the device from shocks during storage and transport.

 Only refer a damaged or faulty product to qualified service personnel.

## Information on product safety

 Failure to follow these instructions could result in damage to the device and void the warranty.

 Do not expose batteries to fire or to direct sunlight.

 Do not drop your device. Dropping onto a hard surface can damage it

 Do not use the device in areas where electronic equipment is prohibited.

 If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.

 If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit [bellman.com](http://bellman.com) for contact information.

 Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35°C, 32°– 95° F.

 Rechargeable batteries should only be changed by Bellman & Symfon or an authorized service center. There is a risk of explosion if the batteries are not replaced correctly.

 This device should not be used on aircrafts unless specifically permitted by flight personnel.

## Configuration options

This bed shaker can be used with the following mobile phone models and operating systems:

- iPhone 6s and later or Modern Android devices
- iOS11 or Android 4.4 and later.

## Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

## Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

## Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

## Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

## Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below.

The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit [bellman.com](http://bellman.com) for contact information.

- Medical Device Regulation (MDR)
- Radio Equipment Directive (RED)
- EMC directive
- RoHS Directive
- REACH Regulation
- EC General Product Safety Directive
- Waste Electrical & Electronic Equipment (WEEE)
- EC Battery Directive

## ISO Certification of legal manufacturer

Bellman is certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.

SS-EN ISO 9001 Certification Number: CN19/42071  
SS-EN ISO 13485 Certification Number: CN19/42070

### Certification Body

SGS United Kingdom Ltd, Rossmore Business Park  
Ellesmere Port Cheshire CH65 3EN UK

## Technical specifications

### Dimensions and weight

Size: 27 × 94 × 94 mm, 1.1 × 3.7 × 3.7"  
Weight: 153 g, 5.4 oz.

### Power and Battery

Battery power: 1 × 3.7 V AA Lithium-ion  
Power consumption: < 1000 mA, Idle: < 100 µA  
Operating time: Up to 10 days / charge  
Charging time: Up to 1.5 hour @ 500mA  
Charging current: Up to 500 mA, 5V DC

### Cables and connections

Cable length: 1 m, 39"  
Contacts: USB type A to USB Micro-B

### Compatibility

iOS11 / Android 4.4 and later,  
iPhone 6s and later / Modern Android devices

### Frequency and coverage

Bluetooth frequency: 2402 – 2480 MHz  
Output power: Max 2.5mW (4dBm)  
Coverage: Up to 30 m, 33 yd, free field.  
Connectivity: Bluetooth® 5

### In the box

- Vibio bed shaker
- USB charging cable

## Regulatory symbols

 With this symbol, Bellman & Symfon confirms that the product meets the Medical Device Regulation EU 2017/745.

 This symbol indicates the manufacturer's serial number so that a specific medical device can be identified. It's available on the product and gift box.

 This symbol indicates the manufacturer's catalogue number so that the medical device can be identified. It's available on the product and gift box.

 This symbol indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.

 This symbol indicates that the user should consult the instructions and this leaflet.

 This symbol indicates that it is important for the user to pay attention to the relevant warning notices in the user guides.

 This symbol indicates important information for handling and product safety.

 Temperature during transport and storage: -10° to 50° C, 14° – 122° F  
Temperature during operation: 0° to 35° C, 32° to 95° F

 Humidity during transportation and storage: <90%, non-condensing  
Humidity during operation: 15% – 90%, non-condensing

 Atmospheric pressure during operation, transportation and storage: 700 hpa to 1060 hpa

**Operating conditions** This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in the user guide or this leaflet.

 With this CE symbol, Bellman & Symfon confirms that the product meets EU standards for health, safety, and environmental protection as well as the Radio Equipment Directive 2014/53/EU.

 This symbol indicates that the product shall not be treated as household waste. Please hand over your old or unused product to the applicable collection point for the recycling of electrical and electronic equipment or bring your old product to your hearing care professional for appropriate disposal. By ensuring this product is disposed of correctly, you will help prevent potential negative effects on the environment and human health.

 The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Bellman & Symfon is under license. Other trademarks and trade names are those of their respective owners.

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TM and © 2020 Bellman & Symfon AB. All rights reserved.
Bellman & Symfon Group AB