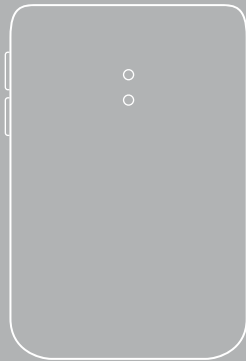
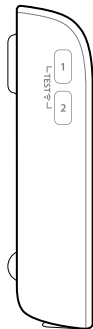
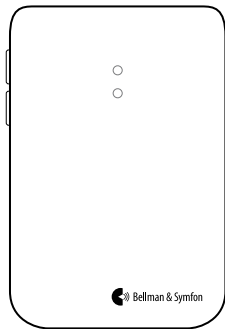
BE1431 Visit telephone transmitter

---

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---

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---

**36** User manual

---

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---

**68** Käyttöopas

---

**84** Mode d'emploi

---

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---

**116** Brukermanual

---

**132** Bruksanvisning

---

**149** Appendix (EN)

---

DE

DK

EN

ES

FI

FR

NL

NO

SE

App

## Les dette først

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### Sertifiseringsorgan

SGS United Kingdom Ltd, Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

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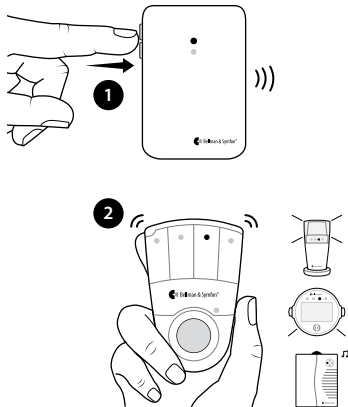
## Testing the signal

### Using the test buttons

- 1 Press both test buttons simultaneously on the transmitter. The top LED lights up in green to show that a radio signal is being transmitted.
- 2 The yellow Visit LED on the receiver lights up to show that the signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

**Note:** If you have changed the signal pattern, it will react in accordance with the table on the following spread.


## Testing the signal

### Using a mobile phone

- 1 Use for instance a mobile phone to call the landline telephone. The top LED on the transmitter lights up in green to show that an incoming call is detected.
- 2 The yellow Visit LED on the receiver lights up to show that the radio signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

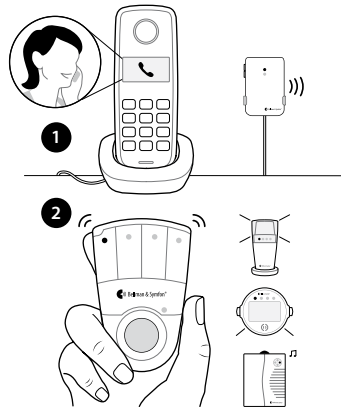**Note:** If you have changed the signal pattern, it will react in accordance with the table on the following spread.

## Default signal pattern

When the telephone transmitter is activated by an incoming call or a triggered accessory, the following happens:

- 1 The LED on the transmitter lights up in green to show that it is signalling the receiver.
- 2 The Visit LED on the receiver lights up in yellow and it starts to sound, flash or vibrate with a certain pace, called signal pattern. The transmitter and the connected accessories determine the signal pattern. The default is as follows:

Transmitter		Visit receiver			
Source	LED	LED	Sound	Vibration	Flash
Landline phone	Green, top	Yellow light	Ring signal, low	Medium	Yes
Accessory	Green, bottom	Green light	Door chime, low	Slow	Yes

## Changing the signal pattern

The transmitter controls the signal pattern. Open the transmitter front cover and move the signal switches according to the tables to change it:

### Transmitter

Switch

Source

Landline/test button  
Mobile phone sensor  
Other accessory

Landline/test button  
Mobile phone sensor  
Other accessory

Landline/test button  
Mobile phone sensor  
Other accessory

Landline/test button  
Mobile phone sensor  
Other accessory

### Receiver signal pattern

LED

Yellow light  
Yellow blinks  
Green light

Yellow light  
Yellow blinks  
2 x green blinks

Yellow light  
Yellow blinks  
3 x yellow blinks

Yellow light  
Yellow blinks  
2 x orange blinks

Sound

1 x ring signal, low  
2 x ring signal, high  
1 x door chime, low

1 x ring signal, low  
2 x ring signal, high  
2 x door chime, low

1 x ring signal, low  
2 x ring signal, high  
1 x ring signal, high

1 x ring signal, low  
2 x ring signal, high  
Baby melody

Vibration

Medium ■□■□  
Medium ■□■□  
Slow ■□□□

Medium ■□■□  
Medium ■□■□  
Slow ■□□□

Medium ■□■□  
Medium ■□■□  
Medium ■□■□

Medium ■□■□  
Medium ■□■□  
Fast ■□■□■□

Flash

Yes  
Yes  
Yes

Yes  
Yes  
Yes

Yes  
Yes  
Yes

Yes  
Yes  
Yes

## Transmitter

## Receiver signal pattern

Switch	Source	LED	Sound	Vibration	Flash
	Landline/test button	2 x yellow blinks	2 x ring signal, low	Medium ■□■□	Yes
	Mobile phone sensor	Yellow light	1 x ring signal, low	Medium ■□■□	Yes
	Other accessory	3 x orange blinks	Baby melody	Fast ■■■■■■	Yes
	Landline/test button	2 x yellow blinks	2 x ring signal, low	Medium ■□■□	Yes
	Mobile phone sensor	Orange blinks	Baby melody	Fast ■■■■■■	Yes
	Other accessory	Green blinks	2 x door chime, high	Slow ■□□□	Yes
	Landline/test button	Orange blinks	Baby melody	Fast ■■■■■■	Yes
	Mobile phone sensor	3 x yellow blinks	1 x ring signal, high	Medium ■□■□	Yes
	Other accessory	2 x green blinks	2 x door chime, low	Slow ■□□□	Yes
	Landline/test button	3 x yellow blinks	1 x ring signal, high	Medium ■□■□	Yes
	Mobile phone sensor	2 x yellow blinks	2 x ring signal, low	Medium ■□■□	Yes
	Other accessory	Green blinks	2 x door chime, high	Slow ■□□□	Yes

## Transmitter

## Receiver signal pattern

Switch	Source	LED	Sound	Vibration	Flash
	Landline/test button	Yellow blinks	2 x ring signal, high	Medium ■□■□	Yes
	Mobile phone sensor	Yellow light	1 x ring signal, low	Medium ■□■□	Yes
	Other accessory	Green light	1 x door chime, low	Slow ■□□□	Yes
	Landline/test button	Yellow blinks	2 x ring signal, high	Medium ■□■□	Yes
	Mobile phone sensor	Yellow light	1 x ring signal, low	Medium ■□■□	Yes
	Other accessory	2 x green blinks	2 x door chime, low	Slow ■□□□	Yes
	Landline/test button	Yellow blinks	2 x ring signal, high	Medium ■□■□	Yes
	Mobile phone sensor	Yellow light	1 x ring signal, low	Medium ■□■□	Yes
	Other accessory	3 x yellow blinks	1 x ring signal, high	Medium ■□■□	Yes
	Landline/test button	Yellow blinks	2 x ring signal, high	Medium ■□■□	Yes
	Mobile phone sensor	Yellow light	1 x ring signal, low	Medium ■□■□	Yes
	Other accessory	2 x orange blinks	Baby melody	Fast ■■■■■■	Yes

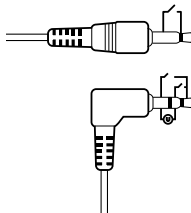
## Transmitter

## Receiver signal pattern

Switch	Source	LED	Sound	Vibration	Flash
	Landline/test button	3 x yellow blinks	1 x ring signal, high	Medium ■□■□	Yes
	Mobile phone sensor	Yellow light	1 x ring signal, low	Medium ■□■□	Yes
	Other accessory	3 x orange blinks	Baby melody	Fast ■■□■□■	Yes
	Landline/test button	Orange blinks	Baby melody	Fast ■■□■□■	Yes
	Mobile phone sensor	3 x yellow blinks	1 x ring signal, high	Medium ■□■□	Yes
	Other accessory	Green blinks	2 x door chime, high	Slow ■□□□	Yes
	Landline/test button	Orange/Red blink	VMA	Long ■■□□	Yes
	Mobile phone sensor	Orange/Red blink	VMA	Long ■■□□	Yes
	Other accessory	Orange/Red blink	VMA	Long ■■□□	Yes
	Landline/test button	Red blinks	Fire alert	Long ■■□□	Yes
	Mobile phone sensor	Red blinks	Fire alert	Long ■■□□	Yes
	Other accessory	Red blinks	Fire alert	Long ■■□□	Yes

## Trigger input

You can use both stereo and mono plugs. The 3.5 mm input is located on the top right side and the 2.5 mm input under the lid.

### Mono

Trig is made by closing the two connectors

### Stereo

Trig is made by:

- 1 Closing the outer and middle connector or closing the outer and inner connector
- 2 Applying 2-30VDC or 3-24VAC 5-150Hz between the middle and inner connector

Please note that when you use the 2.5 mm ext trig, the event must last for more than 2 seconds

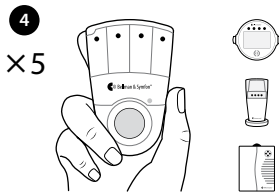
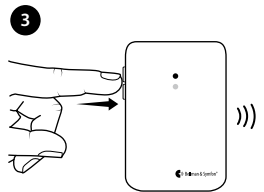
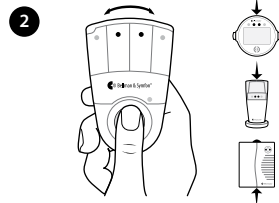
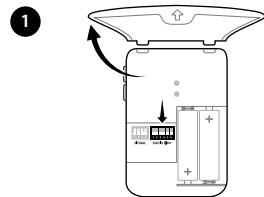
## Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located under the transmitter cover.

### Here is how you change the radio key:

- 1 Carefully open the front cover on the transmitter. Move any radio key switch to the up (on) position to change the radio key.
- 2 Press and hold the test/function button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press both test buttons simultaneously on the transmitter within 30 s to send the new radio key.
- 4 All Visit LEDs on the receiver blink 5 times to show that the radio key has been changed. It then returns to normal mode.

Please note that all Visit units must be set to the same radio key in order to operate as a group.

## Manufacturer

Bellman & Symfon Group AB  
Södra Långebergsgatan 30  
436 32 Askim Sweden  
Phone +46 31 68 28 20  
E-mail [info@bellman.com](mailto:info@bellman.com)  
[bellman.com](http://bellman.com)

Revision: BE1431\_144MAN1.0

Date of issue: 2022-08-25

TM and © 2022 Bellman & Symfon AB.

All rights reserved.