BE9122-P02 Headphones

EN	Product information
DE	Produktinformationen
ES	Información del producto
FI	Tuotetiedot
FR	Informations sur le produit
IT	Informazioni sul prodotto
NO	Produktinformasjon
PL	Informacje o produkcie
SE	Produktinformation
SK	Informácie o výrobku

Product information

Read this first

This leaflet guides you on how to use and maintain your new Headphones. Ensure you read this booklet carefully, including the Warnings section. If you have additional questions, contact your hearing care professional.

About the BE9122-P02 Headphones

Intended purpose
The intended purpose of the audio product family is to amplify the volume and improve the speech intelligibility during conversations and TV listening. It can also be used with other sound sources.

Intended user group
The intended user group consists of people of all ages experiencing mild to severe hearing loss who need sound amplification in different situations.

Intended user
The intended user is a person with mild to severe hearing loss who needs sound amplification.

Principle of operation
The audio product family consists of several amplifiers and sound transmitters which have been specifically developed to provide sound enhancement even in demanding situations. Depending on the assigned function of the specific amplifier or sound transmitter, different microphones can be used to pick up direct sound or enhance the ambient sound.

⚠ This device will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.

Important safety information

This section contains important information about safety, handling and operating conditions. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

⚠ Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Protect cables from any potential source of damage.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.

- The connected listening device has a very powerful amplifier and the sound level can cause discomfort or, in serious cases, endanger hearing if the product is used carelessly. Always turn the volume down to a low level before putting on the headphones!

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit bellman.com for contact info.
- Do not drop your device. Dropping onto a hard surface can damage it.
- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.

Operating conditions

Operate the device in a dry environment within the temperature and humidity range stated in this leaflet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product for 90 days from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes.

This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied

warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Symbol explanation	
MD	Indicates the item is a medical device.
UDI	Indicates a carrier that contains unique device identifier information.
	Indicates that the product should not be discarded as unsorted waste but must be sent to separate collection facilities for recovery and recycling.
SN	Indicates the manufacturer's serial number so that a specific medical device can be identified.
REF	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	Indicates the medical device manufacturer.

ⓘ This symbol indicates the need for the user to consult the instructions for use.

⚠ This symbol indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

📍 To identify a place where information may be found, especially in an emergency.

🔧 Indicates the temperature limits to which the medical device can be safely exposed.

📏 Indicates the range of humidity to which the medical device can be safely exposed.

Compliance information

Herby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation (EU) 2017/745. The full text of the declaration of conformity can be obtained from the Bellman & Symfon website bellman.com.

Technical specifications

Weight: 57 g
Frequency range: 20 Hz – 20 kHz
Impedance: 32 Ω +/- 10%
Sensitivity: 109 dB/mW @ 1 kHz
Colour: Black
Cable length: 1.25 meters
Connector: 3.5 mm stereo plug
Operation: Temp: 0° to 35° C

Accessories: BE9150 Headphone covers
In the box: BE9122-P02 Headphones

Configuration options

These headphones can be configured with all of Bellman & Symfon's personal amplifiers and listening systems.

ⓘ For detailed product information, see the corresponding user manual.

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