
BE1441/BE1442
Visit Flash Receiver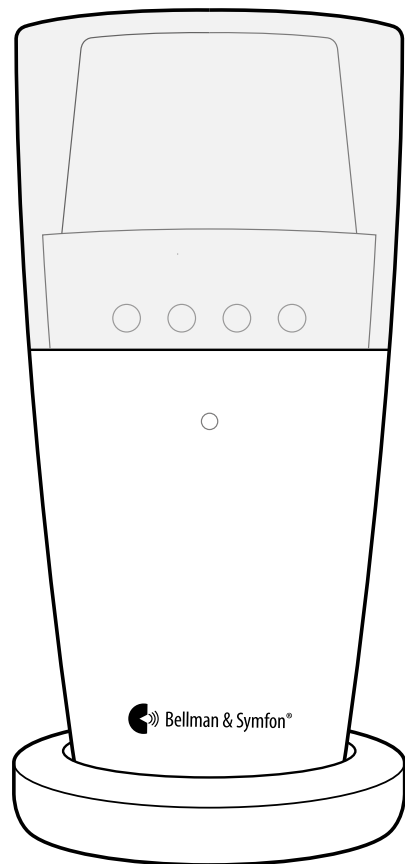

BE1441/BE1442 Visit flash receiver

4	Bedienungsanleitung
12	Brugervejledning
20	User manual
28	Manual de usuario
36	Käyttöopas
44	Mode d'emploi
52	Gebruiksaanwijzing
60	Brukermanual
68	Bruksanvisning
76	Appendix (EN)

DE
DK
EN
ES
FI
FR
NL
NO
SE
App

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
Read this first

This booklet guides you on how to use and maintain your new Visit flash receiver. Ensure you read this booklet carefully, including the **Warnings** section. This will help you get the most out of your product. If you have additional questions, contact your hearing care professional.


- Intended purpose** The intended purpose of the Visit system is to alert deaf and hard of hearing people of important signals in their home.
- Intended user** The intended user is a person with mild to severe hearing loss who needs sound amplification.
- Intended user group** The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.
- Principle of operation** The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

Overview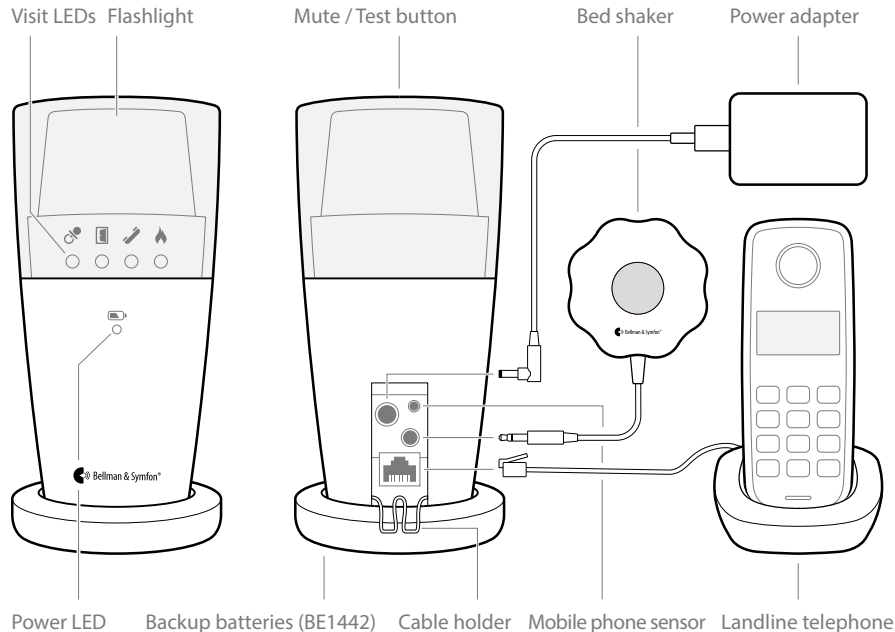

Getting started

- 1 Connect the power supply to the receiver and the mains outlet. Pull the battery tab on the bottom (BE1442 only). Place the receiver on a level surface or mount it on the wall using the wall bracket accessory (see separate instructions).
- 2 To test the radio link you need a Visit transmitter. Press the test button/s on the transmitter.
- 3 The receiver lights up a Visit LED and starts to flash. If a bed shaker is connected, it will vibrate. A short press on the mute/test button repeats the last indication. If nothing happens, see **Troubleshooting**.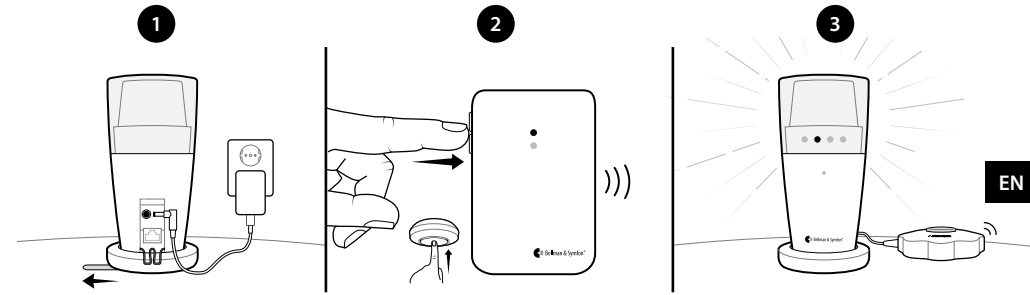

Default signal pattern

When a transmitter is activated, the flash receiver lights up an LED, flashes and the bed shaker starts to vibrate with a certain pace. This is called signal pattern. The transmitters determine the pattern, and the default is as follows:

Transmitter	Flash receiver		Bed shaker
Activated source	Visit LED	Flash	Vibration
▪ Door transmitter	Green	Yes	Slow
▪ Push button transmitter	Green	Yes	Slow
▪ Telephone transmitter	Yellow	Yes	Medium
▪ Baby monitor	Orange	Yes	Fast
▪ Smoke alarm	Red	Yes	Long
▪ CO alarm	Orange/red	Yes	Long

Changing the signal pattern

The signal pattern can only be changed on the transmitters. See **Changing the signal pattern** for the relevant transmitter.

Power LED indications

When the flash receiver is connected to mains power, the power LED lights up in green. The BE1442 model is also equipped with battery backup and the power LED indicates the following:

Power LED	Status
▪ Green light	The flash receiver is connected to mains power. The backup batteries are detected.
▪ Green blinks	The flash receiver is connected to mains power. No backup batteries are detected.
▪ Red light	The flash receiver is running on battery backup.
▪ Red blinks	The backup batteries are nearly depleted.

Personalizing your receiver


Directing the flash

The flash receiver features a rotating top that makes it easy to direct the light. Point it for example towards a wall if you feel that the flash is too intense. A silicone slip-on top is also available in a variety of colors (art. no. BE9164-BE9167).

Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located on the **transmitters**.

- 1 Open the transmitter cover and move any radio key switch to the up (on position) to change the radio key. See **Changing the radio key** for the relevant transmitter.
- 2 Press and hold the test button located on the top of the flash receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press the test button/s on the transmitter within 30 seconds to send the new radio key.
- 4 All Visit LEDs on the receiver blink 5 times to show that the radio key has been changed. Please note that all Visit units must be set to the same radio key.

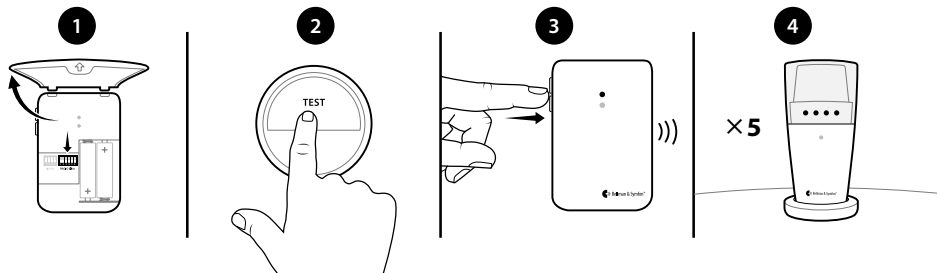General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet must be given to the householder.

⚠ Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Keep this device out of reach of children under 3 years.
- Be aware that alarms and notifications can be missed if batteries run out.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Flashes can trigger epileptic seizures for people with photosensitive epilepsy (PSE).
- Use only power adapters and battery types that are specified in this booklet.
- Protect cables from any potential source of damage.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This device is designed for indoor use only. Do not expose the device to moisture.
- Protect the device from shocks during storage and transport.
- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

ⓘ Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35°C, 32°– 95° F.
- Rechargeable batteries should only be changed by Bellman & Symfon or an authorized service center. There is a risk of explosion if the batteries are not replaced correctly.
- Do not drop your device. Dropping onto a hard surface can damage it
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit bellman.com for contact info.
- The programming pin on the back of the flash receiver is for authorized service personnel only. Unauthorized use may damage the device.
- Without permission granted by the NCC, no company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this booklet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship.

This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes.

This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Configuration options

This BE1441 and BE1442 Visit flash receiver can be configured with the following Visit transmitters and Bellman & Symfon original accessories:

Compatible Visit transmitters

- BE1411 Visit door transmitter
- BE1420 Visit pushbutton transmitter
- BE1431 Visit telephone transmitter
- BE1433 Visit mobile phone transceiver
- BE1481 Visit smoke alarm transmitter
- BE1551 Visit smoke alarm transmitter
- BE1555 Visit CO alarm transmitter
- BE1491 Visit baby monitor

Available accessories

- BE9075 Wall mount bracket
- BE1270 Bed shaker
- BE9105 Telephone cable
- BE9250 Mobile phone sensor

 For detailed product information, see the corresponding user manual.

ISO Certification of legal manufacturer

Bellman is certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.

SS-EN ISO 9001 Certification Number: CN19/42071

SS-EN ISO 13485 Certification Number: CN19/42070

Certification Body

SGS United Kingdom Ltd

Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

Regulatory symbols

With this symbol, Bellman & Symfon confirms that the product meets the Medical Device Regulation EU 2017/745.

This symbol indicates the manufacturer's serial number so that a specific medical device can be identified. It's available on the product and gift box.

This symbol indicates the manufacturer's catalogue number so that the medical device can be identified. It's available on the product and gift box.

This symbol indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.

This symbol indicates that the user should consult the instructions for use.

This symbol indicates that it is important for the user to pay attention to the relevant warning notices in the user guide.

This symbol indicates important information for handling and product safety.

Temperature during transport and storage: -10° to 50° C, 14° to 122° F
Temperature during operation: 0° to 35° C, 32° to 95° F

Humidity during transportation and storage: <90%, non-condensing
Humidity during operation: 15% to 90%, non-condensing

Atmospheric pressure during operation, transportation and storage: 700hpa to 1060hpa

This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in the instructions for use.

With this CE symbol, Bellman & Symfon confirms that the product meets EU standards for health, safety, and environmental protection as well as the Radio Equipment Directive 2014/53/EU.

This symbol indicates that the product shall not be treated as household waste. Please hand over your old or unused product to the applicable collection point for the recycling of electrical and electronic equipment or bring your old product to your hearing care professional for appropriate disposal. By ensuring this product is disposed of correctly, you will help prevent potential negative effects on the environment and human health.

Technical specifications

Dimensions	BE1441: 70 × 140 mm, 2.7 × 5.5" (Ø × H) BE1442: 78 × 155 mm, 3.1 × 6.1" (Ø × H)
Weight	308 g, 10.8 oz. (both models)
Mains power	7.5 V DC / 1500 mA Only use the supplied power adapter. This product is also designed for IT power distribution systems with phase to phase voltage of up to 230V. The outlet shall be installed near the equipment and shall be easily accessible.
Power consumption	Active: 750 mA, idle position: 8 mA
BE1442 battery backup	Battery type: 4 × 1.2 V Lixel 600/900 mAh Operating time: ~ 48 h when fully charged
Connections	<ul style="list-style-type: none"> 2.5 mm external trigger jack 3.5 mm bed shaker outlet, vibration power: 2.0 – 4.0 VDC (max. 500 mA) RJ-11 telephone modular jack, 26 – 120 V RMS, 15 – 100 Hz Barrel jack for the external power adapter Programming pin for service purposes. Output: 3.3 V/max. 150mA.
Flash	Built-in ~30 Candela Xenon light Warning! Flashes can cause epileptic seizures
Radio frequency	868.30 MHz
Coverage	50 – 250 m, 55 – 273 yards depending on the characteristics of the building.
In the box	<ul style="list-style-type: none"> BE1441 Visit flash receiver or BE1442 Visit flash receiver with battery backup 7.5 V DC / 1500 mA power adapter 4 × 1.2 V AAA NiMH batteries (BE1442 model only)

Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below. The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit bellman.com for contact information.

Radio Equipment Directive (RED)	REACH Regulation
Medical Device Regulation (MDR)	EC Battery Directive
Restriction of Hazardous Substances Directive	Waste Electrical & Electronic Equipment, WEEE

Troubleshooting

If	Try this
The receiver seems to be turned off	<ul style="list-style-type: none"> Check that the power supply is connected correctly. Charge the backup batteries for at least 24 hours (BE1442 only).
The power LED blinks in red	<ul style="list-style-type: none"> The backup batteries are nearly depleted and the power supply is disconnected. Connect the power supply and charge the batteries for at least 24 hours.
The power LED blinks in green	<ul style="list-style-type: none"> The receiver detects no backup batteries. Pull the battery tab, see Getting started.
The receiver does not respond when a transmitter is activated, but works when I use the test button	<ul style="list-style-type: none"> Check the transmitter batteries and connections. Move it closer to the transmitter to make sure it's within range. Check that the receiver is set to the same radio key as the other units in the Visit system, see Changing the radio key.
The receiver is activated for no apparent reason	<ul style="list-style-type: none"> There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see Changing the radio key.
The flash is too bright	<ul style="list-style-type: none"> Redirect the light by rotating the top or use a silicone slip-on top accessory to dim the light.

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
 - 79 Deleting advanced programming
-

Testing the connection


A Visit transmitter set to the same radio key as the flash receiver is required to test the system. You will need a bed shaker accessory to test the vibration.

- 1 Press the transmitter test button(s).
- 2 The flash receiver will:
 - Start to flash
 - Light up the corresponding Visit LED
 - If a bed shaker accessory is connected, it will start to vibrate.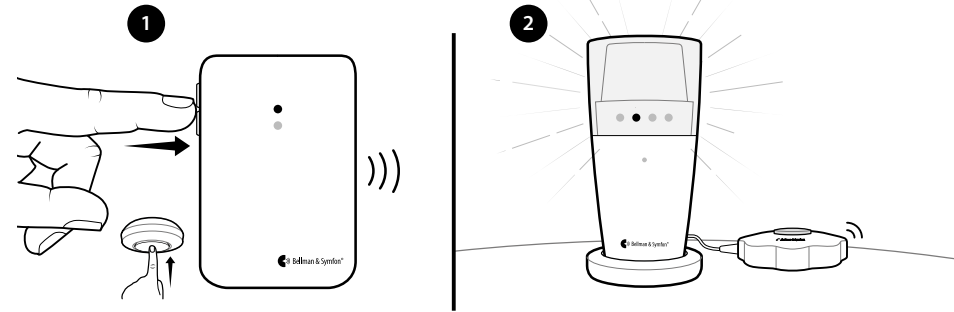

Testing the landline telephone


- 1 Connect the flash receiver to an analogue telephone socket via the telephone Input. You will need a telephone flex cable accessory and an adapter plug accessory.
- 2 Use for instance your mobile phone to call your landline telephone. The flash receiver will start to flash and light up the yellow Visit LED. If a bed shaker accessory is connected, it will start to vibrate. To repeat the last indication, press the flash receiver test button.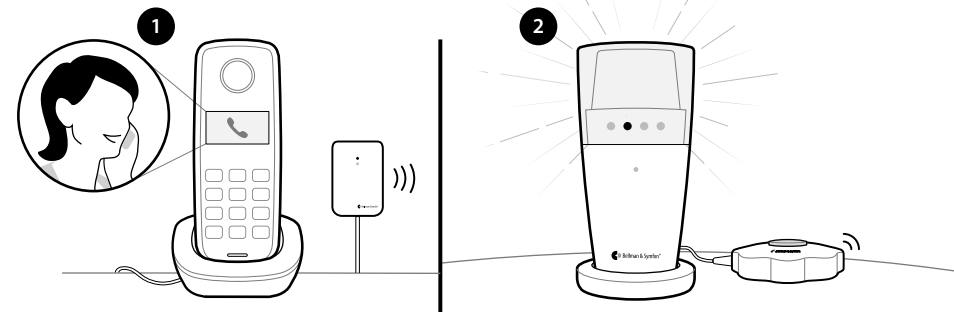

Battery backup


BE1442

Advanced programming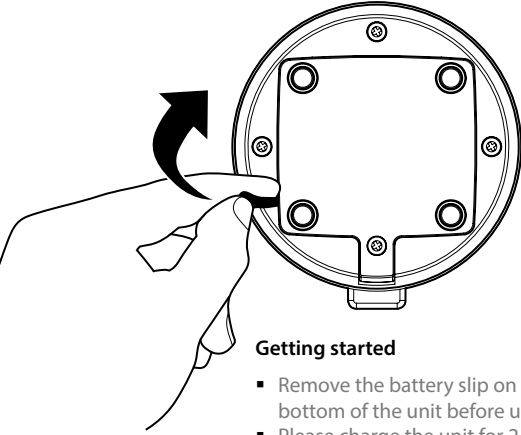

Getting started

- Remove the battery slip on the bottom of the unit before use.
- Please charge the unit for 24 hours to ensure full backup time.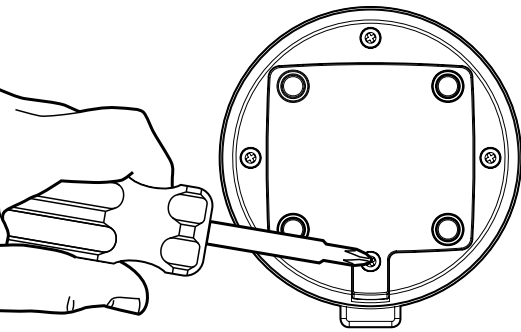


Replacing batteries (Done by the operator only)

- Remove the screw on the bottom of the unit to exchange the batteries.

⚠ Caution

Risk of explosion if battery is replaced by an incorrect type.

Dispose of used batteries according to the instructions in this user manual.

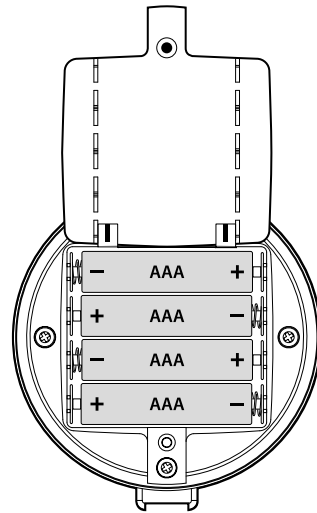- Use only 4x1.2VNiMH 600AAA/900AAA Lexel rechargeable batteries to ensure full backup time.

By using advanced programming, you can customize the signal pattern from a specific transmitter and event, displaying the LED colour and vibration pattern of your choice. The advanced programming overrides the radio key and pairs the units via the serial number.

ⓘ Please note

- Smoke- and CO alarms cannot be programmed for safety reasons.
- The transmitter must be activated as it is intended to be used in the system to generate the right signal. This means that you can't always use the transmitter test button (see **Default signal pattern** for the relevant transmitter).

Here is how you program the receiver:

- 1 Press and hold the mute/test button on the receiver. The green and yellow Visit LEDs will start to blink alternately. While still holding down the button, activate the desired transmitter as intended. Release the button.
- 2 Scroll through the different **Visit LED options** by pressing the mute/test button on the receiver. Select the desired Visit LED colour by holding down the mute/test button until the power LED goes out and lights up again.
- 3 Scroll through the different **vibration options** by pressing the test button on the receiver (bed shaker required). Select the desired vibration pattern by holding down the mute/test button until the power LED goes out and lights up again.
- 4 The receiver will now show the new Visit LED colour and vibration pattern. Press the mute/test button briefly to end the demonstration. After a short while, it will return to normal mode.

Deleting the advanced programming

Follow the procedure below to delete the advanced programming.

- 1 Hold down the mute/test button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 2 Press the mute/test button on the receiver 3 times in quick succession.
- 3 All Visit LEDs will light up for ~2 seconds to show that it has been deleted.

DESIGN FOR EARS™

Manufacturer

Bellman & Symfon Group AB
Södra Långebergsgatan 30
436 32 Askim Sweden
Phone +46 31 68 28 20
E-mail info@bellman.com
bellman.com

Revision: BE1441_181MAN1.0
Date of issue: 2022-08-29
TM and © 2022 Bellman & Symfon AB.
All rights reserved.